ADHESOR™ CARBOFINE Zinc Polycarboxylate cement

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ADHESOR™ CARBOFINE Cemento de policarboxilato de zinc

Specification:
ADHESOR CARBOFINE is a non-irritant zinc polycarboxylate dental cement which does not attack the pulp, provides natural adhesion to hard dental tissues and is X-ray-contrastive.

Indications:

- Liners under amalgam and composite fillings. The application is especially suitable for covering the applied CALXYD. ADHESOR CARBOFINE can also be used when the cavity cannot be sufficiently dried, as it provides very good adhesion even in a wet environment.
- Temporary filling for a period of up to 6 months after the application.
- Cementing of smaller fixed dentures (1-2 crowns).
- For cementation of meniscus fixings (maxillary 1/2 arches).

Contra-indications:
The application of ADHESOR CARBOFINE is contra-indicated in the case of a known hypersensitivity for any component of the product.

Working:
The recommended mixing ratio for liners and temporary fillings is 3 g of powder to 1 g of liquid, i.e. 1 levelled scoops of powder to 5 drops of liquid. For cementing jobs, the recommended mixing ratio is 1.8 - 2.2 g of powder to 1 g of liquid, i.e. 2 levelled scoops of powder to 5 drops of liquid. The powder is to be mixed into the liquid in two doses during a 30 second period. The working time is approx. 2 minutes at 23 °C and 60% relative humidity. The setting time is 5 - 8 minutes. When keeping to the prescribed weight ratios, the prepared material must have a glossy surface, otherwise it loses its qualitative properties.

Warning:
Prášek: H410 Velmi toxický pro vodní organismy, s dlouhodobými účinky. P273 AVOID release to the environment.
Liquid: H319 Causes serious eye irritation. H315 Causes skin irritation. H335 May cause respiratory irritation.
P262 Do not get in eyes, on skin, or on clothing. P270 Do not eat, drink or smoke when using this product.

Waste material disposal:
The residue of unused product and contaminated packaging material should be transferred to a company authorized to dispose of dangerous waste materials or dispose of it in an incineration plant for dangerous waste.

Storage:
Store in a dry and dark place at 5 - 25 °C, in a well sealed container.

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