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ELASTIC™ CROMO

Alginate Impression Material

DEVICE DESCRIPTION
Elastic Cromo is an alginate impression material suitable for all preliminary impressions. It features a color-change property for timely procedure control, dimension stability and precise detail reproduction. Elastic Cromo is available in 450g bags.

INDICATIONS FOR USE
Elastic Cromo is an alginate impression material with a color-change indicator and prolonged pouring time indicated for partial dentures, orthodontics, study models, temporary crowns and bridges, reproduction of removable restorations, preliminary impressions for fabricating of individual impression trays.

CONTRAINDICATIONS
None known

WARNINGS
1. Avoid the longest exposure to the eyes, inhalation or ingestion.
2. In case of contact with the eyes or skin, flush with copious amounts of water. If irritation persists, seek medical attention.
3. In case of inhalation, move subject to clear throat and blow the nose to evacuate material. If irritation persists, seek medical attention.
4. Do not swallow or take internally. If accidental swallowing occurs, drink copious amounts of water. Seek medical attention in the event of digestive irregularities.

PRECAUTIONS
1. This product is intended to be used by qualified dental professionals on a general patient population.
2. Wear personal protective equipment when using and handling this product.
3. Use the product in a well-ventilated area.
4. Take precautions when removing the set material from periodically-compromised teeth.
5. Do not leave residues of the impression material in the gingival sulcus or oral cavity.
6. The use of distilled or deionized water at 23°C is recommended for the preparation of the Elastic Cromo impression material. Higher temperatures and water hardness reduce the working time and color intensity of the alginate material.
7. Immediate pouring is recommended, maximum storage time of the impression is up to 120 hours at 100% humidity.

INSTRUCTIONS FOR USE
1. Elastic Cromo mixing ratio: 20ml water per 9g powder (one measure of water per one scoop of powder).
2. Shake the container of Elastic Cromo powder well before use.
3. Measure 20 ml of water by filling the supplied scoop and add it to the mixing bowl. Avoid compacting the powder.
4. Mix the water and powder thoroughly with a spatula along the mixing bowl walls.

5. Elastic Cromo color-change phases allow for timely procedure control:
• Purple: material mixing
• Pink: material loading into a tray and application
• White: set material
6. Apply the homogeneous material to an impression tray and insert the impression tray with the material into the patient's mouth.
7. When the material has set, snap the impression from the patient's mouth.
8. Clean off saliva and debris by rinsing the impression with tap water. Disinfect the impression according to the instructions in the "Cleaning and Disinfection" Section.
9. For best results, pour the impression immediately. If pouring of the cast is delayed, wear the impression in a wet paper towel and store in a sealed plastic bag. Do not pour under water.

Mixing Time	Working Time (including mixing time)	Setting Time (including mixing time)
30 seconds	1:50 minute*	2:30 minutes*

* ± 0.15 minute
Recommended gypsums for casting the impression: Supratone (Type 5), Vel-Mix (Type 4), Hydrex (Type 3).

CLEANING AND DISINFECTION INSTRUCTIONS
Impressions may be safely disinfected with aqueous disinfection solutions after rinsing with water. Refer to the disinfectant manufacturer's instructions for contact time, dilution and temperature of the solution.
To prevent cross contamination, use clean gloves when handling the container with the product and drying accessories. After each use, thoroughly clean the cup and scoop using a cleaning solution of your choice. Follow the manufacturer's instructions regarding cleaning solution concentration, temperature and contact time.

STORAGE, SHELF LIFE AND DISPOSAL
To be stored in a dry and dark place, at a temperature of 5-25°C, in a well-sealed container. Refer to the outer packaging for the expiration date. Do not use after the expiration date. For proper disposal, always follow local and national regulations.

VIGILANCE
In an adverse medical event occurs, report it to the manufacturer and to the competent authority for that country.

DISCLAIMER
The instructions provided above have been validated by SpofaDental as being CAPABLE of preparing a medical device, including reusable ones. It remains the responsibility of the dental professional / user to ensure that the procedures actually performed by the dental professional and personnel is done per the instructions provided above in order to achieve the desired result. Any deviation by the dental professional / user from the instructions provided will be done at your own risk. SpofaDental will not be able to address any requests for refund or exchanges under guarantee of any products that have not been handled or reprocessed in accordance with the above instructions.

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