#### 24. Sterilization

- It is recommended to use sterilisation with damp heat in a steam sterilizer at a temperature of 134 °C and the time of sterilization of 7 min. for the implants.
- It is recommended, when using a hot air sterilization, to perform it in facilities with forced air circulation at the temperature of 160 °C for 60 minutes or at the temperature of 170 °C for 30 minutes or at the temperature of 180 °C for 20 minutes, unless specified otherwise. The actual temperature deviation of sterilization exposure must be within the range of -1 °C to + 5 °C. The hot air sterilizer is opened at the end of the cycle after cooling to at least 80 °C.
- If other methods of sterilization than recommended are used, MEDIN, a.s. is not responsible for the result of the sterilization method and recommends this procedure to be validated by the supplier of the corresponding sterilization method.
- Packaged sterilized instruments can be sterilized repeatedly.
- Sterilize the stands with the instruments separately, open the containers and sterilize also separately.

#### 25. Storage

- Store the packed tools in a dry clean environment without extreme temperature and humidity, away from direct sunlight, animals, insects, dust, fungi and chemicals.

#### 26. Other information

- The process of washing with thermal disinfection has been verified by MEDIN company. In case of use of some other methods of the washing procedure than recommended, the MEDIN company does not guarantee the result. When using some other method of washing than recommended, the MEDIN company recommends this process to be validated by the supplier of such washing facility.
- The MEDIN company has validated the method of sterilization with the damp heat at then temperature of 134 °C and the time of the sterilization of 7 min for the representatives of the WORST-CASE MEDIN products.
- The instructions for cleaning and sterilization are stated in accordance with the norms and standards ISO 15883, ISO 17664, ISO 17665-1. The instructions listed above have been confirmed by the manufacturer of the medical device as appropriate for the preparation of MEDIN non-sterile medical devices. It is the responsibility of the provider to achieve the desired outcome by preparing a product with the use of the equipment, materials and workers on the pre-site. This requires the validation and routine monitoring of the procedure. Likewise every deviation of the provider from these recommendations should be properly assessed in terms of the efficiency and any possible undesired consequences.
- Company MEDIN, a.s. has validated the above mentioned procedures for cleaning, disinfection, drying and sterilization. These procedures were recognized by validation as eligible in the preparation of the reusable medical devices.
- The user is responsible for ensuring the necessary materials, equipment and training of staff in accordance with these prescribed procedures, in order to achieve the desired result.

#### 27. Disposal

- The used instruments may cause the packaging at the disposal to be pierced, so they must be stored in the safe hard containers intended for disposal – plastic containers or containers secured against opening when sorting waste.

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