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Modell: PLUS Modell: NORMAL

PRODUCT DESCRIPTION

DentixPro* disposable dental bibs can be used in dental surgeries, doctor’s surgeries, both general and paediatric, physiotherapy clinics, nursing facilities and health care centres, nurseries and kindergartens. **DentixPro* disposable foil dental bibs** (NORMAL) are made of polyethylene foil which constitutes an efficient mechanical barrier preventing moisture permeation and protecting against pathogenic microorganisms. **Disposable dental bibs made of foil and absorbent paper** (PLUS) are made of the polyethylene layer and have additional protection in the form of two layers of absorbent paper with high absorbency rate for bodily fluids (saliva) and pathological fluids (exudate, vomit, pus, blood, etc.). The application of **DentixPro* disposable dental bibs** improves hygienic conditions required to perform dental procedures and reduces likelihood of the occurrence of periprocedural infections at the average risk of contamination by fluids.

PRODUCT APPLICATION

DentixPro* disposable dental bibs are a non-sterile medical product allowing to maintain proper hygienic conditions in order to protect the patient against contamination and disease causing infections during dental procedures and are necessary to ensure that appropriate procedural routine is applied in dental surgeries.

DentixPro* disposable dental bibs are designed:

- to protect the patient against contamination occurring during dental procedures, such as saliva, particles of dentine left after drilling, dental materials and medicines, etc.;

- to protect the patient during dental procedures against disease causing infections;
- as a barrier – to protect the patient against contamination by fluids that may occur during procedures, which are absorbed by the layer of absorbent paper and in preventing pathogenic microorganisms, contamination and dust particles from the patients’ clothes from penetrating into the oral cavity. The NORMAL model, which only consists of foil layer, does not have absorbent properties.

- for protection of the oral cavity against pathogenic microorganisms and contamination from the patients’ clothes during dental procedures;

- to protect the patient against staining during feeding, in case of disabled patients or children;

DentixPro* disposable dental bibs are a non-sterile and non-invasive disposable medical product, with limited exposure time (up to 24 hours).

APPLICATION

According to the valid graphic description:

Graphic description:
After a single application, the piece used should be put in a plastic bag, after which appropriate procedures on disposal of medical waste should be applied.

CONTRAINDICATIONS

- Product may not be used by persons with allergy to any of the components.

- Product is not designed to be in immediate contact with the skin. It may not be applied directly to injured skin, open undressed wounds and ulcerated skin.

- In case of the occurrence of acute inflammatory and hypertrophic conditions of the skin on the neck, dermatological consultation is recommended.

WARNING

- Product is designed for single application and cannot be re-used.

- To prevent infection, after single application product should be treated as contaminated or potentially contaminated waste and put in containers intended for this purpose, after which appropriate procedures on disposal of medical waste should be applied.

- Product should be stored in a dry place. It should not be directly exposed to sunlight. It should be protected against dampening.

- Product may not be used in case of prior opening or damaging of the packaging.

- Product may not be used after expiry date on the label/packaging.

COMPOSITION:

- NORMAL – polyethylene foil (PE).

- PLUS – hygienic absorbent paper (cellulose), polyethylene foil (PE).

No. and date of issuance of the instructions: W&W/JSS/Wyd.1/Rev.2/03.12.12

DentixPro* “CLASSIC- disposable dental towels

PRODUCT DESCRIPTION

DentixPro* disposable dental towels can be used in dental surgeries, doctor’s surgeries, both general and paediatric, physiotherapy clinics, nursing facilities and health care centres, nurseries and kindergartens. They are composed of two layers of absorbent paper, highly absorbent of bodily fluids (saliva) and pathological fluids (exudate, vomit, pus, blood, etc.) and a layer of polyethylene foil which effectively serves as a mechanical barrier protecting against permeability and pathogenic microorganisms. The application of **DentixPro* – disposable dental towels** improves hygienic conditions required to perform dental procedures and reduces likelihood of the occurrence of periprocedural infections at the average risk of contamination by fluids.

PRODUCT APPLICATION

DentixPro* disposable dental towels are a non-sterile medical product allowing to maintain proper hygienic conditions in order to protect the patient against contamination and disease causing infections during dental procedures and are necessary to ensure that appropriate procedural routine is applied in dental surgeries for adults (CLASSIC model) and children (IT’S 4 KIDS model).

DentixPro* disposable dental towels are designed:

- to protect the patient against contamination occurring during dental procedures, such as saliva, particles of dentine left after drilling, dental materials and medicines, etc.;

- to protect the patient during dental procedures against disease causing infections;

- as a barrier – to protect the patient against contamination by fluids, which is absorbed by the layer of absorbent paper and in preventing pathogenic microorganisms, contamination and dust particles from the patients’ clothes from penetrating into the oral cavity;

- for protection of the oral cavity against pathogenic microorganisms and contamination from the patients’ clothes during dental procedures;

- to protect the patient against staining during feeding, in case of disabled patients (CLASSIC model) or children (IT’S 4 KIDS model);

DentixPro* disposable dental towels are a non-sterile and non-invasive medical disposable product, with limited exposure time (up to 24 hours).

APPLICATION

According to the valid graphic description:

Graphic description:

After a single application, the towel used should be put in a plastic bag, after which appropriate procedures on disposal of medical waste should be applied.

CONTRAINDICATIONS

- Product may not be used by persons with allergy to any component.

- Product is not designed to be in immediate contact with the skin. It may not be applied directly to injured skin, open undressed wounds and ulcerated skin.

- In case of the occurrence of acute inflammatory and hypertrophic conditions of the skin on the neck, dermatological consultation is recommended.

WARNING

- Product is designed for single application and cannot be re-used.

- To prevent infection, after single application product should be treated as contaminated or potentially contaminated waste and put in containers intended for this purpose, after which appropriate procedures on disposal of medical waste should be applied.

- Product should be stored in a dry place. It should not be directly exposed to sunlight. It should be protected against dampening.

- Product may not be used in case of prior opening or damaging of the packaging.

- Product may not be used after expiry date on the label/packaging.

COMPOSITION:

Hygienic absorbent paper (cellulose), polyethylene foil (PE).

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