roeko

Jodoform Tamponade

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Jodoform Tamponade

Instructions for use

Please read carefully!
For dental use only!

Composition
100 cm² of impregnated ribbon gauze contain: iodoform: 23 mg
Other components: cotton wool, liquid paraffin
Indications
Jodoform Tamponade is used for aseptic wound tamponing and drainage of cavities after surgical debridement, drainage of abscesses, fistulae, ulcers and body cavities.

Contraindications
Hypersensitivity to iodine
Hypersensitivity to one of the other components of this medical product
Hyperthyroidism
Pregnancy
To be used only under the supervision of and at the responsibility of the attending dentist during nursing periods, with infants and newborns, as well as in cases of thyroid dysfunction, in particular with repeated applications or use on large areas.

Side effects
May cause contact allergies (see contraindications). In such cases, *Jodoform Tamponade* with iodoform must be removed immediately.
In isolated cases with large wounds, confusion, hallucinations, unconsciousness, vomiting, fever or accelerated heartbeat with cardiac arrhythmia may occur. In these cases, the dressing must be removed immediately and the dentist consulted. As a rule, these side effects are completely regressive within 3-8 days after the end of treatment and should be treated symptomatically by the dentist if necessary.

Interactions with other preparations
Interference with enzymatic preparations.

Dosage and type of application
Jodoform Tamponade with iodoform should only be applied, changed and removed by the dentist only. The dentist determines the quantity to be used. The total quantity of iodoform used per dressing change and day should not exceed 2 g. For removal, the primary package (= laminated aluminium pouch) is opened with a pair of scissors at the marked position. For single use.

Duration of administration
The dentist determines how long *Jodoform Tamponade* is to remain in the wound. As a rule, the individual dressing should not remain in situ for

longer than 24 hours. The dentist determines exceptions and total duration of administration. Dressing change depends on the indication and the wound condition.

Important information
Occlusive dressings should not be applied with use in large areas.
If used in large quantities the relation of iodoform to chloroform may cause the patient to show narcotic symptoms. In that case, *Jodoform Tamponade* must be removed immediately.
Do not store above 25 °C! *Jodoform Tamponade* with iodoform must not be used after the expiry date! Store *Jodoform Tamponade* with iodoform out of reach of children! Iodoform is volatile. For this reason full effectiveness can only be guaranteed when the package is opened for the first time.

Jodoform Tamponade must not be sterilized.

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