EuroSept® Xtra Bur Disinfection

- (EN) Bur disinfection
- (ES) Desinfectante para fresas
- (FR) Désinfectants pour fraises
- (DE) HS - Bohrerbad
- (IT) Disinfettante per manipoli
- (NL) Borenbad
- (PT) Banho para brocas
- (CS) Dezinfekce na rotační nástroje
- (PL) Środek do czyszczenia wiertel
- (SE) Borrbad
- (DK) Borbad

UFI: R5R9-3KXV-G00W-X1S9

Exclusively produced for:

HENRY SCHEIN®**PRISMAN GmbH**
Otto-Hahn-Ring 6-18

DE-64653 Lorsch

Phone: +49 (0) 6251-866 980-0

Rev.2021-07

00/21

LOT

2L e

EN: Type of application: Ready to use bur disinfection solution for manual disinfection of burs, cutters, grinders and other rotary instruments. Also suitable for use in an ultrasonic bath (without heat) at room temperature. Not suitable for the final disinfection of semi-critical and critical medical devices! **Notes on application:** Always use Eusept Xtra Bur disinfection undiluted! After use, immediately place instrument in the ready-to-use solution and ensure they are fully immersed. After the application time, the instruments should be rinsed, ideally with demineralised water, and dried thoroughly. Do not mix with other disinfectants or cleaners, not suitable for aluminium. Depending on the level of contamination, the working solution may need to be replaced (RRI recommendation: at least once per working day). Water down the solution before disposal in any waste water system. Observe the reprocessing instructions of the instrument manufacturers. Semi-critical medical devices must be reprocessed with a suitable disinfectant for a final disinfection or, where applicable, sterilised. Critical medical devices are to be finally treated with a suitable disinfectant or sterilised. For professional users only. Suitable for Dental and Medical areas or application. Wearing protective gloves and eye protection is recommended. **Application times:** Bactericidal (incl. MRSA) as per EN 13727, EN 14561, VWH (dirty conditions): Conc.: 5 min., Yeastcidal (Candida Albicans) EN 13624, EN 14562, VAH, (dirty conditions): Conc.: 5 min., Mycobacteria (M. terre and M. avium) as per EN 14348, EN 14563 (clean conditions): Conc.: 5 min., "Limited virucidal" (enveloped viruses e.g. HBV, HIV, HCV) as per DVW/RKI and EN 17111 (dirty conditions): Conc.: 5 min. (dirty conditions). **Active ingredients:** 100 g contain: 0.396 g Didecyl dimethyl ammonium chloride, 0.2475 g N-(3-Aminopropyl)-N-dodecyl-propan-1,3-diamine. **Safety instructions:** H412 Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of contents/container in accordance with all local / regional / national / international regulations. (EC) 648/2004: Contains <5% nonionic surfactants, <5% cationic surfactants, fragrances. **Notes on incidents:** All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority.

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