DEKASEPTOL™ Gel

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EN
Type of application: The product is intended for cleaning and disinfecting suction hoses, mouth rinsing basins and the suction system of KaVo dental units.

Instructions for use: The disinfectant is mixed ready for use. Do not mix with cleaners or other disinfectants! Observe the KaVo care instructions for the respective treatment centre. The KaVo care instructions and the electronic instructions for use can be downloaded from www.kavo.com/download.

Application times: Bactericidal and levurocidal (Candida albicans) according to EN 13727, EN 13624, EN 14561, EN 14562 and VAH with high load: 5 min.

„Limited virucidal“ (enveloped viruses such as HBV, HCV, HIV, corona and influenza A-virus) according to EN 14476 and EN 17111 with high load: 15 min.

Safety instructions: H 412 Harmful to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves / eye protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove any contact lenses if possible. Continue rinsing. IF skin irritation occurs: Seek medical advice/attention.

Notes on incidents: Any serious incident that has occurred in relation to the device shall be reported by the user/patient to the manufacturer and the competent authority of the Member State in which the user/patient is established.

User instructions: KaVo and DEKASEPTOL are either registered trademarks or trademarks of KaVo Dental GmbH. All other trademarks are the property of their respective owners.

KaVo Technical Service: If you have any technical questions or complaints, please contact KaVo Technical Service: +49 (0) 7351 56-2500 or service.treatmentunits@kavo.com

Target group: The instructions for use are intended for healthcare professionals, in particular dentists and dental practice staff.

Transport and storage: Protect from sunlight. Store in a dry place. If stored properly, the product has a shelf life of 2 years. Keep the top closed until use. The bottle can be used for up to 12 months after opening.

Currently valid packaging law: Properly dispose of packaging in accordance with the applicable packaging law via waste disposal companies/recycling firms.

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