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EN MD 535 Plaster remover

MD 535 is a special cleaner with strong cleaning power for the removal of plaster and alginates from impression trays, spatulas, plaster knives, mixing beakers (also from vacuum mixing devices), prosthetic materials of all types, orthodontic appliances, etc.

Composition:

MD 535 contains complexing agents (15 - 30%), alkaline cleansing agents (< 5%) and auxiliary substances.

Instructions for use: MD 535 is used undiluted. Place the parts to be cleaned in MD 535 and allow the solution to take effect. After treatment, rinse off the cleaning agent with water.

The cleaning process is accelerated by the use of an **ultrasonic unit** and by heating up to a maximum of 50 °C. Renew solution when necessary. Rising gas bubbles indicate a cleaning action. If these are no longer present, a fresh solution must be used. Residues in the MD 535 solution do not influence its action. For further information please also see product information on the Internet.

Note: For chrome- and nickel-plated impression trays, compatibility with MD 535 must be checked. Aluminium impression trays must not be left in MD 535 for longer than one hour. Changes in the colour and odour of MD 535 during storage have no impact on efficacy. For use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

LOT

see imprint.

Wear protective gloves and eye/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Con-

tinue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dis- pose of contents/container to hazardous or special waste collection point. Safety data sheet available on request.

UK-REP: DÜRR DENTAL (Products) UK Ltd.,
14 Linnett Way - Telford Way Industrial Estate,
Kettering, Northants NN16 8PS,
United Kingdom, info@duerruk.com

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En Italia: Dür Dental Italia S.r.l.,
Na Itália: Dür Dental Italia S.r.l.,
Na Portugal: DURR DENTAL MEDICS IBERICA S.A.U.,