EN MTApex™ Bioceramic Root Canal Sealer

Product Description
MTApex™ sealer is a powder and gel system consisting of an extremely fine, inorganic powder of tricalcium and dicalcium silicate which is mixed with the water-based gel before placement. For all products descriptions, please read and understand all instructions and SDS information prior to use.

Delivery Form(s)

- Skini syringe
- Recommended delivery tip:
 - Navitip™ 29 ga Sideport tip

Indications for Use/Intended Purpose
Obturation and sealing of root canals.

Contraindications

- Hypersensitivity against caustic (high pH) solutions.
- Do not use for primary tooth pulp therapy (obturation root canal filling) unless the permanent successor tooth is absent.
- Do not use in patients with allergic reactions to dental products.
- For patients or users with allergic concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- Adverse reactions: reversible inflammation of the oral mucosa if contacted with the unset paste.
- Avoid contact with oral mucosa. After accidental contact, wash and rinse with water.
- Minimize desiccation beyond what protective gloves and eyewear with side shields when handling MTApeax sealer. If contact with eyes happens wash with abundant amount of water.
- Always wear eye/wear protection with side shields when treated with MTApeax sealer.

- MTApex sealer is caustic, as are all tricalcium silicates.
 - In the case of filling root canals with wide foramen (e.g., apical rootlets), inadequate placement and/or inconsistent application add more powder to adjust the consistency and decrease the risk of mucosal extrusion beyond the apical foramen.
 - When using MTApeax sealer into the periapical tissue or periapical lesion. Extra pressure must be taken if the root canal is communicating with the inferior alveolar canal or maxillary sinus.
 - If excess material is extruded into the mandibular canal (inferior alveolar canal), or maxillary sinus space, immediately refer to an oral surgeon for evaluation.
 - Always wipe up any spillage from humidity. Close the container.
 - DO NOT contaminate the powder with an uncured or moist instrument.
 - DO NOT contaminate the gel. Do not remove the dropper tip or insert any instrument into the bottle.
 - AVOID touching the bottle in a contaminated surface.
 - Setting of tricalcium silicates is inhibited in acidic environments such as unvitrified zirconia.
 - Always verify the flow of MTApeax sealer prior to expressing it into the canal.
 - The rubber stopper on the Navitip 29 ga Sideport tip is custom fit for 3.4 mm. Ensure that the rubber stopper is at the predetermined length of 3.4 mm of the apex before inserting the tip into the canal. Verify the position of the stopper before each reinsertion into the canal.
 - Do not express MTApeax sealer if the Navitip 29 ga Sideport tip is wedged or bound inside the canal. The cannula must be kept loose in the canal.
 - Do not use capillary tips on any other plastic type tip for the delivery of MTApeax sealer.
 - Ensure that the tooth is properly isolated using a rubber dam.
 - Skini syringes and tips are single-use only.
 - Use only recommended tips.
 - MTApex sealer is provided in clean, non-sterile packaging. Clinician should follow their established protocols for cleaning and disinfection. See: www.cdc.gov/infectioncontrol/pdf/guidelines/disinfection-guidelines-H.pdf

- Preparation Instructions**
 - Under rubber dam isolation, remove carious tooth structure.
 - Debride, clean, and shape the root canal system using intracanal instruments and desired irrigation and chemical protocol.
 - Dry the canal with paper points.**Step-By-Step Instructions**
Powder Dispensing
 - Shake the bottle to fluff the powder.
 - Dispense one scoop (big end of spoon) of MTApeax sealer powder onto the mixing spatula. Use a spatula edge to level the powder in the spoon.**Gel Dispensing**
 - Shake the gel bottle 3 times. Make sure that all gel is concentrated in the bottle before dispensing drops. Open the cap and dispense 4 drops of MTApeax sealer gel next to the powder.**Mixing Instructions**
 - Gradually combine the gel, in small amounts, into the powder as described in the consistency instructions. Incorporate the gel/spatulating the powder/gel mixture to ensure all the powder is thoroughly wetted by the gel. If the powder is not added gradually, in small amounts, the consistency may be too thick to dispense with the Navitip 29 ga Sideport tip.
 - If the material is not to be used immediately, cover the mixed material with a gauze sponge moistened with sterile water, or a clean cover cloth. Within 15 minutes, remove the moist cover cloth and use the mixed material before it sets.
 - If the mixture is too thin, add a small amount of powder, if too thick, add a small amount of gel to achieve the desired consistency.
 - A thick consistency of MTApeax sealer may be used for obturation of primary teeth with no success.

Applications	MTApex sealer
Powder (measuring spoon)	1 big end
Gel drops	4 drops
Consistency	Stringy
Delivery tip	Navitip 29 ga Sideport tip

NOTE: If more material is needed continue to mix at the same recommended ratio. (1 big scoop end = 4 drops; 1 small scoop end = 2 drops)

For Delivery with Navitip 29 ga Sideport Tip

- Remove the plunger of the clean Skini syringe. Take small portions of the mixed MTApeax sealer with the mixing spatula and insert the sealer in the back part of the barrel. Replace the plunger. Attach the tip securely onto Skini syringe. Gently press the plunger to remove the air inside the syringe. Verify flow of sealer prior to applying irrigation.
- If resistance is met, replace the tip and re-check flow. Use only recommended tips and do not reuse.
- NOTE: MTApeax sealer mixed and placed inside the clean Skini syringe can be used for up to 15 minutes after mixing. Be sure no air is present inside the syringe.
- Place the sealer in the recommended tip, in the treatment site, following the previously listed precautions.
- Confirm placement of the material in the complete root canal system and check for adequate contact between sealer and dentin.
- As with many gutta percha points as your desired obturation technique requires.
- Remove excess gutta percha at the orifice following the manufacturer’s directions.
- Remove excess sealer with a moist cotton pellet.
- Perform a core build up following the manufacturer’s instructions of the core build-up material being used.
- Optional Delivery Method
MTApex sealer may also be placed by dipping gutta percha points into mixed MTApeax sealer prior to placing into prepared root canal.
- For notes of root canal filling, use conventional instrumentation to remove set MTApeax sealer, including hand/rotary files and ultrasonic instrument tips, as with any other sealer.

Storage and Disposal
Store at room temperature. Do not refrigerate. Keep bottles tightly closed. Mixture will reduce the shelf life of the powder. Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information
Report any serious incident to the manufacturer and to the competent authority.
www.ultradent.com / 1 800 552 5512 / 801 572 4200

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