QUADRANT UNIVERSAL LC

NL (Nederlands)

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EN (English)

QUADRANT UNIVERSAL LC

Quadrant Universal LC is a light-curing, fluoride releasing radiopaque micro hybrid, barium glass composite for universal application. Due to the ideal particle size of the glass filler it is polishable to a very high gloss. Together with a universal bonding system (such as Quadrant Uni-SE-Bond or Quadrant Uni-1-Bond) it is indicated for adhesive cavity filling therapy.

Composition

Quadrant Universal LC contains fillers, 61% by volume and 72% by weight. The filler particle size is 0.005 - 10 µm. It contains: Barium Aluminium Bor Fluor Silicate Glass, BisGMA, Tectosilicate, TEGDMA, UDMA, Silicic Acid, Polymer, Titanium Dioxid, Fluorescent Pigments, Metallic Oxide Pigments, Organic Pigments, Aminobenzoicacidester, BHT, Camphorquinone.

Indications

- Restorations of class I,II,III,IV and V cavities, e.g.:
- reshaping
- shade correction
- direct veneers
- closing of diastema
- correction of development abnormalities (e.g. enamel dysplasia)
- splinting of traumatized teeth
- restoration of primary teeth

Interactions with other materials

Do not use in combination with eugenol containing restoratives, as these impair the polymerization.

Use of Quadrant Universal LC Syringes

Remove the cap from the syringe. Turn the plunger clockwise to express material from the syringe. After use, turn the plunger back for 1/4 revolution to release the pressure from the material. When the syringe is empty, dispose it properly.

Use of Quadrant Universal LC Capsules

Place the capsule in the Q-Gun. Remove the cap from the capsule. Squeeze the handle with a slow, steady pressure. Do not force. If the capsule blocks, remove and replace it with a new one. Apply by placing the tip at the bottom of the restoration, and fill it from the bottom up. After use, remove the capsule and dispose it properly. The Q-Gun may be cleaned and sterilized by autoclave or dry-heat at temperatures not exceeding 140°C, or soak in an approved disinfectant solution.

Clinical procedure

1. Prepare the cavity according to the general directions for composite restorations. Clean and dry the cavity. The use of rubber dam is recommended. When restoring exposed root surfaces, a preparation may not be necessary.
2. Apply Quadrant Uni-SE-Bond or Quadrant Uni-1-Bond (or another universal bonding system) following the instructions for use for that product. The oxygen inhibited layer should remain intact.
3. Apply Quadrant Universal LC in layers of max. 2 mm onto the cavity walls. Intensely discolored teeth may require a block-out with an opaqueur.
4. Cure with an appropriate curing light. The curing time is indicated below. During the polymerization, an oxygen inhibited layer develops on the surface. Do not touch or remove this layer, as it is essential for the chemical bond of the next layer.
5. Quadrant Universal LC can be finished and polished with fine diamonds, flexible discs and silicon polishers.

Shade A1, A2, A3, A3.5, B2, B3, C2, C3, I
A4, OA2

Polymerization time per 2 mm

20 sec.
40 sec.

Quadrant Universal LC radiopacity is equivalent to 2.0 - 2.5 mm of aluminum (dentine = 1 mm, enamel = 2 mm). Use a curing light with a wavelength of 440 - 480 nm and a minimum output of 600 mW/cm2. The indicated polymerization times are minimum times and, to obtain an optimal polymerization, should under no circumstances be reduced. To prevent premature setting of Quadrant Universal LC in cases that take more time, the operating light may be dimmed or turned away.

Safety Advice

Quadrant Universal LC contains monomers capable of polymerization (e.g. multi-functional (meth)acrylic acid ester), which may cause irritations in particularly sensitive individuals. If signs of such irritations appear or if allergies to methacrylate-based resins are known to exist, the product should not be used. Wear suitable protective gloves and avoid skin contact. In the event of contact with the eyes or extended contact with oral mucous membranes, the affected area should be thoroughly rinsed with water. In case of contact with the eyes consult a doctor. According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

Storage advice

Do not store at temperatures higher than 25°C (77°F). Do not use this material beyond its expiry date. Avoid direct exposure to sunlight. Close containers firmly after use and keep out of reach of children. Dispose of in accordance with local regulations. Do not dispose of contents or partially emptied packaging in household waste or allow them to enter the sewage system.

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