**Product description:**

Light-curing one-component dentine/enamel bond for a durable bond without marginal gaps between hard substance (dentine/enamel) and light-curing restorative material. **Solobond M** is used for the "wet-bonding" technique and all light-curing restorative materials that are composite based.

**Indications:**

**Solobond M** is suited for direct restorations of all filling classes with light-curing restorative materials.

**Contraindications:**

**Solobond M** contains methacrylate, BHT and organic acids. **Solobond M** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

**Patient target group:**

**Solobond M** is suitable for use in all patients without any age or gender restrictions.

**Performance features:**

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

**User:**

**Solobond M** should only be used by a professionally trained dental practitioner.

**Use:**

The adhesive bond within the total-etch technique is achieved as follows:

**1. Drying**

Due to the subsequent application of light-curing restorative material, it is necessary to dry the tooth surface. Avoid any contamination of the cavity with blood or saliva. The use of a rubber dam is recommended.

**2. Preparation**

Prepare the cavity according to the principles of the adhesive restoration technique. Areas close to the pulp should be protected with a suitable calcium hydroxide based liner.

**3. Etching of enamel margins**

Etch entire cavity with etching gel (**Vococid**) starting from the enamel margin (enamel: approx. 20 - 30 s, dentine: 15 s maximum). Remove etching gel by suction, rinse for approx. 20 s and remove excess moisture with an air blower. Do not overdry dentine. The surface should remain slightly moist. The etched enamel should have a chalk-white/matt appearance. If not, repeat etching.

**4. Solobond M dispensing**

Dispense **Solobond M** onto a disposable brush. As an alternative, dispense **Solobond M** onto a mixing pallet.

**5. Solobond M application**

Apply **Solobond M** homogeneously to all prepared enamel and dentine surfaces with a disposable brush and allow to act for 30 s. Then disperse **Solobond M** with a faint air jet and polymerise with LED/halogen light for 20 s. Do not remove or contaminate the inhibition layer since it is important for the adhesion to the composite.

**6. Filling**

Work the filling material according to the manufacturer's instructions.

**Warnings, precautionary measures:**

- Phenolic substances (e.g. temporary materials containing eugenol) impair polymerisation and must not come in contact with **Solobond M**.

- Avoid contact with the oral mucosa. In case of contact of **Solobond M** with the eyes, rinse immediately with plenty of water and seek medical advice.

- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

**Constituents (in descending order according to content):**

Acetone, BisGMA, HEMA, methacrylated phosphoric acid ester, methacrylate functionalised polyalkenoic acid, HPMA, initiators, stabilisers

**Storage instructions and application method:**

Tightly close **Solobond M** bottles after use. Store at temperatures between 4°C - 28°C (storage at 5°C - 8°C prolongs the shelf life of the product). Do not expose **Solobond M** to direct sunlight or the chairlight.

Do not use after the expiry date.

**Disposal:**

Dispose of the product in accordance with local regulations.

**Reporting obligation:**

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Solobond M** must be reported to VOCO GmbH and the responsible authority.

**Note:**

The Summary of Safety and Clinical Performance of **Solobond M** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at [www.voco.dental](http://www.voco.dental).

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VC 60 AA1228 E1 0223 99 © by VOCO