Prior to use, carefully read the instructions for use.

EN

## CERAMIC PRIMER II

PRIMER FOR CERAMIC AND COMPOSITE BONDING

CERAMIC PRIMER II is a priming agent designed to treat the adherent surface of ceramic and composite prostheses to promote a strong bond to resin-based materials as resin-based cements in case of luting procedures or direct compositing in case of repairs.

For use only by a dental professional in the indications for use.

### INDICATIONS FOR USE

1. Priming of the adherent surfaces of indirect restorations made of ceramics (glass ceramics, zirconia and alumina), hybrid ceramics (e.g. CERASMART) and composite for luting procedures.

2. Indirect extra-oral repair of fractured indirect restorations made of ceramics, hybrid ceramics and composites.

**CONTRAINDICATIONS**

Avoid the use of this product in patients with known allergies to methacrylate monomer, methacrylic polymer or ethanol.

### COMPOSITION

Ethanol, dimethylacrylate, phosphoric acid ester monomer, silane coupling agent

### DIRECTIONS FOR USE

1. For luting procedure:

1.1 Treat the bonding surfaces of prosthesis according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. We recommend cleaning in an ultrasonic bath with alcohol, then rinse with dry air or oil-free air syringe.

1.2 Bottle type: Dispense one drop of CERAMIC PRIMER II into a dispensing dish.

Unit dose type: Unscrew cap of unit dose counter-clockwise.

Make sure to identify top and bottom of the unit dose capsule before opening.

1.3 Apply a thin layer to the bonding surface of the ceramic or composite prosthesis using a micro-tip applicator and dry with an air syringe.

4. Apply suitable resin-based luting cement to the primed surface of the prosthesis. For these steps, refer to the manufacturer's instructions for use of the corresponding luting cement.

### II. For intra-oral repair:

1. Rinse the bonding surface of ceramic or composite using a coarse abrasive.

2. Treat the bonding surfaces of the prosthesis according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with an oil-free air syringe.

3. Bottle type: Dispense one drop of CERAMIC PRIMER II into a dispensing dish.

Unit dose type: Unscrew cap of unit dose counter-clockwise.

Make sure to identify top and bottom of the unit dose capsule before opening.

4. Apply a thin layer to the fractured surface of the ceramic or composite using a micro-air applicator and dry with an oil-free air syringe.

5. Continue the repair by using a light-cured adhesive (e.g. G-Premio BOND) and a light-cured composite (e.g. G-universal FLO), referring to their respective manufacturer's instructions for use.

Note: If the treated adherent surface is contaminated with saliva or blood, rinse with water and dry. Re-treat the surface according to the manufacturer's instructions and re-apply CERAMIC PRIMER II according to the instructions above.

### STORAGE

Recommended for optimal performance, store at room temperature (4-25°C, 39-27.7°F).

### PACKAGE

1. CERAMIC PRIMER II, 1 x bottle of 3mL.

2. CERAMIC PRIMER II, 10 x unit dose of 0.1mL.

### CAUTION

1. Personal protective equipment (PPE) such as gloves, face masks and eyewear should always be worn.

2. Avoid inhalation or ingestion of the material.

3. CERAMIC PRIMER II is highly flammable liquid and vapor. Do not use near open flame. Keep away from sources of ignition.

4. Keep away from direct sunlight. Store in a well-ventilated place.

5. In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse immediately after luting or repairing.

6. In case of contact with eyes, flush immediately with water and seek medical advice.

7. In case of contact with eyes, flush immediately with water and seek medical advice.

8. Unit dose is disposable. Do not recast or reuse to avoid cross-contamination between patients.

9. If the cap of this product is too tight to open, do not try to open it with force to avoid accidental contact of the liquid with skin and eyes. Use another unit dose.

10. Do not mix with any other primer.

11. Dispose of as waste according to local regulations.

12. This product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.geoeurope.com>. These can also be obtained from your supplier.

### CLEANING AND DISINFECTION:

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between devices and label for decontamination. Immediately after use inspect device and label for decontamination. Disinfect device if damaged.

DO NOT IMMERSIVE: Thorougly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUAMED database (<https://ec.europa.eu/tools/euamed>) or contact us at [regulatory@gc.dental](mailto:regulatory@gc.dental).

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar effect, please report it of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)

In this way we will contribute to improve the safety of this product.

Last revised: 01/2024

CE 2797

'GC'

MANUFACTURED by:

GC DENTAL PRODUCTS CORP.,

285-285 Toranisho-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by:

GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.

Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,

B-3001 Leuven, Belgium TEL: +32 16 74 10 00

PRINTED IN JAPAN

Prior to use, carefully read the instructions for use.

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