VOCO

Registrado® X-tra

0482

EN

Instructions for use

MD

EU Medical device

Product description:
Registrado X-tra is an addition-curing silicone impression material for occlusal registrations, which stands out for its high final hardness and very fast setting time. Its high stability prevents it from running down the dental arch and also allows for application in thicker layers. The special, temperature-controlled curing formula allows for application without time pressure and an extremely short retention time in the mouth of only 40 sec, after which the registration can be removed safely, without breaking or distorting. After curing, **Registrado X-tra** can be finished in a simple and controlled manner using a scalpel or bur. It remains dimensionally stable in the long term.

- Indications:**
- Occlusal bite registration
 - Fixing of face bow registrations
 - Location material for intraoral pin tracing templates

Contraindications:
Registrado X-tra contains platinum catalysts. **Registrado X-tra** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:
Registrado X-tra is suitable for use in all patients without any age or gender restrictions.

Performance features:
The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:
Registrado X-tra should only be used by a professionally trained dental practitioner.

Use:
Usable mixing tips: type 16

- 1. Preparation:**
To dispense **Registrado X-tra**, use a **VOCO** Dispenser type 2 or any other suitable dispenser. For technical reasons there may be slight differences in the fill levels of both chambers. You should therefore remove the cap and press out the material until an equal amount has been ejected from both outlets. Then attach the mixing tip and lock (with a 90° turn clockwise). The material is automatically mixed by the mixing tip in the correct ratio. An application tip type 2 may be placed atop the mixing tip type 16 to facilitate a finer, more precise application.
The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.
The attached mixing tip is intended for single use only. After use, store the cartridge tightly sealed. Check that material flows freely through the openings before reusing.

- 2. Processing:**
To obtain an occlusal registration, **Registrado X-tra** is applied directly to the patient's upper or lower dental arch.
Registrado X-tra works for max. 30 seconds (as of start of mixing at 23°C room temperature). After a retention time in the mouth of 40 seconds after closing of the mouth, it can be removed.

Processing time (at 23°C)	≤ 30 sec
Retention time in mouth (at 37°C)	≥ 40 sec

- Warnings, precautionary measures:**
- Do not use **Registrado X-tra** in combination with condensation-curing silicones.
 - The setting behaviour of **Registrado X-tra** can be affected by the use of latex gloves, hand creams, cleaning agents, mouth rinses, retraction solutions and other material residue (e.g., impression materials). Thus, these must be removed prior to application (spray clean if necessary).
 - Avoid contact between incompletely cured silicone material and clothing.
 - Disinfection with an aqueous disinfectant is recommended prior to further processing of the occlusal registration created. When doing so, observe the instructions for use of the disinfectant used.
 - Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:
Storage at 4°C-23°C. Do not use after the expiry date.

Disposal:
Dispose of the product in accordance with local regulations.

Reporting obligation:
Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Registrado X-tra** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA2035 E1 0924 99 © by VOCO