VOCO Registrado® Clear

EN Instructions for use
MD EU Medical device

Product description:
Registrado Clear is an addition-curing silicone impression material for occlusal registration, which stands out for its transparency, high final hardness of more than 70 Shore A, and a fast setting time.

Indications:
– Bite registration
– Silicone key for composite restorations
– Front and occlusal record
– Registration key for intraoral bearing device registration
– Fixing of x-ray balls

Contraindications:
Registrado Clear contains platinum catalysts. **Registrado Clear** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:
Registrado Clear is suitable for use in all patients without any age or gender restrictions.

Performance features:
The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:
Registrado Clear should only be used by a professionally trained dental practitioner.

Use:
Usable mixing tips: type 16

1. Preparation:
To dispense **Registrado Clear**, use a **VOCO** Dispenser type 2 or any other suitable dispenser. For technical reasons there may be slight differences in the fill levels of both chambers. You should therefore remove the cap and press out the material until an equal amount has been ejected from both outlets. Then attach the mixing tip and lock (with a 90° turn clockwise). The material is automatically mixed by the mixing tip in the correct ratio. An application tip type 2 may be placed atop the mixing tip type 16 to facilitate a finer, more precise application.
The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.
The attached mixing tip is intended for single use only. After use, store the cartridge tightly sealed. Check that material flows freely through the openings before reusing.

2. Processing:
To obtain an occlusal registration, **Registrado Clear** is applied directly to the patient's upper or lower dental arch. Bite down within 20 seconds. At room temperature (23°C), **Registrado Clear** has a processing time of max. 25 seconds. Following an intraoral retention time of 90 seconds after closing the mouth, the occlusal registration can be removed. After curing, **Registrado Clear** can be finished in a simple and controlled manner using a scalpel or scissors. It remains dimensionally stable in the long term.

Processing time (at 23°C)	≤ 25 sec
Retention time in mouth (at 37°C)	≥ 90 sec
Shore A hardness	> 70
Degree of resilience	> 99%

Warnings, precautionary measures:
– Do not use **Registrado Clear** in combination with condensation-curing silicones.
– The setting behaviour of **Registrado Clear** can be affected by the use of latex gloves, hand creams, cleaning agents, mouth rinses, retraction solutions and other material residue (e.g., impression materials). Thus, these must be removed prior to application (spray clean if necessary).
– Avoid contact between incompletely cured silicone material and clothing.
– Disinfection with an aqueous disinfectant is recommended prior to further processing of the occlusal registration created. When doing so, observe the instructions for use of the disinfectant used.
– Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:
Storage at 4°C-23°C. Do not use after the expiry date.

Disposal:
Dispose of the product in accordance with local regulations.

Reporting obligation:
Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Registrado Clear** must be reported to VOCO GmbH and the responsible authority.

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