Instructions For Use

Kerr Corporation
1889 W. Mission Blvd.,
Pomona, CA 91766 USA
1-800-KERR-123
kerrdental.com

Distribution Facility:
Kerr Australia Pty. Ltd.
Unit 6, 12 Mars Road
Lane Cove West, NSW 2066
Australia
+61 2 8870 3000

DMC4477e_Rev.01
2025-07

Rx only*
*for US law only

©2025 Kerr Corporation. All Rights Reserved.

Kerr™

SYMBOLS DESCRIPTION

Full explanation of symbols used on Kerr packaging is located at:

<http://www.kerrdental.com/symbols-glossary>

	Caution
	Manufacturer
	Consult instructions for use
	Use-by date
	Batch code
	Catalogue number
	Do not re-use
	Medical device
	Temperature storage limitations: low 2°C/36°F to high 28°C/82°F
	Keep away from sunlight
	Caution: US Federal law restricts this device to sale by or on the order of a dentist

CONTENTS

	1. Syringe (4 g)
	2. Unidose (0.2 g)

en - ENGLISH

Point 4™

Nanohybrid Composite Restorative

ISO 4049: Polymer-Based Restorative Material, Class 2/Group 1.

GENERAL INFORMATION

Point 4 represents years of research aimed at bridging the gap between microfill and hybrid composite materials. A proprietary, grinding process combined with sophisticated rheological modifiers allow Point 4 to polish to a very high, long lasting luster similar to microfills, while providing strength equivalent to today's microhybrids. It is an intra-oral, light cured, resin-based, dental restorative. Suitable for occlusal surface. Product is available in a variety of Vita™ shades.

PRODUCT COMPOSITION

The term nanohybrid is used to describe the filler system with composition of both nano-sized (particles within the range of 1 nm – 1,000 nm) along with other micron-sized fillers.

100% Aluminium has a radio-opacity equivalent to that of dentine and 200% Aluminium is equivalent to enamel.

All shades are radiopaque with about 250% Aluminium.

The inorganic fillers comprise barium glass filler and silica (about 76% by weight or 57% by volume). All shades have a primary particle size range of 40 nm to 0.6 microns.

Product contains Bis-EMA, bis-GMA, TEGDMA, and/or UDMA resins.

None of the ingredients in the product contain pharmacologically active components.

INDICATIONS FOR USE

Point 4 is a dental composite restorative material intended to be used in all classes of cavities.

CONTRAINDICATIONS

None known.

WARNINGS

1. Do not use this product on patients with known methacrylate allergies.
2. Avoid product application in direct contact with pulp tissue since exposure to this product may injure the pulp.
3. Avoid skin, eye, and soft tissue exposure to this product since uncured methacrylate may cause an irritation reaction. Wash thoroughly with water after contact. Get medical attention for eyes. Do not take internally.
4. OptiBond™ Gel Etchant contains 37.5% phosphoric acid. Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact. Get medical attention for eyes. Do not take internally.

PRECAUTIONS

1. This product is intended to be used by qualified dental professionals on a general patient population.
2. Wear personal protective equipment when using and handling this product.
3. Use of protective gloves and a no-touch clinical technique is strongly recommended.
4. If the composite is touched with commonly used gloves, acrylate may penetrate the gloves. Dispose of gloves immediately and wash hands.
5. Suitable protective eyewear for patients and medical staff is recommended when light curing the material.
6. Use a rubber dam to isolate the treated tooth and avoid fluid contamination during material placement.
7. Protect the Point 4 material from ambient light that may initiate polymerization.
8. Once the OptiBond Solo Plus is placed in a mixing well, use immediately or protect from ambient light.
9. Replace the syringe cap after each use to prevent the resin from hardening in the syringe.
10. OptiBond Solo Plus bottle should be tightly closed immediately after use.
11. Unidose tips, Gel Etchant Dispensing tips, disposable mixing wells, and Kerr Applicators are intended for single patient use only, to prevent cross contamination between patients.
12. Do not use this product near eugenol-containing products.

Precautions regarding use of OptiBond Gel Etchant:

1. Never use excessive force on syringe plunger to dispense etchant which may lead to over dispensing of the material. If resistance is felt replace current dispensing tip with a new tip and check flow outside the oral cavity.
2. Replace the syringe cap after each use to prevent the product from drying in the syringe.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS

1. Select the desired shade. Point 4 uses the VITA shading system for its Base and Opaque shades. Furthermore, Point 4 is available in 6 specialty shades: Translucent shades: T1 – a clear translucency; T2 – a slightly yellow translucency; T3 – a slightly gray translucency. Extra Light Shades: XL1, XL2, XL3 – all 3 of them are lighter than the lightest VITA shade (B1), with the lightest

shade being XL3. A composite mock-up or VITA-matched shade guide may be used to check the color of the tooth to be restored and any adjacent teeth.

- After shade selection, isolate the tooth with your preferred isolation method.
- Prepare the tooth using your preferred rotary instruments and using contemporary cavity designs.
- If a pulp exposure has occurred, place a small amount of calcium hydroxide followed by the placement of a conventional glass ionomer base.
- Complete the bonding procedure per manufacturer's instructions. Point 4 is compatible with all methacrylate dental adhesive systems. Please see instructions below for proper bonding technique using OptiBond Solo Plus.
- Dispense the Point 4.

Syringe Delivery: After dispensing the composite onto the pad, relieve pressure by turning the syringe handle back 2 turns. This will prevent material build-up in the cap and provide reliable composite use after storage.

Unidose Delivery: Insert tip and properly align within dispenser. Place Unidose tip at the deepest portion of the preparation.

- Increments should be no more than 1.5-2 mm at a time. After placing an increment, stroke the composite to ensure marginal adaptation.
- Light cure each increment. When multiple surfaces are available, cure each surface. Point 4 is intended to be cured by an LED light with a minimum light intensity of 1,000 mW/cm² and a light wavelength output within the 400-520 nm range. Please refer to the table below for curing recommendations.

Point 4	Opaque, XL2, XL3 Shades	Base, Translucent (Incisal), XL1 Shades
Incremental Depth	1.5 mm	2.0 mm
Demi™ Plus (or light with output >1,000 mW/cm ²)	10 seconds	

For all other lights, see manufacturer's recommendations.

- Adjust occlusion as needed. Complete the finishing and polishing procedure.
- Direct Bonding to enamel/dentin with OptiBond Solo Plus – Total-Etch technique:**

- Isolate and dry.
- Place OptiBond Gel Etchant on enamel and dentin for 15 seconds.
- Rinse thoroughly, ensuring that all etch is removed.
- Dry lightly, do not desiccate.
- If using Solo Plus in bottle delivery, shake bottle briefly before dispensing. Dispense 1 drop of adhesive into the disposable mixing well. If using Solo Plus in Unidose delivery, twist capsule at middle, and discard side with the long stem. Insert the applicator brush into the opening to saturate the tip.
- Apply OptiBond Solo Plus to enamel/dentin surface with light brushing motion for 15 seconds.
- Lightly air thin for 5 seconds.
- Light cure the material. The OptiBond Solo Plus is intended to be cured by an LED light with a minimum light intensity of 1,000 mW/cm² and a light wavelength output within the 400-520 nm range. Please refer to the table below for curing recommendations.

Light Type	OptiBond Solo Plus Curing Time
Demi™ Plus (or light with output >1,000 mW/cm ²)	5 seconds

For all other lights, see manufacturer's recommendations.

- Proceed with placement of Point 4 composite.

CLEANING AND DISINFECTION

Unidose Tips: Do not re-use Unidose tips.

Unidose tip guns: Refer to the reprocessing instructions for the manufacturer of the tip gun you are using.

Composite syringes: To prevent syringes from getting contaminated with saliva or other body fluids, use protective barriers. Always handle the syringe with clean

gloves. Do not re-use if it becomes contaminated.

STORAGE AND DISPOSAL

Product should be stored between 2°C to 28°C (36°F to 82°F). Avoid excess heat. If product is stored in a refrigerator allow the product to reach room temperature prior to use.

Every effort should be made to protect the composite from visible light prior to use.

The expiration date is marked on the packaging. Do not use after expiration date.

For proper disposal always follow your local and national regulations.

DISCLAIMER OF LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

©2025 Kerr Corporation. All trademarks applicable to Kerr are property of Kerr Corporation. All other trademarks are property of their respective owners. All Rights Reserved.

Full explanation of symbols used on Kerr packaging is located at:

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