

Prime&Bond universal™

Universal Adhesive



		
20 sec Agitate slightly	5 sec Remove solvent	Cure ≥ 500 mW/cm ² : 20 sec ≥ 800 mW/cm ² : 10 sec



Caution, consult instructions for use

THE DENTAL
SOLUTIONS
COMPANY™

 Dentsply
Sirona

Step by step instructions:



Preparation

Pull off originality seal.

Flip Top Bottle can be opened and closed with one hand. Do not screw off the cap.

Dispense Prime&Bond universal™ into a CliXdish™.

In a closed CliXdish™ Prime&Bond universal™ remains usable for up to 30 minutes.

To close the bottle, press down the cap until it locks in place with a perceptible "click".



Cleaning & Disinfection
Disassemble CliXdish™ for reprocessing.

OPTIONAL

7 min. 15sec.

Etching

Start etching at enamel margins.

8 max. 15sec.

Extend to dentin.

9 min. 15sec.

Rinsing

Remove acid gel and rinse.




Remove excess water.
Do not desiccate!



Application

Apply Prime&Bond universal™ to all cavity surfaces. Avoid pooling.

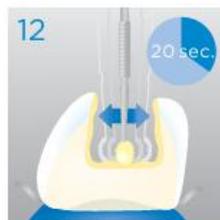

Keep Prime&Bond universal™ slightly agitated for 20 seconds.



Evaporation

Evaporate solvent with air for at least 5 seconds.



Light-Curing

≥ 500 mW/cm² = 20s
≥ 800 mW/cm² = 10s

Dentsply DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
www.dentsplysirona.com

© 2016 Dentsply Sirona