3M ESPE

Ketac Universal

Glass Ionomer Restorative Material – Radiopaque

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Vita is a trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.

Bad Säckingen, Germany.

A.R.T. (atraumatic restorative technique):

► Apply Ketac Universal in several portions using a suitable instrument. Undercuts may need to be filled.

► Carefully avoid trapping air bubbles under enamel undercuts.

► Slightly overfill cavities; apply the material to adjacent grooves and fissures.

► Apply a small amount of petroleum jelly to your forefinger and use this finger to firmly press the glass ionomer cement into cavities, fissures, and grooves.

► Repeatedly move the tip of the forefinger in a mesio-distal and bucco-lingual direction to distribute the material over the entire occlusal surface (Finger press technique).

► Finally, remove the finger sideways to avoid lifting the restorative material out of the cavity.

Fissure sealing with the A.R.T. technique:
The same steps for cleaning, application, and processing as in the A.R.T. technique are performed to seal fissures, the only difference being that there is no preparation involved.

Filling Protection
Apply a protective coat to the filling is not mandatory. If a protective coat is desired, vaseline and/or cotton rolls can be used as follows:

► Immediately after shaping, apply vaseline to all free surfaces of the glass ionomer cement.

Finishing

► Begin finishing no sooner than 5 minutes after the start of mixing. Use Arkansas stones, fine-grain diamonds, polishing discs of decreasing grain size, or silicone polishers for finishing.

► Optionally, apply vaseline again to seal the surface.

Storage and Shelf Life
Store the product at 15–25 °C/59–77 °F.

Do not use after the expiration date.

Customer Information

For more information, please provide any information which deviates from the information provided in this instruction sheet.

Waranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH’s sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

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3M, ESPE, Ketac and Scotchbond are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada.
© 2020, 3M. All rights reserved.