**General Information**

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**Indications****Final cementation of:**

- Metal and porcelain fused to metal (PFM) crowns and bridges
- Metal inlays and onlays
- Crowns and bridges made with all-alumina or all-zirconia cores such as 3M™ ESPETM Lava™ or Procera® AllCeram
- Prefabricated all cast posts
- Orthodontic bands and appliances
- Porcelain fused to metal (PFM), metal, all-alumina or all-zirconia core restorations on implant abutments

**Precautionary Information for Patients:**

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylics may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M™ ESPE MSDS information can be obtained from [www.3MESPE.com](http://www.3MESPE.com) or contact your local subsidiary.

**Instruction for Use****Preparation**

1. Prepare teeth using accepted clinical guidelines.  
2. Cover areas in close proximity to the pulp by applying small amounts of hard-setting calcium hydroxide material or resin-modified glass ionomer liner (e.g. 3M™ ESPETM Vitebond™ Light Cure Glass Ionomer Liner/Base) prior to taking an impression for the final restoration.

3. Remove the temporary restoration and thoroughly clean the preparation of any temporary cement residue using an oil-free pumice paste.

4. Rinse the prepared tooth or cavity with a water spray, and dry with air, cotton, or a paper tip.

4.1. Leave tooth surface moist. Do not dry.

4.2. The preparation or cavity should be just dry enough that the surface has a slightly glossy appearance. As is the case with any permanent cement, over drying can lead to post-operative sensitivity.

5. Try in final restoration and check for fit. Adjust if necessary.

6. Sandblast the bonding surface of the material with Al<sub>2</sub>O<sub>3</sub> (50 µm, 2 bar, 1min).

7. Thoroughly clean the bonding surface of the restoration.

8. Keep area isolated from blood and saliva contamination during cementation process.

**Directions****1. Application**

1.1. Remove cap from the Clicker dispenser by holding down the cap lever and sliding the cap off of the dispenser.

1.2. For first use from a new cartridge, dispense a small amount of material to ensure even dispensing. Discard this material.

1.3. Fully depress clicker lever to dispense "1 Click" of cement onto mix pad.

1.4. Leave tooth surface moist. Do not dry.

1.5. The preparation or cavity should be just dry enough that the surface has a slightly glossy appearance. As is the case with any permanent cement, over drying can lead to post-operative sensitivity.

1.6. Apply as thin layer of cement to the inside surface of the restoration. The cement may also be applied directly to the tooth surface for inlay/onlays.

1.7. Seat the restoration with light pressure. Maintain light pressure on the restoration to hold proper positioning during setting process.

**2. Clean Up and Finishing**

2.1. Excess cement is best removed after brief light exposure (5 seconds per surface with a conventional polymerization device) or during self-curing wave stage (starting 2 minutes after seating in the mouth). Use a suitable instrument (e.g. scalar, explorer) for this process.

2.2. Final restoration and check occlusion when material has completely set after 5 minutes from placement in the mouth.

**Storage and Use**  
This product is designed to be used at room temperature. If stored in cooler allow product to reach room temperature prior to use. Seal all room temperature is 24 months. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. See outer package for expiration date.

Do not allow the pastes to dry out. Store with cap securely attached to the dispenser.

**Cleaning and Disinfection**

Disinfection can be done on an intermediate level disinfection process (liquid contact) as recommended by the Centers for Disease Control and endorsed by the American Dental Association. Guidelines for Infection Control in Dental Health-Care Settings – MMWR, December 19, 2003;52(RR-17), Centers for Disease Control and Prevention.

**References:**

CDC Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008. Rutala WA, Weber DJ, Healthcare Infection Control Practices Advisory Committee (HICPAC).

**Disposal** – See the Material Safety Data Sheet (available at [www.3MESPE.com](http://www.3MESPE.com) or through your local subsidiary) for disposal information.

**Customer Information**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Caution:** U.S. Federal Law restricts this device to sale or use on the order of a dental professional.

**Warranty**

3M™ ESPE warrants this product will be free from defects in material and manufacture. 3M™ ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M™ ESPE's sole obligation shall be repair or replacement of the 3M™ ESPE product.

**Limitation of Liability**

Except where prohibited by law, 3M™ ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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