3M ESPE	
Ketac™ Universal Aplicap™	
en Glass Ionomer Restorative Material – Radiopaque	
ru Материал стеклиономерный пломбирочный – рентгеноконтрастный	
bg Глас-иономерен obturuvъчен материал – рентгеноконтрастен	
hr Staklenoionomerni materijal za ispunе – vidljiv na rendgenskim snimkama	
hu Üvegionomer tömőanyag – rادیopak	
pl Szkło-jonomerowy materiał do wypełnień – nieprzepuszczalny dla promieniowania	
ro Ciment ionomer de sticlă pentru obturații – radioopac	
sk Skloionomerový výplňový materiál – rádiokontraštný	
sl Glasionomerni cement za restavracije – radiopacn	
cs Skloionomerní výplňový materiál – radiopakní	
tr Cam ionomer dolgu maddesi – radyopak	
et Klaasionomeerne täidismaterjal – röntgenkontrastne	
lv Stikla jononēra restaurācijas materiāls – rentgenkontrastains	
lt Stiklo jonomerinė plombavimo medžiaga – rentgenkontrastiška	
uk Склоіономерний реставраційний матеріал – рентгеноконтрасний	

Instructions for Use	Navodila za uporabo
Инструкция по применению	Нáвод к пoуžití
Указания за употреба	Kullanma Talimatları
Használati utasítás	Kasutusjuhend
Instrukcja użycia	Lietošanas Instrukcija
Instrucţiuni de Utilizare	Variojomo instrukcija
Návod na použitie	Інструкці з використання

 	3M Deutschland GmbH Dental Products Carl-Schurz-Str. 1 14453 Neuss - Germany
--	--

3M, ESPE, Aplicap, CapMix, Ketac, RotoMix and Scotchbond are trademarks of 3M or 3M Deutschland GmbH. © 2020, 3M. All rights reserved.

Vita is a trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG, Bad Säckingen, Germany.

 	44000769404
--	--

en ENGLISH	
Description Ketac® Universal Aplicap™ is a radiopaque glass ionomer restorative in capsules used for bulk fillings. The ability of the material to bond chemically to enamel and dentin ensures gentle preparation and excellent sealing on the margins of fillings. Ketac Universal Aplicap can be applied without lining; it releases fluoride ions and is available in various shades corresponding to the Vita® Classical color system. The dispensable quantity of a capsule is a minimum of 0.13 ml. ^{en} These Instructions for Use should be kept for the duration of product use. Details of all of the other products mentioned here can be found in the relevant Instructions for Use.	
Indications - Linings for single- and multiple-surface composite fillings - Core build-up prior to crown placement - Primary tooth fillings - Stress-bearing Class I restorations with at least one additional support outside of the filling area - Stress-bearing Class II restorations when the isthmus is less than half of the intercuspal distance and with at least one additional support outside of the filling area - Cervical fillings, if aesthetics is not the prime consideration - Single- and multiple-surface temporary fillings - Fissure sealing	
Precautionary Measures 3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.	
Cavity Preparation - Remove only carious tooth substance; undercuts are not required. - Do not prepare thin beveling margins since the material requires a minimal wall thickness of 0.5 mm! - Clean the cavity with water and dry with air. Do not desiccate.	
Pulp Protection In healthy, cover areas in close proximity to the pulp with local application of a hard-setting calcium hydroxide material to prevent pulp irritation.	
Times The following times apply for an ambient temperature of 23 °C/73 °F:	
Activation	min:sek 0:02
Mixing in RotoMix™	0:08
Mixing in high-frequency mixer, e.g., CapMix™	0:10
Working time from start of mixing	1:50
Setting time from start of mixing	3:40
Room temperatures above 23 °C/73 °F will accelerate the setting time. Exceeding the working time will cause loss of adhesion to dental enamel and dentin.	

Activation - Place the Aplicap® Activator on a sturdy surface and insert the Aplicap capsule into the activator, keeping the application nozzle closed. - The Aplicap Activator and the Applier are both color-coded in orange.	
Using the ball of your hand, depress the activator lever firmly as far as it will go and hold it down for 2 to 4 seconds. Firmly and completely depressing the lever and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or if the lever is not depressed completely, it is possible that too little liquid will be mixed with the powder, increasing the viscosity and changing the product characteristics.	

Precautionary Measures 3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.	
---	--

Cavity Preparation - Remove only carious tooth substance; undercuts are not required. - Do not prepare thin beveling margins since the material requires a minimal wall thickness of 0.5 mm! - Clean the cavity with water and dry with air. Do not desiccate.	
---	--

Pulp Protection In healthy, cover areas in close proximity to the pulp with local application of a hard-setting calcium hydroxide material to prevent pulp irritation.	
Times The following times apply for an ambient temperature of 23 °C/73 °F:	
Activation	min:sek 0:02
Mixing in RotoMix™	0:08
Mixing in high-frequency mixer, e.g., CapMix™	0:10
Working time from start of mixing	1:50
Setting time from start of mixing	3:40
Room temperatures above 23 °C/73 °F will accelerate the setting time. Exceeding the working time will cause loss of adhesion to dental enamel and dentin.	

Capsule Activation - Place the Aplicap® Activator on a sturdy surface and insert the Aplicap capsule into the activator, keeping the application nozzle closed. - The Aplicap Activator and the Applier are both color-coded in orange.	
Using the ball of your hand, depress the activator lever firmly as far as it will go and hold it down for 2 to 4 seconds. Firmly and completely depressing the lever and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or if the lever is not depressed completely, it is possible that too little liquid will be mixed with the powder, increasing the viscosity and changing the product characteristics.	

Application - Insert the capsule into the Aplicap® Applier and open the application nozzle. - Apply the glass ionomer cement in the cavity and shape the filling. – Ensure that there is no contamination from water, saliva, etc., for the entire duration of application and shaping; the working environment must be kept dry. - Ketac Universal Aplicap clings to metal instruments and therefore should be washed off with cold water before the material sets.	
---	--

Attention Avoid any delay between activation, mixing, and application as the material will begin to set in the capsule, rendering the extrusion difficult or even impossible.	
---	--

Filling Protection Applying a protective coat to the filling is not mandatory. If a protective coat is desired, vaseline and/or cotton rolls can be used as follows: Immediately after shaping, apply vaseline to all free surfaces of the glass ionomer cement.	
--	--

Finishing - Begin finishing no sooner than 3:40 minutes after start of mixing. Use Arkansas stones, fine-grain diamonds, polishing discs of decreasing grain size, or silicone polishers for finishing. - Optionally, apply vaseline again to seal the surface.	
---	--

Storage and Shelf Life Store the capsules in the blister packs. Store the product at 15–25 °C /59–77 °F. Do not use after the expiry date.	
--	--

Customer Information No person is authorized to provide any information which deviates from the information provided in this instruction sheet.	
---	--

Warranty 3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for use as intended. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.	
---	--

Limitation of Liability Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental, or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.	
---	--

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