

DURELON[®]/DURELON FAST SET TECHNIQUE



STEP 1:
Remove provisional
restoration.



STEP 2:
Thoroughly clean
preparation with water.




STEP 3:
Gently dry preparation.
Do not over dry.



STEP 4:
Dry the restoration with air.



STEP 5:
Dispense one spoonful of
powder onto mixing pad.

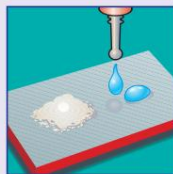

STEP 6:
Dispense liquid.



STEP 7:
Mix powder and liquid.



STEP 8:
Place a thin, even coat of
cement on inner surface of
restoration. Do not use
the mixture if it has lost
its gloss.



STEP 9:
Seat the restoration
without pressure.



STEP 10:
Allow cement to set,
then remove excess.