3M ESPE	
Durelon®	
Carboxylate Cement	
Стоматологичен карбоксилатен цемент	
Карбоксилатен цемент	
Karboksilatni cement	
Karboxilat cement	
Cement karboxylovy	
Ciment policarboxilic	
Karboxylatový cement	
Karboksilatni cement	
Karboxylatový cement	
Karboksilat Siman	
Karboksülaattsement	
Karboksiläta cements	
Karboksilatintis cements	

Instructions for Use
Инструкция по применению
Указания за употреба
Návod k použití
Navodila za uporabo
Návod k použití
Kullannma Talimatları
Kasutusjuhend
Lietošanas instrukcija
Instrucţiuni de Utilizare
Vartojimo instrukcija

3M Deutschland GmbH Dental Products Carl-Schurz-Str. 1 41463 Neuss - Germany	CE 0123
3M ESPE Dental Products St. Paul, MN 55144-1000 U.S.A.	4400183203101
3M, ESPE and Durelon are trademarks of 3M Company or 3M Deutschland GmbH. Used under license in Canada. © 2012, 3M. All rights reserved.	

ENGLISH

Product Description
Durelon® is a carboxylate cement for fixation and lining. Two variants of the powder, "normal curing" and "fast curing", and two consistencies of the liquid, "universal" and "low viscosity", are available. Two delivery versions of the "universal" liquid, dropper bottle and dosing syringe, are available. The "low viscosity liquid" is available with a dropper bottle only.

Either of the powder variants and the "universal" liquid may be used for lining. Any of the combinations can be employed for fixation.

⚠ These Instructions for Use should not be discarded for the duration of product use.

Indications

- Cementation of inlays, onlays, crowns, and bridges made from metal or metal-ceramics or veneered with composite
- Cementation of inlays, onlays, crowns, and bridges made from composite or ceramics provided these are suitable for conventional cementing
- Cementation of pins and screws provided these are suitable for conventional cementation
- Cementation of orthodontic bands
- Linings

Durelon is not suitable for cementation from/to titanium-based restorations, since cement discoloration can occur on the points which come into contact with titanium.

Precautionary Measures
3M MSDS can be obtained from www.mmm.com or contact your local subsidiary.

Preparation
► To provide for optimal adhesion, carefully clean enamel, dentin, and the restoration to be cemented.

► Blow the cavity dry in only 2-3 intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry.

► **The cavity should be just so dry that the surface has a semi-gloss appearance.** As is the case with every type of cement, overdrying can result in post-operative sensitivity.

► Dry the surfaces of the restoration with air.

► Avoid further contamination by saliva.

Pulp protection
► Holding the bottle vertically dispense single droplets next to the powder by lightly squeezing the bottle. Do not immediately after use.

Dosing
Powder:
► Assemble the dosing rod prior to use.
► Press the dosing rod into the powder, and then remove excess powder with some instrument.

► Dose the powder onto a mixing block or glass plate by lightly tapping the dosing rod.
► Avoid under-dosing the powder (especially in free dosing), since this may impair the quality of the cement.

Liquid Dropper Bottle:
► Holding the bottle vertically dispense single droplets next to the powder by lightly squeezing the bottle.
► Reseal the bottle immediately after use.

Liquid Dosing Syringe:
► Insert the enclosed plunger and dose the desired number of scale units.
► Keep the plunger for subsequent use with other syringes.
► Reseal the dosing syringe immediately after use.

Mixing ratio
1 dose powder = 1 dosing rod filling.
1 dose liquid = 1 drop from the bottle or 1 small scale unit from the dosing syringe.

Indication	Powder		Liquid	
	normal setting	fast setting	universal	low viscosity
Number of doses for fixations	1	–	2	–
	–	1	2	–
	1	–	–	1
Number of doses for linings	1	–	1	–
	–	1	1	–

The following powder/liquid mixing ratios apply:

for fixations: 0.15 g powder : 0.10 g liquid universal or 0.20 g powder : 0.10 g liquid low viscosity,

for linings: 0.25 g powder : 0.10 g liquid universal.

These quantities are not identical to the quantities required for one application!

Mixing
► Mix Durelon at an ambient temperature of 20-25°C/68-77°F with a metal spatula.
► Place all powder into the liquid in one move and mix until a homogeneous consistency is attained.

Application
► Protect the working area from water and saliva during the application and setting phase.
► Coat neighboring teeth or crowns with Vaseline to make removal of excess material easier.
► Apply a thin layer of cement to the internal surface of the crown and the prepared tooth.
► Do not overfill the crown as hydrostatic pressure in tightly fitting crowns may lead to discolour of the pulp.
► Then seal the restoration.
► Wash meat instruments with cold water before Durelon has set.

Times
The following times are valid at an ambient temperature of 23° C/73° F and 50% relative humidity:

Mixing min:sec	Processing incl. mixing min:sec	Setting from start of mixing min:sec
Fixation		
Powder, normal setting Liquid, universal	00:30	02:30 10:00
Powder, normal setting Liquid, low viscosity	00:30	02:00 07:00
Powder, fast setting Liquid, universal	00:30	01:00 05:00
Lining		
Powder, normal setting Liquid, universal	00:30	01:30 06:00
Powder, fast setting Liquid, universal	00:30	00:45 03:30

Processing times are indicated at higher temperatures, and prolonged at lower temperatures than those stated (e.g., mixing on a chilled glass plate). Using a larger quantity of powder also shortens the processing time. Exceeding these processing times (apparent by the matte surface of the paste) leads to a loss of adhesion to enamel and dentin.

Removing Excess Material

► To simplify cleaning, remove any excess material shortly before definitive setting of the material using an instrument. Be careful to avoid pulling any material from the seam.

► After the setting time has elapsed remove any remaining excess material, e.g., with a probe.

After Removal of Excess and Setting

► Examine carefully the sulcus of the treated teeth and the surrounding area; remove any remaining cement residues.

Notes

- Use the liquid directly prior to mixing to avoid the evaporation of ingredients.
- Powder and liquid bottles and the dosing syringe should be carefully resealed after use.

Storage and Shelf-life
Store the product at 15-25° C/59-77° F. Do not use after the expiration date.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability
Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. Information valid as of February 2012

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