Prior to use, carefully read the instructions for use.
EN
G-CEMTM Veneer
LIGHT-CURED ADHESIVE RESIN CEMENT

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

Adhesive luting of ceramic and composite veneers, inlays and onlays with a thickness of < 2 mm, and orthodontic resin composite veneers that provide the complete light-curing of the material.

CONTRAINDICATIONS

1. Direct contact of the product in patients with known allergies to methacrylate monomer or methacrylic polymer.

HOW TO USE G-CEM Veneer SYRINGE

a) Remove the wing cap and remove the wing tip by turning counter-clockwise.
b) Place the wing cap onto the dispensing tip and attach the dispensing tip to the syringe body.
c) Place the light protective cap onto the wing cap.
d) NOTE:

i. Take care not to attach the wing cap too tightly. This may damage the threads.
ii. Take care that any possible air is trapped in the dispensing tip when you attach the dispensing tip to the syringe body.
iii. Place the light protective cap onto the wing cap.

e) After use, immediately remove and dispose the wing cap.

PREPARATION PROCEDURE OF VENEERS, INLAYS, ONLAYS

a) Remove the temporary restoration and clean the prepared tooth using a water spray.

b) If necessary, use G-CEM Try-In Paste for trial.

c) Remove the temporary restoration and thoroughly rinse the final preparation.

d) Verify curing and remove wing cap by turning counter-clockwise.

e) Place the wing cap onto the dispensing tip and attach the dispensing tip to the syringe body.

f) Place the light protective cap onto the wing cap.

NOTE:

i. Take care not to attach the wing cap too tightly. This may damage the threads.

ii. Take care that any possible air is trapped in the dispensing tip when you attach the dispensing tip to the syringe body.

iii. Place the light protective cap onto the wing cap.

iv. After use, immediately remove and dispose the wing cap.

TRIAL FIT OF THE RESTORATION

a) Remove the temporary restoration and clean the prepared tooth using a water spray.

b) If necessary, use G-CEM Try-In Paste for trial.

c) Remove the temporary restoration and thoroughly rinse the final preparation.

d) Verify curing and remove wing cap by turning counter-clockwise.

e) Place the wing cap onto the dispensing tip and attach the dispensing tip to the syringe body.

f) Place the light protective cap onto the wing cap.

NOTE:

i. Take care not to attach the wing cap too tightly. This may damage the threads.

ii. Take care that any possible air is trapped in the dispensing tip when you attach the dispensing tip to the syringe body.

iii. Place the light protective cap onto the wing cap.

iv. After use, immediately remove and dispose the wing cap.

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