Instructions for use

myQuickmat Prime kit

sectional
matrix
system

en - ENGLISH	2
de - DEUTSCH	5
fr - FRANÇAIS	8
it - ITALIANO	11
es - ESPAÑOL	14
pt - PORTUGUÊS	17
no - NORSK	20
et - EESTI KEEL	23
fi - SUOMI	26
sv - SVENSKA	29

da - DANSK	32
cs - ČEŠTINA	35
tr - TÜRKÇE	38
el - ΕΛΛΗΝΙΚΑ	41
bg - БЪЛГАРСКИ	44
hu - MAGYAR	47
lv - LATVIEŠU VALODA	50
lt - LIETUVIŲ KALBA	53
ro - LIMBA ROMÂNĂ	56
sk - SLOVENČINA	59

Medical Devices Kit for professional use only
All individual components are CE marked
Sectional matrix system

Instructions for use

Description

The myQuickmat Prime kit provides a simple and natural restoration of the proximal anatomy of both molars and premolars. The removal of the wedge as a separate element of the restorative procedure simplifies the overall procedure and reduces chair time. This kit contains myRing Prime, a sectional matrix ring with integrated wedge for easy restoration of Class II posterior cavities. Thanks to its wedge-shaped extremities, myRing Prime provides efficient bi-lateral separation, good cervical sealing and good matrix adaptation on both molars and premolars and between premolars and molars. The kit includes the Quickmat Prime sectional matrices, which allow accurate positioning of the contact point near the equator in Class II restorations. The central top tab is useful for easy positioning of the Quickmat Prime matrices on both molars and premolars. Optimal reconstruction can be achieved by using myRing Prime with Quickmat Prime sectional matrices. However, all other Polydentia sectional matrices can be used with myRing Prime. Similarly, Quickmat Prime sectional matrices can be used with other sectional matrix rings (myRing Forte, myRing Classico).

Composition

myRing Prime: Stainless steel | myTines Prime: thermoplastic | Quickmat Prime sectional matrices: Stainless steel

Indications

myQuickmat Prime kit is indicated for Class II restoration on both molar and premolar in order to create contact point close to the tooth equator.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to myQuickmat Prime kit is necessary.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure myRing Prime using dental floss. In case of accidental inspiration of a myQuickmat Prime kit component (matrix, ring and its extremities) seek immediate medical attention. In case of ingestion of a myQuickmat Prime kit component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, disinfection and Sterilization Instructions at www.polydentia.ch.

Application

① **Prepare the cavity:** Remove the carious lesion and prepare the tooth with a conservative Class II preparation. For an easiest placement of the matrix, we suggest to prepare a fully open contact. ② **Place the matrix:** Use Quickmat Prime sectional matrices for standard Class II cavities on both molars and premolars. Place the contoured sectional matrix with the central tab towards the occlusal margin using your fingers or with tweezers. For an easy placement, we suggest to grab the central tab of the matrix with the myQuickmatrix Forceps and slide it in place from the occlusal direction. In case of deep cavities, a matrix with cervical extension can be used instead of Quickmat Prime sectional matrices. ③ **Place myRing Prime:** Grasp the ring differently according to the forceps used: 3a - with myForceps use the recess, 3b - with rubber dam forceps use the holes. Enlarge it just enough to position it on the tooth. Carefully place myRing Prime as far gingivally as possible. ④ **Burnish the matrix:** For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezers. ⑤ **Restore the cavity:** Restore the cavity according to the filling material manufacturer's instructions. ⑥ **Remove the sectional matrix system:** Remove myRing Prime and the matrix. Our suggestion for an easy matrix removal is to use Polydentia myQuickmatrix Forceps. ⑦ **Finishing and polishing:** With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations.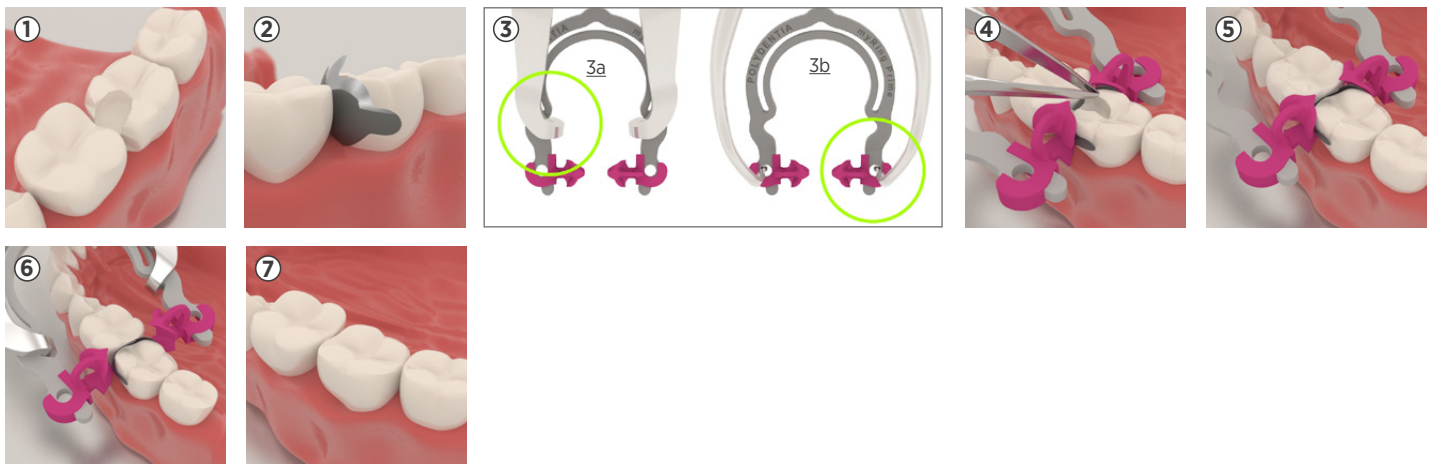

Disposal

Because of their composition, matrices can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

Ring reconditioning

For a proper, effective and safe reconditioning of the myRing Prime and their extremities read the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Tips & tricks using myQuickmat Prime kit


Improving matrix band adaptation

After myRing Prime has been placed, in some cases it may be necessary to improve the adaptation of the matrix in the cervical region. In this case, a Teflon tape can be used. With the ring still in place, insert the Teflon tape between the tips of the two tines and press it tightly against the matrix until the desired cervical fit is achieved.

Tines removal ^(A) ^(B)

Tines are replaceable and supplied in pairs. Tines must be disassembled before cleaning and sterilizing. To remove the tines correctly, it is important to preserve the internal anatomical wings. Make sure to pull the tines by grasping the external part.

✓ Pull from here. ✗ Do NOT pull from here.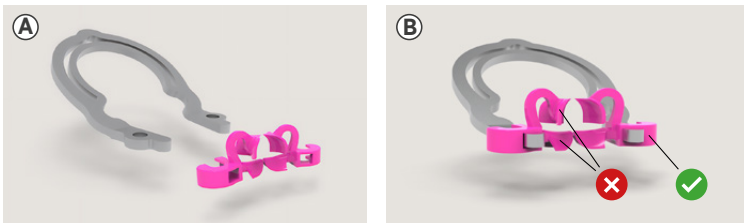


Plastic components are always subject to wear and tear after prolonged use. Always inspect myTines Prime before each use. Replace myTines Prime as soon as signs of damage are visible. Damaged medical devices may no longer be used and must be discarded.

MOD and multiple tooth restorations

When restoring MOD, we recommend restoring the mesial and distal walls as separate proximal cavities.

myRing Prime Reset ^(C) ^(D)

If myRing Prime is over-expanded it can be retensioned by using Polydentia's myForceps as follows: **1.** remove the tines first. **2.** place myRing Prime within the notches on the forceps. **3.** squeeze forceps closed to return myRing Prime in its perfect shape. ✕ Squeeze.

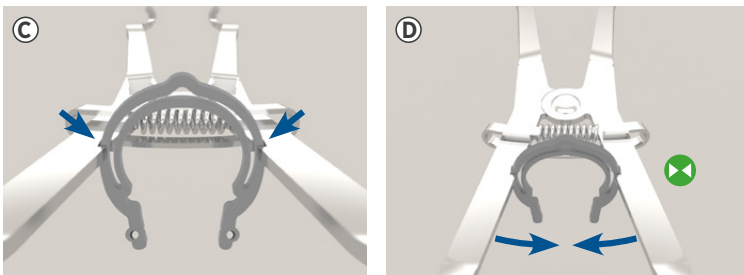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