Instructions for use

myClip 2.0

sectional
matrix
systems

Index

EN - English	2	SE - Svenska	20
DE - Deutsch	4	DK - Dansk	22
FR - Français	6	PL - Polski	24
IT - Italiano	8	CZ - Čeština	28
ES - Español	10	TK - Türkçe	30
PT - Português	12	GR - ελληνικά	32
NL - Nederlands	14	JP - 日本語	34
NO - Norsk	16	CN - 中国	36
FI - Suomi	18	AR - العربية	38

Class II restoration using myClip 2.0

myClip 2.0 is an all in one sectional matrix ring system with integrated forceps that allows for one-handed placement, simply by using the fingers. MyClip 2.0 used in combination with the wooden wedges provides tooth separation, additionally its anatomically shaped, reusable tines ensure perfect matrix adaptation.

Instructions for use:

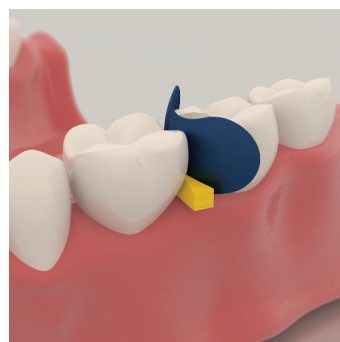
1. Prepare the cavity

Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed.

2. Place the matrix

Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers.


3. Wedging

Select the appropriate sized wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging.


4. Place myClip 2.0.

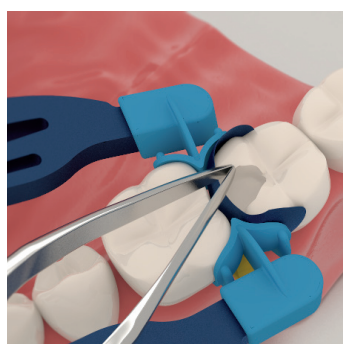Expand myClip 2.0 just by using your fingers, no forceps are needed. Enlarge it just enough to position it on the tooth. Place carefully myClip 2.0 over the wedge and place it as far gingivally as possible.


5. Burnish the matrix

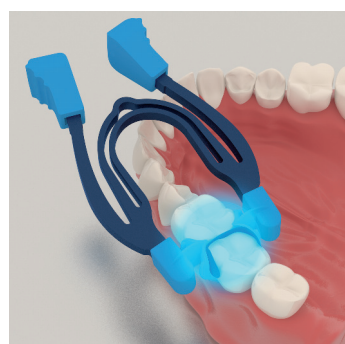For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezers.

6. Restore the cavity

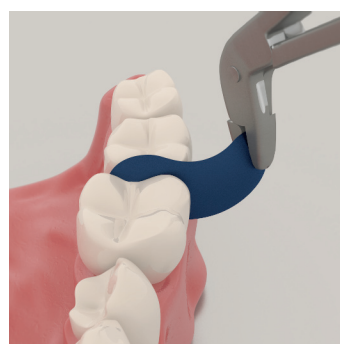Restore the cavity according to the filling material manufacturer's instructions.

7. Remove the sectional matrix system

Remove myClip 2.0, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps.

8. Finishing and polishing

With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations.


Recommendations:

- 1 - Place a rubber dam
- 2 - Read Cleaning, Disinfection and Sterilization Instructions at www.polydentia.com

Tips & Tricks using myClip 2.0


Restoration of adjacent cavities

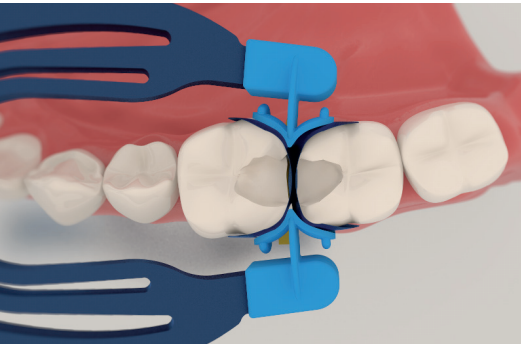MyClip 2.0 tines are shaped to perfectly adapt the mesial and distal surfaces of two adjacent teeth. This feature allows the restoration of two adjacent cavities at the same time. Prepare the cavities as described in the instructions for use, step 1. Place matrices and wedges on both teeth being restored
Position myClip 2.0
Complete your restoration as described in the instructions for use, steps 5 to 8.



Tines removal

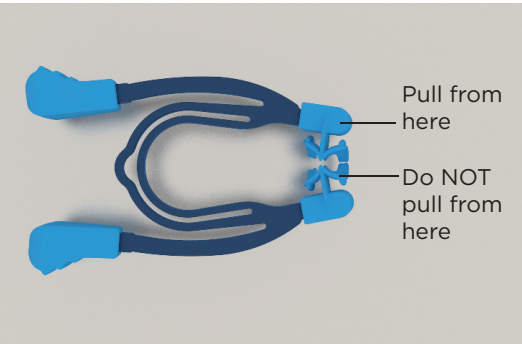Tines must be disassembled before any cleaning and sterilizing.
To remove the tines correctly, it is important to preserve the internal anatomical wings. Make sure to pull the tines by grasping the external part. For more information about cleaning and sterilizing refer to the document “cleaning and sterilization procedure”

Tines placement on myClip 2.0

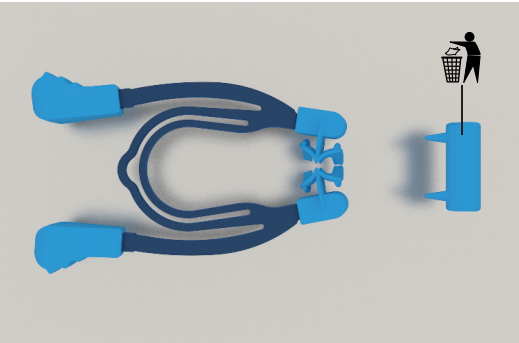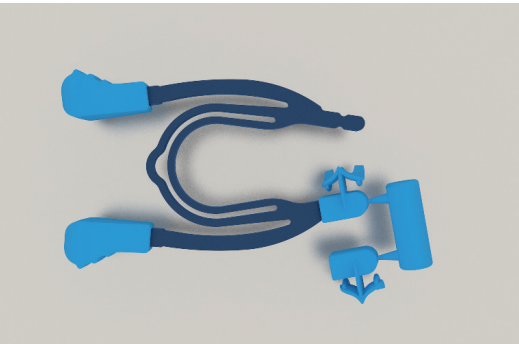Tines are replaceable and supplied in pairs. You can easily detach the tines. Avoid finger pressure on the internal anatomical wings to preserve them. Make sure tines are placed on myClip 2.0 as shown in the picture below.

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