Instructions for use

myClip

2.0

Includes
restorative
trial set

sectional
matrix
systems

en - ENGLISH	2
de - DEUTSCH	5
fr - FRANÇAIS	8
it - ITALIANO	11
es - ESPAÑOL	14
pt - PORTUGUÊS	17
no - NORSK	20
et - EESTI KEEL	23
fi - SUOMI	26
sv - SVENSKA	29

da - DANSK	32
cs - ČEŠTINA	35
tr - TÜRKÇE	38
el - ΕΛΛΗΝΙΚΑ	41
bg - БЪЛГАРСКИ	44
hu - MAGYAR	47
lv - LATVIEŠU VALODA	50
lt - LIETUVIŲ KALBA	53
ro - LIMBA ROMÂNĂ	56
sk - SLOVENČINA	59

Medical Devices Kit for professional use only
All individual components are CE marked
Sectional matrix systems

Instructions for use

Description

All-in-one sectional matrix ring with short handles that act as forceps. There is no need for special placing tools as myClip 2.0 can be easily positioned with your fingers. myClip 2.0 features an optimal separation force that allows to create contact points in the restoration of Class II cavities, making it suitable for all clinical conditions. Thanks to the ring's plastic extremities (myTines Small, myTines Medium and myTines Large), myClip 2.0 adapts to both teeth of different crown heights and wide cavity preparations. In case of dissimilar tooth morphologies, the myTines can be used in combination to enhance the matrix adaptation to the proximal walls and consequently reduce the need for finishing steps.

Composition

myClip 2.0: Stainless steel and thermoplastic | myTines: thermoplastic | LumiContrast matrices: Stainless steel | Wood wedge: Maple wood

Indications

myClip 2.0 is indicated to ensure matrix adaptation in order to create correct contact points in the restoration of class II cavities.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to myClip 2.0 is necessary.

Shelf life

Please refer to the expiration date on the packaging. Expiration date refers to wooden wedges.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure myClip 2.0 using dental floss. In case of accidental inspiration of a myClip 2.0 component (matrix, ring and its extremities, wedge) seek immediate medical attention. In case of ingestion of a myClip 2.0 component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, disinfection and Sterilization Instructions at www.polydentia.ch.

Application

① **Prepare the cavity:** Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed. ② **Place the matrix:** Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers. ③ **Wedging:** Select the appropriate sized wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging. ④ **Place myClip 2.0:** Expand myClip 2.0 just by using your fingers, no forceps are needed. Enlarge it just enough to position it. ⑤ **Burnish the matrix:** For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezers. ⑥ **Restore the cavity:** Restore the cavity according to the filling material manufacturer's instructions. ⑦ **Remove the sectional matrix system:** Remove myClip 2.0, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps. ⑧ **Finishing and polishing:** With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations.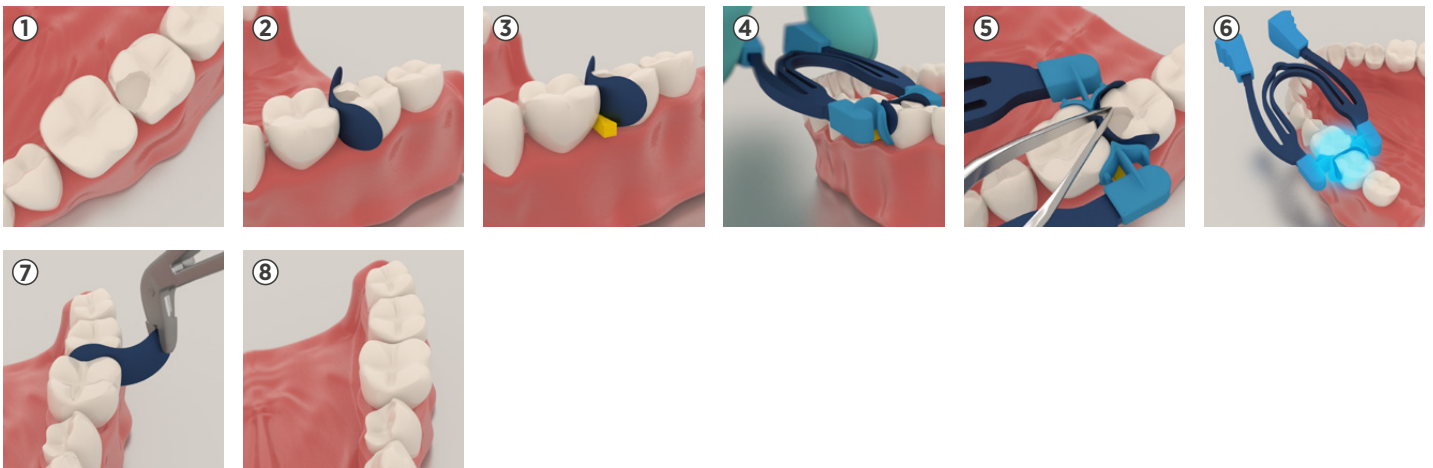

Disposal

Because of their composition, matrices and wedges can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.


Ring reconditioning

For a proper, effective and safe reconditioning of the myClip 2.0 and their extremities read the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Tips & tricks using myClip 2.0


Restoration of adjacent cavities ①

myClip 2.0 tines are shaped to perfectly adapt the mesial and distal surfaces of two adjacent teeth. This feature allows the restoration of two adjacent cavities at the same time. **1.** Prepare the cavities as described in the instructions for use, step 1. **2.** Place matrices and wedges on both teeth being restored. **3.** Position myClip 2.0. **4.** Complete your restoration as described in the instructions for use, steps 5 to 8.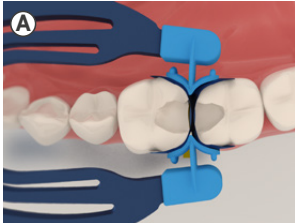




Tines removal ②

Tines must be disassembled before cleaning and sterilizing. To remove the tines correctly, it is important to preserve the internal anatomical wings. Make sure to pull the tines by grasping the external part. Pull from HERE. DO NOT pull from HERE.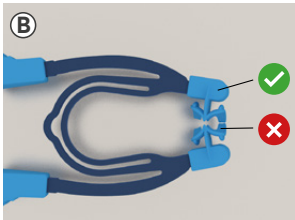

Tines placement on myClip 2.0 ③ ④

Tines are replaceable and supplied in pairs. You can easily detach the tines. Avoid finger pressure on the internal anatomical wings to preserve them. Make sure tines are placed on myClip 2.0 as shown in the picture. **⑤ CORRECT POSITIONING:** The v-shaped section of myTines must face downwards to accommodate the wedge. **CORRECT DISASSEMBLY:** Remove myTines by pulling from their main body. Do not pull from their wings.

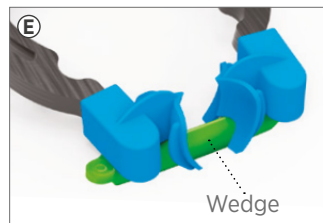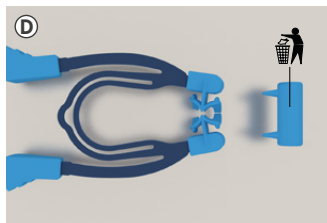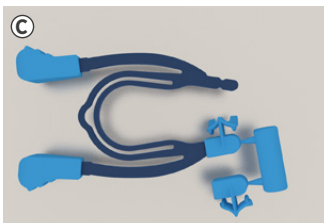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