Instructions for use

myQuickmat classico starter kit

sectional
matrix
systems

en - ENGLISH	2	da - DANSK	32
de - DEUTSCH	5	cs - ČEŠTINA	35
fr - FRANÇAIS	8	tr - TÜRKÇE	38
it - ITALIANO	11	el - ΕΛΛΗΝΙΚΑ	41
es - ESPAÑOL	14	bg - БЪЛГАРСКИ	44
pt - PORTUGUÊS	17	hu - MAGYAR	47
no - NORSK	20	lv - LATVIEŠU VALODA	50
et - EESTI KEEL	23	lt - LIETUVIŲ KALBA	53
fi - SUOMI	26	ro - LIMBA ROMÂNĂ	56
sv - SVENSKA	29	sk - SLOVENČINA	59

Medical Devices Kit for professional use only
All individual components are CE marked
Sectional matrix systems

Instructions for use

Description

myQuickmat Classico starter kit: sectional matrix system developed to offer gentle tooth separation in order to create contact points in the restoration of class II cavities. This kit features Polydentia's myRing Classico, a classic sectional matrix ring made of nickel titanium. It is reinforced to provide an optimal and gentle separation force in combination with interdental wooden wedges (not included). For the restoration of very extensive cavities, the kit also includes Silicone Tubes and Diamond24. These anatomically shaped silicone tubes ensure optimal matrix retention and adaptation to the tooth's anatomy. Quickmat sectional matrices complete the kit's assortment.

Composition

myRing Classico: NiTi Ring with thermoplastic overmolding | Silicone Tubes: silicone | Diamond24: silicone | Quickmat matrices: Stainless steel

Indications

myQuickmat Classico starter kit is indicated for gentle tooth separation in order to create correct contact points in the restoration of class II cavities.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to myQuickmat Classico starter kit is necessary.

Shelf life

Please refer to the expiration date on the packaging. Expiration date refers to silicone tubes.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure the rings using dental floss. In case of accidental inspiration of a myQuickmat Classico starter kit component (matrix, retaining ring and its extremities, wedge) seek immediate medical attention. In case of ingestion of a myQuickmat Classico Kit component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Application

① **Prepare the cavity.** Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed. ② **Place the matrix.** Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers. ③ **Wedging.** Select the appropriate sized wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging. ④ **Place myRing Classico.** Expand myRing Classico using Polydentia's myForceps or standard rubber dam forceps. Place the ring carefully and securely. Please refer to the Tips & Tricks section to see the different ways to position myRing Classico and ensure best teeth separation as well as matrix adaptation. ⑤ **Burnish the matrix.** For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezers. ⑥ **Restore the cavity.** Restore the cavity according to the filling material manufacturer's instructions. ⑦ **Remove the sectional matrix system.** Remove myRing Classico, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps. ⑧ **Finishing and polishing.** With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations.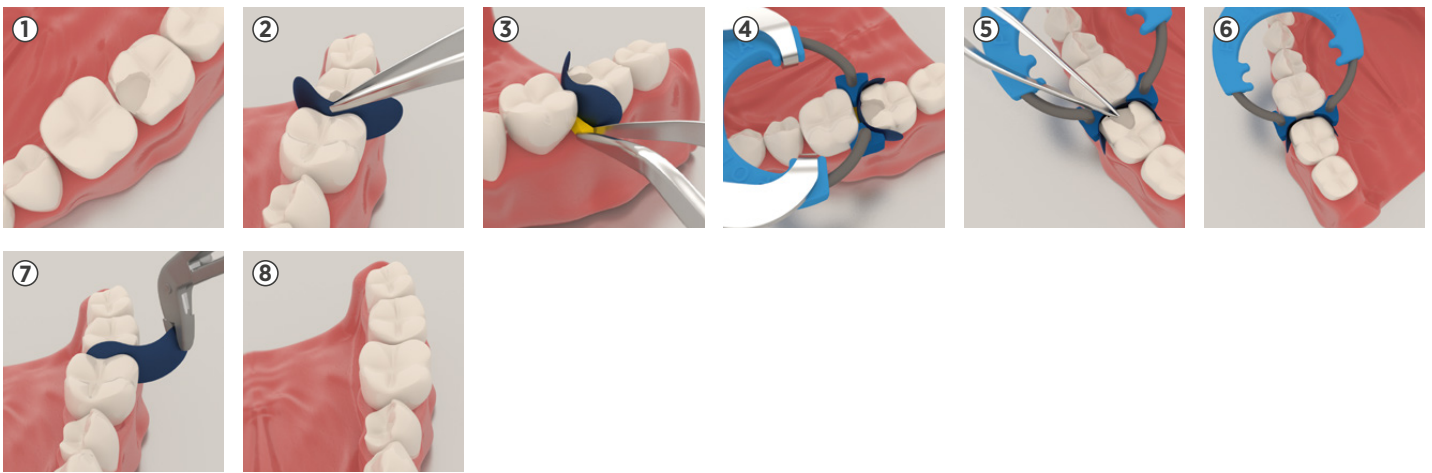

Disposal

Because of their composition, matrices can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.


Ring reconditioning

For a proper, effective and safe reconditioning of the myRing Classico and their extremities read the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Tips & tricks using myQuickmat Classico starter kit

Wedging [Ⓐ]

According to the clinical situation, the wooden wedge can be broken in order to better accommodate myRing Classico with its extremities. This trick is very helpful in case of small teeth. ❌ NO break. ✅ Break.

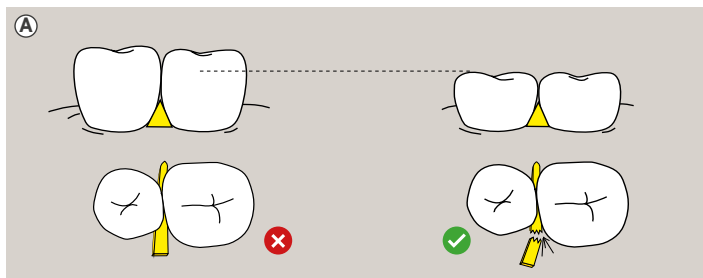myRing Classico tines placed near to the wood wedge

This placement technique is suitable for cases of a wide space in the interproximal area

myRing Classico tines placed over the wood wedge

This placement technique is suitable if the interproximal space is tight or when the cavity is deep

MOD and multiple tooth restorations [Ⓑ]

myQuickmat Classico rings are designed to be stacked over one another to perform a MOD and multiple tooth restorations. **1.** Prepare the cavities as described in the instructions for use, step 1. **2.** Place matrices and wedges on both mesial and distal surfaces of the tooth being restored. **3.** Position myRing Classico. **4.** Complete the restoration as described in the instructions for use, steps 5 to 8.

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