Neo Spectra[®] ST flow

Flowable Composite Restorative

Composite fluide de restauration

Composite restaurador fluido

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Neo Spectra® ST flow

Flowable Composite Restorative

CAUTION: This is a medical device. For dental use by dental professionals only.

USA: Rx only

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	6
4 Hygiene	10
5 Lot number, expiration date and correspondence	14

1 Product description

Neo Spectra® ST flow composite restorative is a visible light-cured, radiopaque composite resin restorative material with flow characteristics that make it ideal for use in anterior and posterior teeth.

Neo Spectra® ST flow composite restorative adapts to the cavity wall without the use of hand instruments. The unique "stackable" flow characteristics allow intimate adaptation without slumping.

Neo Spectra® ST flow is a hybrid composite possessing the high strength and wear resistance required by anterior and certain posterior restorations, combined with an easily achieved high surface luster and smoothness.

1.1 Indications

Neo Spectra® ST flow composite restorative is indicated for the following procedures:

- Direct restoration of cavities (except occlusal stress bearing Class II cavities), lesions.
- Filling of defects and undercuts in crowns, inlays, and onlays.
- Blockouts.
- Repair of defects.
- Pit and fissure sealants.
- Cementation of light transmissible indirect restorations.

1.2 Contraindications

- Neo Spectra® ST flow restorative is contraindicated for use in patients with a history of severe allergic reaction to methacrylate resins or any of the other components.

1.3 Delivery forms

Neo Spectra® ST flow composite restorative is available in:

- Pre-dosed Compula® Tips
- Syringes

1.4 Composition

Neo Spectra® ST flow composite restorative consists of:

- Pre-polymerized filler
- TEGDMA dimethacrylate multifunctional polymethacrylate
- Barium-aluminium-borosilicate glass
- Ytterbium fluoride
- Dimethacrylate resins
- Trimethacrylate
- Highly dispersed silicon dioxides
- UV stabilizer
- Ethyl-4(dimethylamino)benzoate (photoaccelerator)
- Butylated hydroxy toluene (BHT)
- Camphorquinone (photoinitiator)
- Titanium dioxide
- Iron oxide pigments
- Fluorescent agent

Filler system based on SphereTEC®.

The filler matrix of Neo Spectra® ST flow composite restorative consists of barium-aluminum-borosilicate glass, ytterbium fluoride and pigments (iron oxide and titanium dioxide), according to shade.

The filler system based on SphereTEC® fillers ranges from 0.1 to 3.0 µm.

Including catalysts, additives, stabilizers and pigments (~1 weight-%, according to shade), the total filler is:

62.5 weight-% or 40.0 volume-%.

1.5 Compatible adhesives

Neo Spectra® ST flow composite restorative is used following application of a suitable dentin/enamel adhesive and is chemically compatible with conventional (meth)acrylate-based dentin/enamel adhesives including Dentsply Sirona adhesives designed for use with visible light-cured composite restoratives (see complete Instructions for Use of selected adhesive).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Information symbol.

This information symbol is used to highlight important information for the correct use.

2.1 Warnings

1. This material contains methacrylates and polymerizable monomers that may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.
 - **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
 - **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, red-dish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
 - **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.
2. Neo Spectra® ST flow curing reaction is exothermic. Increased temperature may irritate pulp or soft tissue. Cover dentin close to the pulp (less than 1 mm) with a hard setting calcium hydroxide liner (Dycal® Liner) and/or treat according to adhesive manufacturer's instructions.
3. Neo Spectra® ST flow is light-cured only. Incomplete light-curing may lead to compromised physical properties, residual uncured monomer, post-operative sensitivity or premature bond/restoration failure. See [3.1.6 Light-curing](#) for wavelength, output and maximum increment requirements. Check output and curing cycle before each use.
4. When using a Compules® Tips Gun, dental professionals and patients should wear protective eyewear.
5. Neo Spectra® ST flow should not be used in patients with a known hypersensitivity to any of the components.

2.2 Precautions

This product is intended to be used as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Contact with saliva, blood or sulcus fluid during application may cause failure of the restoration. Use adequate isolation, such as rubber dam.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practices.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- The syringes cannot be reprocessed. To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, it is mandatory that the syringes are handled with clean/disinfected gloves. Discard syringes if contaminated.
- As additional precautionary measure, syringes may be protected from gross debris but not from all contamination by applying a protective barrier.
- Use of Compula® Tips with the Dentsply Sirona Compules® Tips Gun is recommended.
- For further information please refer to the Dentsply Sirona Compules® Tips Gun Instructions for Use.
- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may result in unanticipated extrusion of the material or cause the Compula® Tip to rupture or to eject from the Dentsply Sirona Compules® Tips Gun.
- Tightly close syringes immediately after use. Do not allow dispensing tip to remain as a syringe cap. Replace original cap tightly after each use. Discard and properly dispose dispensing tips after use, as the tips may clog if material is allowed to dry or set inside.
- Dentsply Sirona supplies the appropriate dispensing tip for the Neo Spectra® ST flow composite restorative material syringe. This tip is the only tip that should be utilized for placement of the material.

- Use only clean and properly disinfected and sterilized, or ready to use accessories. Do not use dispensers that have not been properly pre-use processed or reprocessed. Before use and each reuse, dispensers must be properly cleaned and autoclaved (see [4 Hygiene](#)).
- As additional precautionary measure, dispensers may be protected from gross debris, but not from all contamination, by applying a protective barrier.
- Compula® Tips are provided ready for use and should not be processed. Use only new, clean unused Compula® Tips, stored in covered storage, such as drawers and cabinets, away from potential contamination, and removed from their original packaging, using either clean gloved hands or a clean suitable instrument at time of use. To reduce risk of infection, do not use tips if packaging is damaged or compromised.
- Insufficient data exists to support the use of Neo Spectra® ST flow in Class I cavities with isthmus widths beyond central groove areas, or when replacing a functional cusp. Excessive wear or restoration failure may result.
- Insufficient data exists to support the use of Neo Spectra® ST flow in root canal fillings. Restoration failure may result.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product, since they may interfere with hardening of the product.
 - Neo Spectra® ST flow is a light-cured material. Therefore, it should not be exposed to high intensity lights sources (e.g., operatory light or LED lights of loupes). Proceed immediately once material has been placed.
 - If mineral-impregnated (e.g., ferric compounds) retraction cords and/or hemostatic solutions are used in conjunction with adhesive procedures, the marginal seal may be adversely affected, allowing microleakage, sub-surface staining and/or restoration failure. If gingival retraction is necessary, use of plain, non-impregnated cord is recommended.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see [2.1 Warnings](#)).

2.4 Storage conditions

- Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.
- Store in a well-ventilated place at temperatures between 2 °C and 28 °C (36 °F and 82 °F).
 - Keep out of direct sunlight and protect from moisture.
 - Do not freeze.
 - Do not use after expiration date.
 - Do not use if the lot number and/or expiration date are missing or illegible.

3 Step-by-step instructions

3.1 Direct application – cavities, lesions, repair

3.1.1 Shade selection

1. Prior to shade selection, clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
2. Select the shade while teeth are hydrated.

Neo Spectra® ST flow composite restorative has been scientifically designed to allow the tooth structure enamel and dentin to blend together with the composite with a lifelike translucent result. The universal CLOUD™ shades of Neo Spectra® ST flow composite restorative adequately reproduce the traditional 16 VITA®¹-shades.

- A1 (= VITA® shades A1, B1, C1)
- A2 (= VITA® shades A2, B2, D2)
- A3 (= VITA® shades A3, C2, D3, D4)
- A3,5 (= VITA® shades A3,5, B3, B4, C3)
- A4 (= VITA® shades A4, C4)

A Bleach White Shade (BW), as well as opaque dentin shades D1 and D3 and translucent enamel shade E1 are also available.

For shade selection, there are two tools that may be used:

- The VITA® classical shade guide in combination with the i-shade label. The Spectra ST i-shade label is applicable for the Neo Spectra® ST flow composite restorative and facilitates shade selection by assigning one of five universal CLOUD™ shades to each of the 16 VITA® shades (see table below).

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- The Spectra ST composite restorative shade guide consists of 9 shade tabs, which are manufactured from composite with optical properties close to those of Neo Spectra® ST flow composite restorative.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.

3.1.2 Cavity preparation

1. Prepare the cavity so that no residual amalgam or restorative is left (unless repairing an otherwise sound restoration).
2. Use adequate isolation such as rubber dam.
3. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

¹ Not a registered trademark of Dentsply Sirona, Inc.

3.1.3 Matrix placement and wedging

1. Place a matrix (e.g., Mylar, AutoMatrix® matrix system or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or Palodent® V3 ring placement is recommended.

3.1.4 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of restorative material.

3.1.5 Application – delivery

Using syringes

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands, it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove cap from the end of the syringe. To assure free flow of material from syringe, express a small amount onto pad, away from the patient field.
2. Attach disposable dispensing tip to end of the syringe. Turn tip clockwise 1/4 to 1/2 turn to assure that it is fully seated. Tug on tip to be sure that it is locked into the collar of the syringe.
3. Neo Spectra® ST flow composite restorative should flow freely with gentle pressure. **DO NOT USE EXCESSIVE FORCE.** If more than gentle pressure is required, remove from patient field and check for obstruction.
4. **DISCARD AND PROPERLY DISPOSE OF DISPENSING TIP IMMEDIATELY AFTER USE. REPLACE ORIGINAL CAP.** Do not store syringe with gray dispensing tip in place. **STORE ONLY WITH ORIGINAL CAP.**

Note: It is recommended to pull back slightly on the syringe plunger after use to prevent excessive flow of material.

Using Compula® Tips

Excessive force – To reduce risk of injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compula® Tip rupture or ejection from Compules® Tips Gun may result.

1. Insert Compula® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compula® Tip is inserted first.
2. Remove the colored cap from the Compula® Tip. The Compula® Tip may be rotated 360° to gain the proper angle of entrance into the cavity.
3. Dispense the material into the cavity preparation using a slow, steady pressure. **DO NOT USE EXCESSIVE FORCE.**
4. To remove the used Compula® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compula® Tip and remove.

3.1.5.1 Applying restorative

1. Dispense Neo Spectra® ST flow composite restorative directly into cavity preparation/tooth surface from the Compula® Tip or syringe tip using slow, steady pressure. Avoid lifting the tip while dispensing to minimize air entrapment. Any visible air bubbles should be pierced with a clean, sharp explorer prior to curing. Material may be placed and light-cured in increments up to 2 mm. In deeper preparations it is recommended to double the curing time to compensate for lower irradiance (see [3.1.6 Light-curing](#)). For maximum light transmission and to minimize finishing, avoid overfilling cavity.

3.1.6 Light-curing

1. Light-cure each area of the restoration surface with a suitable visible light-curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e., peak spectral output 440-480 nm. With irradiance of at least 800 mW/cm², but not higher than 2000 mW/cm², cure universal CLOUD™ shades, BW, and E1 increments for 10 seconds, and D1 and D3 shades for 30 seconds.

When minimum irradiance is between 550 mW/cm² and 800 mW/cm² cure universal CLOUD™ shades, and BW for 20 seconds, E1 for 10 seconds, and D1 and D3 shades for 40 seconds.

Curing time for 2 mm increments

Neo Spectra® ST flow Curing Recommendations		
Shades	Irradiance mW/cm ²	Curing Time
A1-A4 and BW	≥ 550 and < 800	20 seconds
	≥ 800 and ≤ 2000	10 seconds
D1, D3	≥ 550 and < 800	40 seconds
	≥ 800 and ≤ 2000	30 seconds
E1	≥ 550 and ≤ 2000	10 seconds

Insufficient curing – To reduce risk of inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.
5. Check distance to surface to be cured.

3.1.7 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance® finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance® and Enhance® PoGo®, finishing and polishing instruments or Prisma® Gloss™ Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

All shades of Neo Spectra® ST flow composite restorative are radiopaque, with a 1 mm radiopacity equivalent to 1.8 mm radiopacity of aluminum. Aluminum has a radiopacity equivalent to that of dentin. Thus 1 mm of material having a radiopacity equivalent to 1 mm of aluminum has a radiopacity equivalent to that of dentin.

3.2 Cementation of light transmissible indirect restorations

1. Following removal of the temporary restoration and any remaining temporary cement, clean enamel and dentin as directed by the adhesive manufacturer's instructions.
2. Check the fit and esthetics of the restoration.

Technique Tip: Occlusal adjustment of veneers and onlays is best accomplished after final cementation.

3.2.1 Treatment of the restoration

Ceramic/Composite Restorations

Follow the dental laboratory or restoration manufacturer's instructions for pre-treatment, if required. Restorations designed to be silanated or if the internal silanated surface has been disturbed during try-in, apply Calibra® Silane Coupling Agent (available separately) according to the manufacturer's instructions.

3.2.2 Tooth conditioning/dentin pretreatment/adhesive application

1. Proceed with adhesive application as directed in the complete Instructions for Use.

Delayed or reduced adhesion – To reduce risk of adhesive incompatibility.

1. Select compatible light-cure adhesive.
2. Follow selected adhesive Instructions for Use.

2. Always refer to and follow adhesive manufacturer's complete Instructions for Use for selection and application of the appropriate light-cure adhesive. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately following adhesive application to cementation with Neo Spectra® ST flow material.

3.2.3 Cementation (light-cured)

Before proceeding refer to [3.2.1 Treatment of the restoration](#) and [3.2.2 Tooth conditioning/dentin pretreatment/adhesive application](#).

- Adhesives compatible with light-cured, CQ initiated methacrylates are recommended.
 - If restorations exceed 1.0 mm thickness, are heavily opaqued, or otherwise impede the transmission of light, operator is urged to select Calibra® Universal (available separately) and follow the dual-cure cementation technique.
 - Apply adhesive to the internal bonding surface of the restoration if directed by adhesive manufacturer's Instructions for Use.
1. Dispense the desired shade of Neo Spectra® ST flow material from the syringe directly onto the restoration. Protect cement from exposure to light.
 2. Seat the loaded restoration in place. Remove excess from the margins with a blunt instrument. Briefly (10 seconds or less) light-cure the gingival portion only to tack restoration in place. Remove any remaining excess from margins.

Technique Tip: Placement of mylar strips between preparation and adjacent teeth prior to seating veneers aids in isolation and in excess cement clean up. After removing gingival excess and tacking veneer in place, remove interproximal excess cement by pulling mylar strip towards the facial, i.e., from tooth to restoration surface.

3.2.4 Curing

Insufficient curing – To reduce risk of inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.
5. Check distance to surface to be cured.

1. Light-cure with a suitable visible light-curing unit designed to cure materials containing camphorquinone (CQ) initiator, i.e., peak of spectrum in the range of 440-480 nm and irradiance of 800-2000 mW/cm². Cure each area of the restoration surface (buccal, lingual, and interproximal) for at least 20 seconds. To ensure proper irradiance follow curing light instructions (e.g., increase exposure time to compensate lower irradiance due to increased distance between light guide/tip and restorative surface).
2. After light-curing, check and adjust occlusion as necessary. Proceed to finishing and polishing as outlined in section 3.1.7 Finishing and polishing.

4 Hygiene

Cross-contamination – To reduce risk of infection.

- Do not reuse single use products. Dispose of in accordance with local regulations.
- Reprocess reusable products as described below.

4.1 Syringes – cross-contamination

Cross-contamination – To reduce risk of infection.

- The syringes cannot be reprocessed.
- To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.
- Dispose of contaminated syringes in accordance with local regulations.

To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands, or oral tissues, use of a protective barrier is recommended. The use of protective barriers is an additional precautionary measure against gross debris but not against all contamination.

Incidental contact of the syringe with water, soap or a water-based hospital-level disinfection solution will not damage syringe body. Do not allow any solution contact with contained material. Discard composite material that has been in contact with any fluid or non-sterile instrument.

If needed, dry the syringe with a lint-free single-use cloth.

Vigorous wiping can destroy the label.

Wipe syringe gently.

4.2 Compules® Tips Gun

For reprocessing instructions please refer to the Instructions for Use of the Compules® Tips Gun, which is available on our webpage at <https://www.dentsplysirona.com/ifu>. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. In the USA, call 1-800-532-2855. Outside the USA, use the order form provided on <https://www.dentsplysirona.com/ifu> for this purpose.

4.3 Compula® Tips – cross-contamination

Cross-contamination – To reduce risk of infection.

1. Do not reuse Compula® Tips. Dispose of Compula® Tips in accordance with local regulations.

4.4 Processing instructions for the shade guide, shade tabs and holder

Processing instructions for the shade guide, shade tabs and holder

Warnings	<ul style="list-style-type: none">• High-level disinfection procedures have not been validated.• Intermediate-level disinfection is appropriate for the holder.• For individual shade tabs, proceed to sterilization following cleaning and any optional/intermediate disinfection process.• Steam autoclave sterilization is appropriate and recommended for the individual shade tabs.• Do not autoclave the holder in steam autoclave.• Do not clean/disinfect the holder in automated washer/disinfector.• Follow proper infection/prevention activities, such as proper hand washing and donning new puncture and chemical resistant gloves at pertinent steps.• Do not use hard wire brushes to clean the device, as scratching and mechanical degradation can occur.• Use only a disinfecting solution that has been approved for its efficacy, is EPA registered (and/or Health Canada approved) and use in accordance with the Instructions for Use of the disinfecting solution's manufacturer.• Do not allow the shade tabs to exceed 137 °C (279 °F).• Do not use phenol-based glutaraldehyde.• Always use a pH neutral instrument cleaning solution, if authorized.• If the integrity of the packaging pouch has been compromised, the device should be processed again before use.• The shade tabs should be cleaned and sterilized before each use. For the countries that require a three-step process, in which disinfection is required prior to sterilization, the shade tabs must be cleaned, disinfected, and sterilized before use.
Limitations on reprocessing	<ul style="list-style-type: none">• Repeated reprocessing has minimal effect on these instruments. End of life is normally determined by wear and damage due to use.• Use of the device beyond its useful life may cause damage to equipment and increases the risk of patient cross-contamination.• Cold liquid sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.

Initial treatment at the point of use	<ul style="list-style-type: none"> • Immediately after use, remove exposed individual shade tabs from the exposed holder for processing. • Do not allow residue or any form of contamination to dry on the device. Remove excess soil/gross debris by wiping, brushing, and/or rinsing under water. • Transport to processing area and follow validated processing instructions below. • It is recommended that the device be processed as soon as is reasonably practical following use. • Start reprocessing within 1 hour after use.
Preparation before cleaning	<ul style="list-style-type: none"> • Always disassemble the individual tabs from the holder before processing.
Cleaning: Mechanical	<ul style="list-style-type: none"> • The chemical manufacturer's instructions for the cleaning and disinfection solution should be followed, observing concentration rates, contact times, fill level, degassing processes, etc. • The recommended solution is Resurge® Instrument Cleaning Solution mixed to 14.8 ml/3.8 l (0.5 oz/gal), at 35 °C-40 °C (95 °F-104 °F) or equivalent enzymatic cleaning solution. • Immerse tabs and holder in an ultrasonic bath containing the prepared solution for at least 20 minutes, unless otherwise specified by the time specified by the solution manufacturer. • Rinse under tap water of at least drinking quality for a minimum of 30 seconds. • Allow to air-dry. • Visually inspect for visible soil, and repeat the cleaning and disinfection procedure if necessary.
Cleaning: Manual	<ul style="list-style-type: none"> • Prepare instrument cleaning solution per the manufacturer's recommendation. • The recommended solution is Resurge® Instrument Cleaning Solution mixed to 14.8 ml/3.8 l (0.5 oz/gal), at 35 °C-40 °C (95 °F-104 °F) • Using the cleaning solution per the label directions, brush with a soft nylon brush, paying particular attention to seams and crevices. • Rinse under tepid tap water of at least drinking quality (35 °C-40 °C (95 °F-104 °F)) for a minimum of 30 seconds. • Allow to air-dry. • Visually inspect for visible soil, and repeat the cleaning procedure if necessary. • Note: European markets may use FD 370 Cleaner, Dürr Dental AG, Germany.
Disinfection: Manual	<p>Disinfection by Wiping</p> <ul style="list-style-type: none"> • To disinfect the device, thoroughly wipe all surfaces, with a new single-use wipe in combination with an alcohol-based, tuberculocidal, quaternary ammonium solution (41.58% IPA, 0.24% ammonium compounds). • Use additional wipes as needed to ensure the entire outer surface, including seams and crevices, remain visibly wet for the contact time recommended by the solution manufacturer. • Pay special attention to all seams and crevices while wiping. • Rinse the delivery device under tepid running water (40 °C (104 °F), potable) for 30 seconds to remove disinfectant solution. • Dry the device using a dry, lint-free disposable cloth or disposable wipe.

Cleaning and disinfection: Automated	<ul style="list-style-type: none"> • Note: The following instructions apply only to shade tabs; do not use a thermal disinfectant for the holder, as it may become distorted or compromised. • Note: Remove the individual shade tabs for automated cleaning and disinfection. • A thermal disinfectant that complies with ISO 15883 may be used for automatic cleaning and disinfection. • Use only properly maintained, calibrated, and approved washer disinfectors according to ISO 15883. • Follow the manufacturer's instructions to select the solutions and programs. • Remove excess soil with a disposable cloth/paper wipe. • In case of gross contamination of shade tabs, remove it using a soft brush and running water. • Place shade tabs in the washer-disinfectant allowing water and detergent to enter and drain out of device openings. • Run washer-disinfection program with A0 value ≥ 3000 (e.g. 5 min at $\geq 90^\circ\text{C}$ (194°F) e.g. Miele Vario TD) using appropriate detergents (e.g. neodisher® MediClean and neodisher® Z, both from Dr. Weigert, Germany or equivalent). • After the cycle is complete, visually inspect the shade tabs for degradation due to extended use and reprocessing. If the shade tabs are discolored, cracked, worn, distorted, etc., discard and do not use. • Proceed to sterilization following automated cleaning and disinfection.
Drying	<p>The drying instructions are incorporated in the cleaning and/or disinfection section above.</p>
Maintenance, Inspection and Testing	<ul style="list-style-type: none"> • Visually inspect the shade tabs and holder for degradation due to extended use and reprocessing. If the device is discolored, cracked, worn, distorted, etc., discard and do not use.
Packaging	<ul style="list-style-type: none"> • Ensure that the shade tab(s) are completely dry before packaging for sterilization. • A lint-free, disposable cloth may be used to dry the shade tabs. • Uses of FDA-cleared, ISO 11607 compliant paper/plastic steam sterilization pouches (e.g., AssurePlus® Sterilization Pouches) are recommended.
Sterilization/ Steam autoclaving	<ul style="list-style-type: none"> • The shade tabs must be sterilized utilizing steam sterilization. • The following pre-vac sterilization cycles may be used for shade tabs: <ul style="list-style-type: none"> – 132°C (270°F) for 4 minutes with a minimum drying time of 20 minutes. – 134°C (273°F) for 3 minutes with a minimum drying time of 18 minutes (recommended only for non-US markets). – 135°C (275°F) for 3 minutes with a minimum drying time of 16 minutes. • The following Gravity Displacement cycles may be used: <ul style="list-style-type: none"> – 132°C (270°F) for 15 minutes with a minimum drying time of 30 minutes. – 135°C (275°F) for 10 minutes with a minimum drying time of 30 minutes. <p>An alternate method of sterilization is to place non-bagged instruments into the steam sterilizer and run at one of the above listed cycles</p>
Storage	<ul style="list-style-type: none"> • Do not remove from the pouch until ready for use to prevent contamination. • Instruments sterilized non-bagged should be used immediately. • Store at room temperature, away from moisture or excessive humidity. • To prevent contamination, store processed device in covered storage such as a drawer or cabinet until use. • Prior to use, inspect the pouch. If the integrity of the pouch has been compromised, the device should be processed again before use.

Additional Information	<ul style="list-style-type: none"> • Prior to use, inspect the device. Discard any device which has become damaged, worn, or distorted. • Reassemble for use (replace individual shade tabs in holder) as desired. <p>The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials, and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.</p>
Manufacturer Contact	<ul style="list-style-type: none"> • Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.

4.5 Disposal

Dispose of in accordance with local regulations.

5 Lot number (), expiration date () and correspondence

1. Do not use after expiration date.
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”
2. The following numbers should be quoted in all correspondence:
 - Reference number (REF)
 - Lot number
 - Expiration date
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
4. Device Identification (Basic UDI-DI): ++D010FIM04R5

A summary of the safety and clinical performance (SSCP) for this product can be found (upon activation) at <https://ec.europa.eu/tools/eudamed> by searching using the Basic UDI-DI number listed above and at <https://www.dentsplysirona.com/ifu> using the reference number (REF).

© Dentsply Sirona 2024-03-12

[These Instructions for Use are based on Master Version 04]

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Dentsply DeTrey GmbH