

G-nial® Universal Flo

UNIVERSAL LIGHT-CURED RADIOOPQUE FLOWABLE

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Direct restorative Class II, III, IV and V cavities.
- Fissure sealant.
- Sealing hypervascular areas.
- Repair of (in) direct aesthetic restorations, temporary crown & bridge, defect margins when margins are not visible.
- Blocking out undercuts.
- Linear or bar.
- Realization of crown & bridges, inlays and veneers using the indirect technique in combination with GC Grada components (please refer to its dedicated I.D.).
- Sealing teeth in combination with fibres such as GC Everstick fibres. Follow manufacturer's instructions.
- Adhesive luting of ceramic and composite inlays, onlays and veneers with a thickness (≥ 2.0 mm) that enables the use of a purely light-curing technique.

COMPOSITION

Storium glass, dimethacrylate, silicon dioxide, initiator, pigments, stabilizers.

CONTRAINDICATIONS

Avoid use of this product in patients with known methacrylate monomer and methacrylate polymer allergies.

PRODUCT DESCRIPTION

G-nial Universal Flo is a light-cure, radio-opaque, flowable restorative material to be used intra-oral and classified as a Type I and Class 2 Group 1 (ISO standard 4049). This material has a radiopacity equivalent to 1.5 - 2.0 mm of aluminium (density = 1 mm, length = 8 mm). The particle size of monomer, fillers ranges is 0.1 - 10 µm.

The total amount of organic tin is approximately 50 ppm.

Operation

- Preparations
 - Hold the syringe upright and remove the cap by pulling the plunger upwards.
 - Promptly and securely attach the dispensing tip to the syringe by turning clockwise.
 - Place the light-protective cap over unit use.
 - Note: This material must not be attached the dispensing tip too tightly. This may damage its syringe, in order to ensure a tight connection make sure that the threads are free of any contaminant material.
- Shade Selection
 - Classify the tooth with pumice and water. Shade selection should be made prior to any preparation.
 - Remove the cap and select a shade of A1, A2, A3, A3.5, A4, B1, B2, B3, C1, A02, A03, BW, C.V, JE, A.E, A, B, C shades are based on the VITA® classical shade guide.
 - Note: The shade of G-nial Universal Flo is Z10, Z10.5, Z11, Z12, Z13, Z14, Z15, Z16, Z17, Z18, Z19, Z20, Z21, Z22, Z23, Z24, Z25, Z26, Z27, Z28, Z29, Z30, Z31, Z32, Z33, Z34, Z35, Z36, Z37, Z38, Z39, Z40, Z41, Z42, Z43, Z44, Z45, Z46, Z47, Z48, Z49, Z50, Z51, Z52, Z53, Z54, Z55, Z56, Z57, Z58, Z59, Z60, Z61, Z62, Z63, Z64, Z65, Z66, Z67, Z68, Z69, Z70, Z71, Z72, Z73, Z74, Z75, Z76, Z77, Z78, Z79, Z80, Z81, Z82, Z83, Z84, Z85, Z86, Z87, Z88, Z89, Z90, Z91, Z92, Z93, Z94, Z95, Z96, Z97, Z98, Z99, Z100, Z101, Z102, Z103, Z104, Z105, Z106, Z107, Z108, Z109, Z110, Z111, Z112, Z113, Z114, Z115, Z116, Z117, Z118, Z119, Z120, Z121, Z122, Z123, Z124, Z125, Z126, Z127, Z128, Z129, Z130, Z131, Z132, Z133, Z134, Z135, Z136, Z137, Z138, Z139, Z140, Z141, Z142, Z143, Z144, Z145, Z146, Z147, Z148, Z149, Z150, Z151, Z152, Z153, Z154, Z155, Z156, Z157, Z158, Z159, Z160, Z161, Z162, Z163, Z164, Z165, Z166, Z167, Z168, Z169, Z170, Z171, Z172, Z173, Z174, Z175, Z176, Z177, Z178, Z179, Z180, Z181, Z182, Z183, Z184, Z185, Z186, Z187, Z188, Z189, Z190, Z191, Z192, Z193, Z194, Z195, Z196, Z197, Z198, Z199, Z200, Z201, Z202, Z203, Z204, Z205, Z206, Z207, Z208, Z209, Z210, Z211, Z212, Z213, Z214, Z215, Z216, Z217, Z218, Z219, Z220, Z221, Z222, Z223, Z224, Z225, Z226, Z227, Z228, Z229, Z230, Z231, Z232, Z233, Z234, Z235, Z236, Z237, Z238, Z239, Z240, Z241, Z242, Z243, Z244, Z245, Z246, Z247, Z248, Z249, Z250, Z251, Z252, Z253, Z254, Z255, Z256, Z257, Z258, Z259, Z260, Z261, Z262, Z263, Z264, Z265, Z266, Z267, Z268, Z269, Z270, Z271, Z272, Z273, Z274, Z275, Z276, Z277, Z278, Z279, Z280, Z281, Z282, Z283, Z284, Z285, Z286, Z287, Z288, Z289, Z290, Z291, Z292, Z293, Z294, Z295, Z296, Z297, Z298, Z299, Z300, Z301, Z302, Z303, Z304, Z305, Z306, Z307, Z308, Z309, Z310, Z311, Z312, Z313, Z314, Z315, Z316, Z317, Z318, Z319, Z320, Z321, Z322, Z323, Z324, Z325, Z326, Z327, Z328, Z329, Z330, Z331, Z332, Z333, Z334, Z335, Z336, Z337, Z338, Z339, Z340, Z341, Z342, Z343, Z344, Z345, Z346, Z347, Z348, Z349, Z350, Z351, Z352, Z353, Z354, Z355, Z356, Z357, Z358, Z359, Z360, Z361, Z362, Z363, Z364, Z365, Z366, Z367, Z368, Z369, Z370, Z371, Z372, Z373, Z374, Z375, Z376, Z377, Z378, Z379, Z380, Z381, Z382, Z383, Z384, Z385, Z386, Z387, Z388, Z389, Z390, Z391, Z392, Z393, Z394, Z395, Z396, Z397, Z398, Z399, Z400, Z401, Z402, Z403, Z404, Z405, Z406, Z407, Z408, Z409, Z410, Z411, Z412, Z413, Z414, Z415, Z416, Z417, Z418, Z419, Z420, Z421, Z422, Z423, Z424, Z425, Z426, Z427, Z428, Z429, Z430, Z431, Z432, Z433, Z434, Z435, Z436, Z437, Z438, Z439, Z440, Z441, Z442, Z443, Z444, Z445, Z446, Z447, Z448, Z449, Z450, Z451, Z452, Z453, Z454, Z455, Z456, Z457, Z458, Z459, Z460, Z461, Z462, Z463, Z464, Z465, Z466, Z467, Z468, Z469, Z470, Z471, Z472, Z473, Z474, Z475, Z476, Z477, Z478, Z479, Z480, Z481, Z482, Z483, Z484, Z485, Z486, Z487, Z488, Z489, Z490, Z491, Z492, Z493, Z494, Z495, Z496, Z497, Z498, Z499, Z500, Z501, Z502, Z503, Z504, Z505, Z506, Z507, Z508, Z509, Z510, Z511, Z512, Z513, Z514, Z515, Z516, Z517, Z518, Z519, Z520, Z521, Z522, Z523, Z524, Z525, Z526, Z527, Z528, Z529, Z530, Z531, Z532, Z533, Z534, Z535, Z536, Z537, Z538, Z539, Z540, Z541, Z542, Z543, Z544, Z545, Z546, Z547, Z548, Z549, Z550, Z551, Z552, Z553, Z554, Z555, Z556, Z557, Z558, Z559, Z560, Z561, Z562, Z563, Z564, Z565, Z566, Z567, Z568, Z569, Z570, Z571, Z572, Z573, Z574, Z575, Z576, Z577, Z578, Z579, Z580, Z581, Z582, Z583, Z584, Z585, Z586, Z587, Z588, Z589, Z590, Z591, Z592, Z593, Z594, Z595, Z596, Z597, Z598, Z599, Z600, Z601, Z602, Z603, Z604, Z605, Z606, Z607, Z608, Z609, Z610, Z611, Z612, Z613, Z614, Z615, Z616, Z617, Z618, Z619, Z620, Z621, Z622, Z623, Z624, Z625, Z626, Z627, Z628, Z629, Z630, Z631, Z632, Z633, Z634, Z635, Z636, Z637, Z638, Z639, Z640, Z641, Z642, Z643, Z644, Z645, Z646, Z647, Z648, Z649, Z650, Z651, Z652, Z653, Z654, Z655, Z656, Z657, Z658, Z659, Z660, Z661, Z662, Z663, Z664, Z665, Z666, Z667, Z668, Z669, Z670, Z671, Z672, Z673, Z674, Z675, Z676, Z677, Z678, Z679, Z680, Z681, Z682, Z683, Z684, Z685, Z686, Z687, Z688, Z689, Z690, Z691, Z692, Z693, Z694, Z695, Z696, Z697, Z698, Z699, Z700, Z701, Z702, Z703, Z704, Z705, Z706, Z707, Z708, Z709, Z710, Z711, Z712, Z713, Z714, Z715, Z716, Z717, Z718, Z719, Z720, Z721, Z722, Z723, Z724, Z725, Z726, Z727, Z728, Z729, Z730, Z731, Z732, Z733, Z734, Z735, Z736, Z737, Z738, Z739, Z740, Z741, Z742, Z743, Z744, Z745, Z746, Z747, Z748, Z749, Z750, Z751, Z752, Z753, Z754, Z755, Z756, Z757, Z758, Z759, Z760, Z761, Z762, Z763, Z764, Z765, Z766, Z767, Z768, Z769, Z770, Z771, Z772, Z773, Z774, Z775, Z776, Z777, Z778, Z779, Z780, Z781, Z782, Z783, Z784, Z785, Z786, Z787, Z788, Z789, Z790, Z791, Z792, Z793, Z794, Z795, Z796, Z797, Z798, Z799, Z800, Z801, Z802, Z803, Z804, Z805, Z806, Z807, Z808, Z809, Z810, Z811, Z812, Z813, Z814, Z815, Z816, Z817, Z818, Z819, Z820, Z821, Z822, Z823, Z824, Z825, Z826, Z827, Z828, Z829, Z830, Z831, Z832, Z833, Z834, Z835, Z836, Z837, Z838, Z839, Z840, Z841, Z842, Z843, Z844, Z845, Z846, Z847, Z848, Z849, Z850, Z851, Z852, Z853, Z854, Z855, Z856, Z857, Z858, Z859, Z860, Z861, Z862, Z863, Z864, Z865, Z866, Z867, Z868, Z869, Z870, Z871, Z872, Z873, Z874, Z875, Z876, Z877, Z878, Z879, Z880, Z881, Z882, Z883, Z884, Z885, Z886, Z887, Z888, Z889, Z890, Z891, Z892, Z893, Z894, Z895, Z896, Z897, Z898, Z899, Z900, Z901, Z902, Z903, Z904, Z905, Z906, Z907, Z908, Z909, Z910, Z911, Z912, Z913, Z914, Z915, Z916, Z917, Z918, Z919, Z920, Z921, Z922, Z923, Z924, Z925, Z926, Z927, Z928, Z929, Z930, Z931, Z932, Z933, Z934, Z935, Z936, Z937, Z938, Z939, Z940, Z941, Z942, Z943, Z944, Z945, Z946, Z947, Z948, Z949, Z950, Z951, Z952, Z953, Z954, Z955, Z956, Z957, Z958, Z959, Z960, Z961, Z962, Z963, Z964, Z965, Z966, Z967, Z968, Z969, Z970, Z971, Z972, Z973, Z974, Z975, Z976, Z977, Z978, Z979, Z980, Z981, Z982, Z983, Z984, Z985, Z986, Z987, Z988, Z989, Z990, Z991, Z992, Z993, Z994, Z995, Z996, Z997, Z998, Z999, Z1000, Z1001, Z1002, Z1003, Z1004, Z1005, Z1006, Z1007, Z1008, Z1009, Z1010, Z1011, Z1012, Z1013, Z1014, Z1015, Z1016, Z1017, Z1018, Z1019, Z1020, Z1021, Z1022, Z1023, Z1024, Z1025, Z1026, Z1027, Z1028, Z1029, Z1030, Z1031, Z1032, Z1033, Z1034, Z1035, Z1036, Z1037, Z1038, Z1039, Z1040, Z1041, Z1042, Z1043, Z1044, Z1045, Z1046, Z1047, Z1048, Z1049, Z1050, Z1051, Z1052, Z1053, Z1054, Z1055, Z1056, Z1057, Z1058, Z1059, Z1060, Z1061, Z1062, Z1063, Z1064, Z1065, Z1066, Z1067, Z1068, Z1069, Z1070, Z1071, Z1072, Z1073, Z1074, Z1075, Z1076, Z1077, Z1078, Z1079, Z1080, Z1081, Z1082, Z1083, Z1084, Z1085, Z1086, Z1087, Z1088, Z1089, Z1090, Z1091, Z1092, Z1093, Z1094, Z1095, Z1096, Z1097, Z1098, Z1099, Z1100, Z1101, Z1102, Z1103, Z1104, Z1105, Z1106, Z1107, Z1108, Z1109, Z1110, Z1111, Z1112, Z1113, Z1114, Z1115, Z1116, Z1117, Z1118, Z1119, Z1120, Z1121, Z1122, Z1123, Z1124, Z1125, Z1126, Z1127, Z1128, Z1129, Z1130, Z1131, Z1132, Z1133, Z1134, Z1135, Z1136, Z1137, Z1138, Z1139, Z1140, Z1141, Z1142, Z1143, Z1144, Z1145, Z1146, Z1147, Z1148, Z1149, Z1150, Z1151, Z1152, Z1153, Z1154, Z1155, Z1156, Z1157, Z1158, Z1159, Z1160, Z1161, Z1162, Z1163, Z1164, Z1165, Z1166, Z1167, Z1168, Z1169, Z1170, Z1171, Z1172, Z1173, Z1174, Z1175, Z1176, Z1177, Z1178, Z1179, Z1180, Z1181, Z1182, Z1183, Z1184, Z1185, Z1186, Z1187, Z1188, Z1189, Z1190, Z1191, Z1192, Z1193, Z1194, Z1195, Z1196, Z1197, Z1198, Z1199, Z1200, Z1201, Z1202, Z1203, Z1204, Z1205, Z1206, Z1207, Z1208, Z1209, Z1210, Z1211, Z1212, Z1213, Z1214, Z1215, Z1216, Z1217, Z1218, Z1219, Z1220, Z1221, Z1222, Z1223, Z1224, Z1225, Z1226, Z1227, Z1228, Z1229, Z1230, Z1231, Z1232, Z1233, Z1234, Z1235, Z1236, Z1237, Z1238, Z1239, Z1240, Z1241, Z1242, Z1243, Z1244, Z1245, Z1246, Z1247, Z1248, Z1249, Z1250, Z1251, Z1252, Z1253, Z1254, Z1255, Z1256, Z1257, Z1258, Z1259, Z1260, Z1261, Z1262, Z1263, Z1264, Z1265, Z1266, Z1267, Z1268, Z1269, Z1270, Z1271, Z1272, Z1273, Z1274, Z1275, Z1276, Z1277, Z1278, Z1279, Z1280, Z1281, Z1282, Z1283, Z1284, Z1285, Z1286, Z1287, Z1288, Z1289, Z1290, Z1291, Z1292, Z1293, Z1294, Z1295, Z1296, Z1297, Z1298, Z1299, Z1300, Z1301, Z1302, Z1303, Z1304, Z1305, Z1306, Z1307, Z1308, Z1309, Z1310, Z1311, Z1312, Z1313, Z1314, Z1315, Z1316, Z1317, Z1318, Z1319, Z1320, Z1321, Z1322, Z1323, Z1324, Z1325, Z1326, Z1327, Z1328, Z1329, Z1330, Z1331, Z1332, Z1333, Z1334, Z1335, Z1336, Z1337, Z1338, Z1339, Z1340, Z1341, Z1342, Z1343, Z1344, Z1345, Z1346, Z1347, Z1348, Z1349, Z1350, Z1351, Z1352, Z1353, Z1354, Z1355, Z1356, Z1357, Z1358, Z1359, Z1360, Z1361, Z1362, Z1363, Z1364, Z1365, Z1366, Z1367, Z1368, Z1369, Z1370, Z1371, Z1372, Z1373, Z1374, Z1375, Z1376, Z1377, Z1378, Z1379, Z1380, Z1381, Z1382, Z1383, Z1384, Z1385, Z1386, Z1387, Z1388, Z1389, Z1390, Z1391, Z1392, Z1393, Z1394, Z1395, Z1396, Z1397, Z1398, Z1399, Z1400, Z1401, Z1402, Z1403, Z1404, Z1405, Z1406, Z1407, Z1408, Z1409, Z1410, Z1411, Z1412, Z1413, Z1414, Z1415, Z1416, Z1417, Z1418, Z1419, Z1420, Z1421, Z1422, Z1423, Z1424, Z1425, Z1426, Z1427, Z1428, Z1429, Z1430, Z1431, Z1432, Z1433, Z1434, Z1435, Z1436, Z1437, Z1438, Z1439, Z1440, Z1441, Z1442, Z1443, Z1444, Z1445, Z1446, Z1447, Z1448, Z1449, Z1450, Z1451, Z1452, Z1453, Z1454, Z1455, Z1456, Z1457, Z1458, Z1459, Z1460, Z1461, Z1462, Z1463, Z1464, Z1465, Z1466, Z1467, Z1468, Z1469, Z1470, Z1471, Z1472, Z1473, Z1474, Z1475, Z1476, Z1477, Z1478, Z1479, Z1480, Z1481, Z1482, Z1483, Z1484, Z1485, Z1486, Z1487, Z1488, Z1489, Z1490, Z1491, Z1492, Z1493, Z1494, Z1495, Z1496, Z1497, Z1498, Z1499, Z1500, Z1501, Z1502, Z1503, Z1504, Z1505, Z1506, Z1507, Z1508, Z1509, Z1510, Z1511, Z1512, Z1513, Z1514, Z1515, Z1516, Z1517, Z1518, Z1519, Z1520, Z1521, Z1522, Z1523, Z1524, Z1525, Z1526, Z1527, Z1528, Z1529, Z1530, Z1531, Z1532, Z1533, Z1534, Z1535, Z1536, Z1537, Z1538, Z1539, Z1540, Z1541, Z1542, Z1543, Z1544, Z1545, Z1546, Z1547, Z1548, Z1549, Z1550, Z1551, Z1552, Z1553, Z1554, Z1555, Z1556, Z1557, Z1558, Z1559, Z1560, Z1561, Z1562, Z1563, Z1564, Z1565, Z1566, Z1567, Z1568, Z1569, Z1570, Z1571, Z1572, Z1573, Z1574, Z1575, Z1576, Z1577, Z1578, Z1579, Z1580, Z1581, Z1582, Z1583, Z1584, Z1585, Z1586, Z1587, Z1588, Z1589, Z1590, Z1591, Z1592, Z1593, Z1594, Z1595, Z1596, Z1597, Z1598, Z1599, Z1600, Z1601, Z1602, Z1603, Z1604, Z1605, Z1606, Z1607, Z1608, Z1609, Z1610, Z1611, Z1612, Z1613, Z1614, Z1615, Z1616, Z1617, Z1618, Z1619, Z1620, Z1621, Z1622, Z1623, Z1624, Z1625, Z1626, Z1627, Z1628, Z1629, Z1630, Z1631, Z1632, Z1633, Z1634, Z1635, Z1636, Z1637, Z1638, Z1639, Z1640, Z1641, Z1642, Z1643, Z1644, Z1645, Z1646, Z1647, Z1648, Z1649, Z1650, Z1651, Z1652, Z1653, Z1654, Z1655, Z1656, Z1657, Z1658, Z1659, Z1660, Z1661, Z1662, Z1663, Z1664, Z1665, Z1666, Z1667, Z1668, Z1669, Z1670, Z1671, Z1672, Z1673, Z1674, Z1675, Z1676, Z1677, Z1678, Z1679, Z1680, Z1681, Z1682, Z1683, Z1684, Z1685, Z1686, Z1687, Z1688, Z1689, Z1690, Z1691, Z1692, Z1693, Z1694, Z1695, Z1696, Z1697, Z1698, Z1699, Z1700, Z1701, Z1702, Z1703, Z1704, Z1705, Z1706, Z1707, Z1708, Z1709, Z1710, Z1711, Z1712, Z1713, Z1714, Z1715, Z1716, Z1717, Z1718, Z1719, Z1720, Z1721, Z1722, Z1723, Z1724, Z1725, Z1726, Z1727, Z1728, Z1729, Z1730, Z1731, Z1732, Z1733, Z1734, Z1735, Z1736, Z1737, Z1738, Z1739, Z1740, Z1741, Z1742, Z1743, Z1744, Z1745, Z1746, Z1747, Z1748, Z1749, Z1750, Z1751, Z1752, Z1753, Z1754, Z1755, Z1756, Z1757, Z1758, Z1759, Z1760, Z1761, Z1762, Z1763, Z1764, Z1765, Z1766, Z1767, Z1768, Z1769, Z1770, Z1771, Z1772, Z1773, Z1774, Z1775, Z1776, Z1777, Z1778, Z1779, Z1780, Z1781, Z1782, Z1783, Z1784, Z1785, Z1786, Z1787, Z1788, Z1789, Z1790, Z1791, Z1792, Z1793, Z1794, Z1795, Z1796, Z1797, Z1798, Z1799, Z1800, Z1801, Z1802, Z1803, Z1804, Z1805, Z1806, Z1807, Z1808, Z1809, Z1810, Z1811, Z1812, Z1813, Z1814, Z1815, Z1816, Z1817, Z1818, Z1819, Z1820, Z1821, Z1822, Z1823, Z1824, Z1825, Z1826, Z1827, Z1828, Z1829, Z1830, Z1831, Z1832, Z1833, Z1834, Z1835, Z1836, Z1837, Z1838, Z1839, Z1840, Z1841, Z1842, Z1843, Z1844, Z1845, Z1846, Z1847, Z1848, Z1849, Z1850, Z1851, Z1852, Z1853, Z1854, Z1855, Z1856, Z1857, Z1858, Z1859, Z1860, Z1861, Z1862, Z1863, Z1864, Z1865, Z1866, Z1867, Z1868, Z1869, Z1870, Z1871, Z1872, Z1873, Z1874, Z1875, Z1876, Z1877, Z1878, Z1879, Z1880, Z1881, Z1882, Z1883, Z1884, Z1885, Z1886, Z1887, Z1888, Z1889, Z1890, Z1891, Z1892, Z1893, Z1894, Z1895, Z1896, Z1897, Z1898, Z1899, Z1900, Z1901, Z1902, Z1903, Z1904, Z1905, Z1906, Z1907, Z1908, Z1909, Z1910, Z1911, Z1912, Z1913, Z1914, Z1915, Z1916, Z1917, Z1918, Z1919, Z1920, Z1921, Z1922, Z1923, Z1924, Z1925, Z1926, Z1927, Z1928, Z1929, Z1930, Z1931, Z1932, Z1933, Z1934, Z1935, Z1936, Z1937, Z1938, Z1939, Z1940, Z1941, Z1942, Z1943, Z1944, Z1945, Z1946, Z1947, Z1948, Z1949, Z1950, Z1951, Z1952, Z1953, Z1954, Z1955, Z1956, Z1957, Z1958, Z1959, Z1960, Z1961, Z1962, Z1963, Z1964, Z1965, Z1966, Z1967, Z1968, Z1969, Z1970, Z1971, Z1972, Z1973, Z1974, Z1975, Z1976, Z1977, Z1978, Z1979, Z1980, Z1981, Z1982, Z1983, Z1984, Z1985, Z1986, Z1987, Z1988, Z1989, Z1990, Z1991, Z1992, Z1993, Z1994, Z1995, Z1996, Z1997, Z1998, Z1999, Z2000, Z2001, Z2002, Z2003, Z2004, Z2005, Z2006, Z2007, Z2008, Z2009, Z2010, Z2011, Z2012, Z2013, Z2014, Z2015, Z2016, Z2017, Z2018, Z2019, Z2020, Z2021, Z2022, Z2023, Z2024, Z2025, Z2026, Z2027, Z2028, Z2029, Z2030, Z2031, Z2032, Z2033, Z2034, Z2035, Z2036, Z2037, Z2038, Z2039, Z2040, Z2041, Z2042, Z2043, Z2044, Z2045, Z2046, Z2047, Z2048, Z2049, Z2050

нормально.

Минимизируйте воздействие общего освещения на материалы, находясь в помещении или при выносе рабочего оборудования.

Сразу после использования материала снимите остатки со шпателя и помойте его в латинском растворе выводящей кислоты.

Клинический Совет

Использовать полимеризационное наложение материала для лечения временной реставрации не является эффективным или эффективным. После того, как нужный объем материала был нанесен на зубы, прикройте их пленкой на пленку, а отверстие шпательной области реставрации в направлении, перпендикулярном плоскости окклюзии. При использовании шпательной области шпатель и равномерно распределите, создавая ровную поверхность реставрации.

Специотребования

Производитель систем полимеризации G-enial Universal Pro для временной полимеризационной реставрации. Держите световую лампу как можно ближе к поверхности реставрации. Окклюзия со стороны окклюзии не должна превышать 10 минут. Прочтите инструкции по применению, включая обучение и эффективную гигиену отекновения.

450–480 нм.

Примечания:

1. Материал следует наносить и полимеризовать слоями. Максимально допустимая толщина слоя указана в таблице выше.

2. Низкая интенсивность светового излучения может стать причиной недостаточного отверждения

Skrú alebo začnite nanášať materiál zo striekačky do kvapky:

1. Dávkováci hrot musí byť bezpečne pripojený k striekačke.

2. Z dávkovacieho hrotu vypustíte všetok vzduch tak, že striekačku otočíte hrotom