General Information

3M™ ESPE™ RelyX™ Luting cement is a glass ionomer system comprised of two parts: a powder and a liquid. The powder is a radiopaque, fluoroaluminosilicate glass. The liquid is an aqueous solution of a modified polyalkenoic acid. RelyX Luting Cement provides the major benefits of glass ionomer cements - adhesion to tooth structure, fluoride release along with additional benefits of low solubility, improved fracture toughness and a low viscosity, slump-resistant mix. RelyX Luting Cement is a self-cure system using two dark-cure reactions to provide the improved properties.

RelyX Luting cement is radiopaque and is available in a white shade. The powder jar contains 16gm of material and the liquid bottle contains 9ml (11gm) of material.

Indications

RelyX Luting Cement is indicated for use when luting:

- Porcelain fused to metal crowns and bridges to tooth structure, amalgam composite or glass ionomer core buildsups
- Metal inlays, onlays or crowns
- Crowns made with all-alumina or all-zirconia cores such as 3M™ ESPE™ Lava™ or Procera® AllCeram
- Prefabricated and cast post cementation
- Orthodontic bands and appliances
- Porcelain fused to metal (PFM), metal, all-alumina or all-zirconia core restorations on implant abutments

Precautionary Information for Patients:

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

Contains potassium persulfate that may trigger an allergic respiratory reaction in certain individuals. This product may not be appropriate for use in those individuals with known sensitivity to sulfites, since a cross-reaction may occur with potassium sulfate.

Precautionary Information for Dental Personnel:

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M ESPE MSDS information can be obtained from www.3MESPE.com or contact your local subsidiary.

Instructions for Use

Preparation

- Remove the provisional restoration and all residual temporary cement.
- Thoroughly clean the preparation with an oil-free pumice paste. Rinse with water and let dry. **Note:** Do not desiccate the tooth. Desiccation of tooth structure is believed to cause post operative sensitivity in some individuals.
- Pulp protection: Use a hard setting calcium hydroxide material with near exposures. **Note:** Use of some tooth preparation treatments is not recommended as their residue may adversely affect bond strength or the setting reactions (e.g., Tubulicid Red). Contact your local 3M ESPE representative or the 3M ESPE Customer Care Center for specific details.
- Casting preparation: Thoroughly clean interior surfaces of cast crown, inlay or onlay.

Directions

1. Dispense powder and liquid: The standard powder/liquid ratio of 1.6:1 by weight can be obtained by using an equal number of level scoops of powder and liquid drops. Three scoops of powder and three drops of liquid will provide an adequate amount of material to seat one typical crown.

- 1.1. Shake the jar to fluff the RelyX Luting powder before dispensing.
- 1.2. Insert the scoop into the jar, overfill it with loosely packed powder and withdraw it against the plastic leveler to remove excess powder and obtain a level scoop.

1.3. Dispense the desired number of powder scoops onto the mixing pad. **Note:** The glass ionomer powders are sensitive to high humidity. Store with jar caps securely tightened and away from high humidity. Replace cap immediately after use.

1.4. To obtain a proper liquid drop size, hold the RelyX Luting Cement liquid vial vertically with the dropper tip down and without

Zbrinjavanje – Ako trebate informacije o načinu zbrinjavanja, pogledajte sigurnosno-tehnički list materijala (dostupan je na adresi www.3MESPE.com ili putem lokalne podružnice).

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