VOCO Calcimol

CE 0482

EN Instructions for use MD EU Medical device

Product description:

Calcimol is a self-curing radiopaque calcium hydroxide paste.

Indications:

Calcimol is intended for indirect pulp capping and as a liner under dental filling materials.

Contraindications:

Calcimol contains salicylate. **Calcimol** should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:

Calcimol is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Calcimol should only be used by a professionally trained dental practitioner.

Use:

Mix equal quantities of **Calcimol** base and catalyst (mixing ratio 1.2 : 1 g/g) on a mixing pad or glass plate for approx. 10 s. The mixture should achieve a homogeneous consistency and uniform colour. Do not mix for too long!
Once mixed, insert **Calcimol** into the cavity as quickly as possible. Use **Calcimol** at room temperature (15 °C - 23 °C).
At room temperature **Calcimol** remains workable for approx. 2 min and cures in the mouth within 2 - 4 min. Higher temperatures and humidity accelerate the setting time and reduce the working time, causing **Calcimol** to set more quickly in the patient's mouth.

Warnings, precautionary measures:

- Let totally cure before filling in order to reach a sufficient stability for filling.
- Do not use **Calcimol** as temporary filling or as sole masticatory filling. The catalyst hardens at low temperatures. Allow to reach room temperature before application, if necessary by shortly heating to 40 °C.
- Calcium hydroxide has a caustic effect on the skin and mucosal tissue, it must not be allowed to come into contact with the eyes! In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Salicylate, calcium hydroxide, tricalcium phosphate, N-ethyltoluol-sulfonamide, calcium wolframate, zinc oxide, fumed silica, titanium dioxide, 1,3-butanediol

Storage instructions and application method:

Storage at 15°C-23°C. Reclose the tubes carefully after use. Be certain to match the colour-coded caps to the proper tube. Storage protected from moisture. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Calcimol** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Calcimol** can be found in the European database on medical devices (EUDAMED – [https:// ec.europa.eu/tools/eudamed](https://ec.europa.eu/tools/eudamed)). Detailed information can also be found at www.voco.dental.

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