Instructions for Use

Life Regular Set/Fast Set

Life™ Regular Set / Fast Set

Hard-setting calcium hydroxide for pulp capping and cavity lining.

GENERAL INFORMATION

Kerr Life is recommended for direct or indirect pulp capping and as a cement base under all restorative filling materials, including amalgams. Life is easy to mix and provides ample working time to permit multiple placements yet sets fast and hard in the preparation and is carvable. Kerr Life Regular Set and Fast Set are available in traditional tubes.

Key advantages of Kerr Life include:

- Permitting the formation of secondary dentin.
- Excellent flow characteristics which allows controlled placement.
- Maintaining pulp vitality in case of exposure or near exposure.

PRODUCT COMPOSITION

Life contains calcium hydroxide, resin, and barium sulfate as a contrast agent.

INDICATIONS FOR USE

Life is a hard setting calcium hydroxide cavity liner and pulp capping agent to be used in conjunction with all permanent restorative techniques.

CONTRAINDICATIONS

None known.

WARNINGS

- Avoid prolonged skin, eye, and soft tissue exposure to this product since it may cause an irritation reaction. Wash thoroughly with water after contact. Get medical attention for eyes. Do not take internally.

PRECAUTIONS

- This product is intended to be used by qualified dental professionals on a general patient population.
- Wear personal protective equipment when using and handling this product.
- Use a rubber dam to isolate the treated tooth and to avoid fluid contamination during material placement.
- Mixing pads and Kerr Ballpoint Applicators are intended for single patient use only to prevent cross-contamination between patients.
- Do not place Life material on enamel or cavity margins and do not leave material exposed to the oral environment.
- Place material as 1 thin layer – do not use in multiple layers or a bulk portion of the material.
- The chemical nature of some composite restorative systems may cause Life to impart a temporary pink hue. This pink hue, if formed, begins dissipating within 2 hours and does not affect the physical properties or biocompatibility of the composite.
- The material should be used at 23°/-2°C temperature and 40%-60% relative humidity. Accelerated working and setting time results from

higher temperature and higher humidity. Longer working and setting time results from lower temperature and lower humidity.

- When using a rubber dam, setting time may be slowed due to decreased humidity in the oral cavity. It is possible to accelerate the reaction by touching the material with a water-moistened pellet.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS

1. Isolate a treated tooth with a rubber dam to avoid fluid contamination.
2. Dispense the product immediately before use.
3. Remove the caps. Extrude equal lengths of the Base and Catalyst pastes onto the mixing pad. Wipe the nozzle of each tube after use. Recap the tubes, taking care not to interchange the caps.
4. Mix the Base and Catalyst with a small spatula for approximately 10 seconds until the mix is completely streak-free.
5. Place Life immediately into the preparation using a clean, dry applicator. Kerr Ball-Point Applicator is recommended. Create a thin layer of the material which covers pulp exposure site. Remove any material excess from margins of the preparation or retention areas.
6. A minimum working time for Life Regular Set is 3 minutes from start of mix. A minimum working time for Life Fast Set is 2 minutes from start of mix (at temperature of 23°/-2°C and a relative humidity of 40% to 60%). Allow Life to set completely (maximum 7 minutes).
7. Proceed with a restorative material of your choice per manufacturer's Instructions for Use.

CLEANING AND DISINFECTION

Always handle the tubes with clean gloves. Do not reuse if it becomes contaminated.

STORAGE AND DISPOSAL

Product should be stored between 2°C to 25°C (36°F to 77°F). Avoid excess heat. If product is stored in a refrigerator allow the product to reach room temperature prior to use.

The expiration date is marked on packaging. Do not use after expiration date. For proper disposal always follow your local and national regulations.

LIMITED WARRANTY - LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

SYMBOL DESCRIPTION

Full explanation of symbols used on Kerr packaging is located at:
<http://www.kerrdental.com/symbols-glossary>

	Caution
	Manufacturer
	European Conformity
	Authorized representative in the European Community
	Consult instructions for use
	Do not re-use
	Use-by date
	Batch code
	Catalogue number
	Medical device
	Temperature storage limitations: low 2 °C / 36 °F to high
	Importer

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Kerr Corporation
Kerr Italia S.r.l.