VOCO

LC Dam

EN Instructions for use

MD EU Medical device

Product description:

LC Dam is a light-curing composite for covering the gingiva and neighbouring teeth during whitening therapy. It provides a barrier against highly concentrated whitening agents and protects the surrounding soft tissue and teeth that are not to be whitened.

Indications:

LC Dam covers and protects the gingiva and neighbouring teeth during whitening therapy carried out in the dental surgery (in-office whitening).

Contraindications:

LC Dam contains methacrylate and BHT. **LC Dam** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

LC Dam is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

LC Dam should only be used by a professionally trained dental practitioner.

Use:

Preparation:

Before starting treatment, allow the material to reach ambient temperature and insert the cheek retractor. Create a relatively dry treatment area in the usual way, e.g. with cotton rolls and air syringe. Remove the protective cap and attach a cannula to the syringe.

Application:

A 4 - 6 mm wide strip of 1.5 to 2 mm thickness will suffice to ensure that the barrier provides effective protection. About 0.5 mm of the cervical dental hard tissue must also be covered to ensure effective protection of the gingiva. **LC Dam** must be applied up to and including the first tooth not being whitened. The interdental spaces must also be sealed with **LC Dam** to protect the interdental papillae from oxidation. Apply **LC Dam** onto the gingiva adjoining the tooth in sections of two to three teeth.

Conventional polymerisation devices of 1000 mW/cm² output are suitable for light-curing the material. Shorten the duration of exposure accordingly for lamps with a higher power density and extend the duration of exposure accordingly for lamps with a lower power density. Briefly pre-cure **LC Dam** with the polymerisation device and subsequently light-cure it fully for 20 to 40 seconds, in a sweeping motion following the arch of the teeth. Thoroughly check with an instrument that the gingival protection has fully cured.

Remove any large excess amounts of **LC Dam** from the tooth enamel with a suitable hand instrument (e.g. probe, scaler or curette) to prevent impairment of the whitening effect of **Perfect Bleach Office+**. Whitening treatment can begin immediately after **LC Dam** has been carefully light-cured.

Once attached, the application cannula is for single use only. Detach it after usage and seal the syringe again with the original cap. Use of a protective barrier is recommended to prevent contamination of the syringe by bodily fluids or contact with contaminated hands or oral tissue.

LC Dam can be easily removed with an instrument (see above) on completion of the whitening therapy and the removal of all residual whitening gel. The material can normally be completely removed in a few large pieces. Remove any remaining material from the interdental areas with a probe or dental floss. Allow patients to rinse their mouths thoroughly with water afterwards.

Warnings, precautionary measures:

- If irritation of an allergic reaction should occur, stop the treatment immediately, remove **LC Dam** and rinse the treatment area with plenty of water. Refer patients to a doctor, if necessary.
- Phenolic substances, especially products containing eugenol and thymol, lead to curing failures and are to be avoided.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store at temperatures of 2 °C to 23 °C (refrigerator or ambient temperature). Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **LC Dam** must be reported to VOCO GmbH and the responsible authority.

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