

UNIVERSAL PRIMER™

Dual-Cured Adhesive



UNIVERSAL PRIMER is a dual-cured adhesive designed to be used without having to cure the adhesive layer under indirect restorations. With chemistry based on BISCO's All-Bond Universal® technology, UNIVERSAL PRIMER can be used with any etching technique (self-etch, selective-etch or total-etch) and is compatible with light-cured, dual-cured and self-cured resin cements, and core materials for direct and indirect bonding procedures.


- Can be used with total-etch, self-etch, or selective-etch techniques
- Clinical evaluation confirms the ease-of-use benefit, with virtually no post-operative sensitivity
- Low film thickness (<5µm)
- No light-curing necessary when used with dual-cured or self-cure restorative material

Shear Bond Strength to Various Substrates (Duo-Link Universal™)*

SUBSTRATE	UNIVERSAL PRIMER (S/C)
Cut Enamel (SE/TE)	22.9 / 31.4
Dentin (SE/TE)	26.0 / 39.2
Lithium Disilicate (silane treated)	35.4
Zirconia	29.4
Stainless Steel	34.6
Titanium	27.3
Gold	20.8
Composite	40.2

* Data on file, BISCO, Inc.
Both Universal Primer & Duo-Link Universal were tested in self-cure mode using the Ultradent jig method.

Shear Bond Strength to Dentin of Manufacturer's Resin Cement System

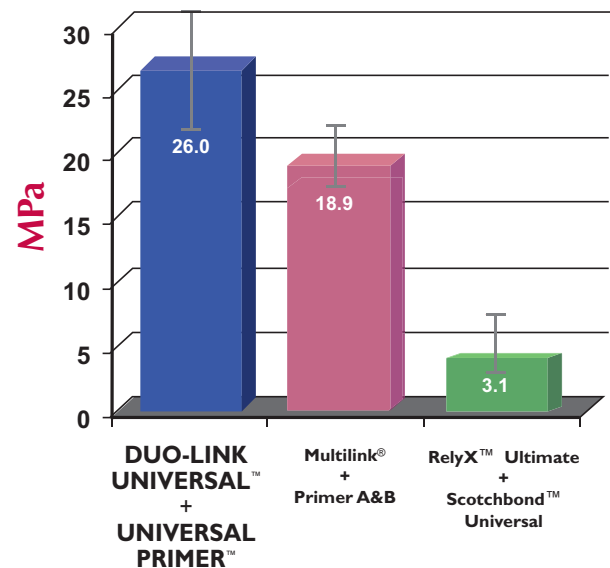

Trademarks are property of the respective manufacturers.

** Both adhesive/primer and cement were tested in self-cure mode using the Ultradent jig method; specimens were aged at 37°C in water for 24 hrs.



ORDER DIRECT FROM BISCO

1-800-247-3368 • 1-847-534-6000

www.bisco.com



Rx Only

MC-3262UP

UNIVERSAL PRIMER™


Dual-Cured Adhesive

INDICATIONS FOR USE:

Primary Indications: Adhesive for cementation of Crowns, Bridges, Inlays/Onlays & Posts
Can also be used as an adhesive to bond Direct Composite Restorations, Desensitization or Sealing of Teeth, if light-cured.

UNIVERSAL PRIMER can be used with any manufacturer's cement.**

Shear Bond Strength Comparison of Manufacturer's Resin Cement System vs. Manufacturer's Cement Paired with BISCO's UNIVERSAL PRIMER

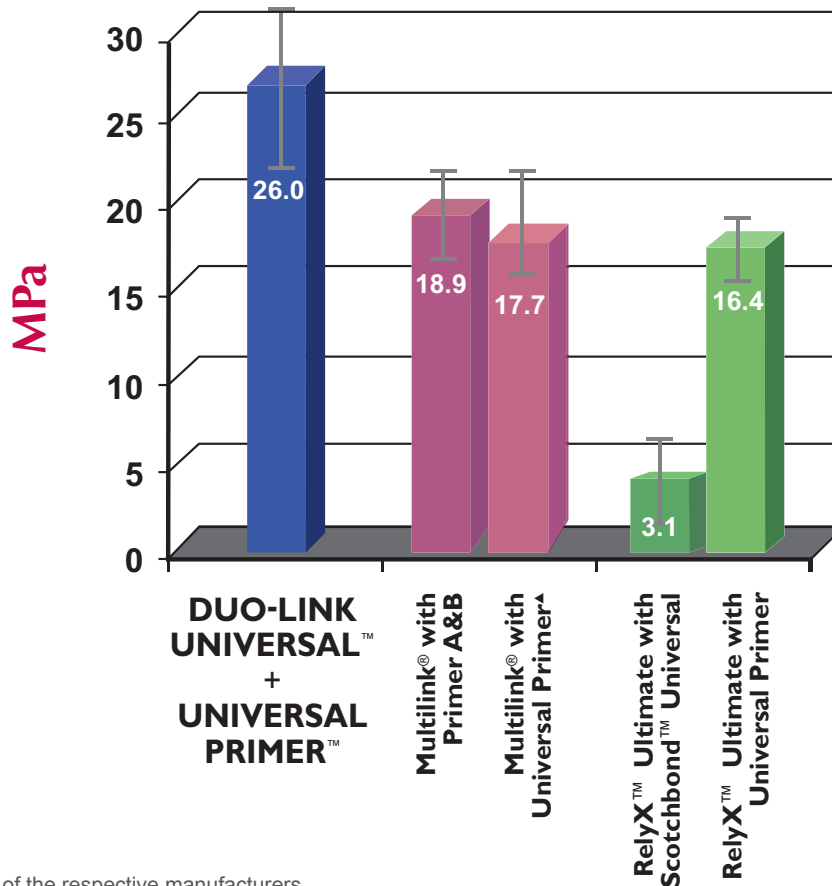

Trademarks are property of the respective manufacturers.

** Both adhesive/primer and cement were tested in self-cure mode using the Ultradent jig method; specimens were aged at 37°C in water for 24 hrs.

▲ Per Instructions for use, Multilink is not recommended for use with other adhesives/primers.

ORDERING INFORMATION

UNIVERSAL PRIMER PART A & B SET **B-4110P**

1 Bottle ea. UNIVERSAL PRIMER PARTS A & B (6ml ea.), Instructions

ACE UNIVERSAL PRIMER STARTER KIT

2 ACE UNIVERSAL Cartridges (2ml ea.), ACE Dispenser, Accessories, Instructions **M-16300K**

ACE UNIVERSAL PRIMER 4 CARTRIDGE PACKAGE

4 ACE UNIVERSAL Cartridges (2ml ea.), Accessories, Instructions **B-41240P**

ACE DISPENSER

1 ACE Dispenser, 1 Mixing Well, Instructions **U-30220P**