fr

COMPOSITION

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en

COMPOSITION

Brown fibrous paste composed of: Penghawar djambi, eugenol, sodium lauryl sulphate, calcium carbonate, mint flavour, excipients.

INTENDED USE

Dry-Socket Surgical dressing

INTENDED POPULATION

Patients with permanent teeth (mature or immature).

INDICATIONS FOR USE

Alveogyl is a paste used as dressing in case of dry socket.

CONTRA INDICATION

Hypersensitivity to one of the components.
Post-extraction of deciduous teeth.

PERFORMANCE CHARACTERISTICS

Technical performance

Alveogyl is for alveolar surgical dressing after tooth extractions, with Penghawar fibers allowing compression.

The paste adheres to the alveolus due to its fibrous consistency and protects it with its barrier effect.

Assisted by the patient's tongue movements, it gradually self eliminates without any intervention by the practitioner as the normal healing of the wound takes place.

CLINICAL PERFORMANCE

Provides a favourable environment for physiological tissue healing.

INSTRUCTIONS

Before inserting the paste, re-examine the socket. The inside of the socket should be thoroughly curedtted to remove all debris.

Take a small pellet (about 0.20 g) of paste and place it gently into the prepared dental socket.

WARNINGS AND PRECAUTIONS FOR USE

Precaution for use

Patients should not wash their mouth vigorously within the 24 hours following the avulsion.

Do not treat more than 4 teeth in one session.

Replace the cap immediately after use.

Confirm the absence of debris before inserting the paste in the alveolus.

Warnings

The device may cause eye irritation and skin burn. Do not handle the device without appropriate personal protective equipment.

Do not suture over the paste.

SIDE EFFECTS

Possible hypersensitivity to one of the components.

DISPOSAL

Harmful to aquatic life with long lasting effects.

STORAGE

Alveogyl is stable until the expiration date indicated on the box label, when stored at 5-30 °C.

PRESENTATION

Jar containing 10 g of paste.

For dental healthcare professional only

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