FR

**COMPOSITION**

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**PROPERTIES**

To act as perfect root canal filling materials, pastes should be neither resorbable once hardened nor retractable over the years. These pastes should be radiopaque and easy to insert in the canal. ENDOMETHASONE N Powder has all the various above-described qualities.

Besides, this medical device has an anti-inflammatory ancillary activity due to the presence of hydrocortisone acetate.

**INDICATIONS**

Permanent root canal filling and sealing in dental procedures with systematic insertion of Gutta-Percha points.

**ADVERSE EFFECTS**

Risk of allergy to any of the constituents particularly in case of contact with gums.

**DIRECTIONS FOR USE**

The treated tooth should be isolated (using a cotton roll or a rubber dam).

After carefully shaping the canal, cleaning it with an antiseptic solution, rinsing it and drying it with paper points, prepare the canal paste extemporaneously, progressively pouring the powder into the liquid (eugenol or ENDOMETHASONE Liquid) at a ratio of 2 spoonfuls of powder to 3-6 drops of liquid, until the paste has the consistency required for the selected filling technique:

- using a paste carrier  
- and/or by coating Gutta-Percha points with a cement sealer for condensation techniques.

Then fill the root canal using a paste-carrier and position into the paste one or several Gutta-Percha points, which have previously been adapted to the canal shape.

Take X-rays to ensure that root canals have been thoroughly filled.

**INTERACTIONS**

Do not use in combination with composite materials, since eugenol-based preparations inhibit their polymerisation.

**SPECIAL WARNINGS AND PRECAUTIONS FOR USE**

As with any other root canal sealer, a canal can only be sealed if it has previously been reamed, cleaned and disinfected thoroughly. Determine the working length of the root canal before filling so as to prevent any apical extrusion, since the paste is not resorbable.

Due to electrostatics, ENDOMETHASONE N, as all powders, tends to agglomerate in the bottle. However, the capacity mentioned on the label corresponds to the weighed contents. It is therefore important to shake the bottle before use in order to loosen the powder.

Danger (contains thymol iodide).

Harmful if swallowed.

Causes severe skin burns and eye damage.

Very toxic to aquatic life with long lasting effects.

Refer to safety data sheets.

**STORAGE**

Do not store above 25°C and protect from moisture.

**PRESENTATIONS**

- Bottle of 14 g or 42 g powder + a spoon.

- 3 bottles of 14 g powder + a spoon.

- Kit: 1 bottle 14 g powder and 1 bottle 10 ml liquid and a spoon.

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