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**EN** Calcium hydroxide paste for temporary root canal filling

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY TO BE USED BY DENTIST ONLY**

**COMPOSITION:** Calcium hydroxide, hydroxyapatite, barium sulphate, excipients.

**INTENDED USE**

CALIPAST is intended for use in dental treatment for temporary root canal filling.

Additionally, CALIPAST may be used for temporary filling of deep cavities during root canal therapy.

It has re-mineralizing effect on dental tissues thanks to its high content of calcium hydroxide. The product supports restoration of damaged periapical tissues.

pH 12.5–13. Radiopaque, non-hardening material.

**INSTRUCTIONS FOR USE**

Before treatment, dip the provided applicator in ethanol, allow to dry and then attach the applicator to the syringe.

Before use, check if the applicator is correctly attached onto the syringe and then expel a small amount from the syringe to check the paste flow through the applicator.

The syringe grip plate provides a comfortable grip for easy dispensing management, reducing the risk of pushing the product through the root apex and minimising the difficulties associated with expelling the product.

Use the syringe grip plate to hold the grip plate onto the syringe before use. The grip plate should be put onto the capped syringe, as shown on the product packaging (a distinct “click” is heard when correctly applied).

Use the syringe grip plate to measure the length of the canal, adjust the endstop on the applicator to a length 2-3 mm shorter than the measured length of the canal, dispense the paste in the canal, remove excess paste, close the cavity tightly.

After the paste has been applied, an X-ray should be taken to check that the paste has adequately filled the canal and has not gone beyond the apex.

When used in a cavity, dispense the paste at the bottom of the cavity, close the cavity tightly.

Leave the filling in the cavity or root canal for 1 week to 30 days.

The product is intended for multiple use, while the applicator is for single use only.

Reusing the applicator may pose a risk of secondary infection.

The grip plate is for multiple use.

**CONTRAINDICATIONS**

Do not use for patients who are hypersensitive to the product components.

**WARNINGS AND PRECAUTIONS**

Irritant to oral mucosa and skin. Causes serious eye damage.

In case of contact, rinse immediately with plenty of water. Seek medical help. If ingested, do not induce vomiting. Drink plenty of water. Seek medical help.

Use dental dam, disposable gloves, safety goggles/visors and protective clothing when using the product.

Endodontic treatment may involve temporary seal and postoperative tenderness.

The paste should be carefully removed from the root canal before the final filling, as restorative materials should be placed directly on the final filling.

Use of temporary materials may be associated with bacterial microleakage. CALIPAST has cytotoxic potential.

Do not expel the paste of the applicator tip is wedged inside the canal. The applicator should be removed by pulling the applicator back.

Particular care should be taken in anatomically sensitive areas such as the mandibular canal, mandibular opening or maxillary sinuses.

Keep out of reach of children.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

**STORAGE**

Keep in original packaging below 25°C.

Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

CALIPAST Syringe containing 2.1 g of preparation, disposable applicators (2 pieces), applicator (4 pieces), retractor.

CALIPAST MEGA PACK 4 syringes of preparation (each containing 2.1g of paste), disposable applicators (4 pieces), grip plate.

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