**RelyX™ Temp E**

en Temporary Cement – Zinc Oxide-Eugenol

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Dental Products

St. Paul, MN 55144-1000 U.S.A.

3M ESPE Customer Care/MSDS Information:

U.S.A. 1-800-634-2249 and Canada 1-888-363-3685.

3M, ESPE and RelyX are trademarks of 3M Company or

3M Deutschland GmbH. Used under license in Canada.

© 2012, 3M. All rights reserved.

## Návod na použitie

Navodila za uporabo

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Kasutusjuhend

Lietošanas Instrukcija

Vartojimo instrukcija

CE

0123

400082509/01

## en ENGLISH

## Product Description

RelyX™ Temp E is a zinc oxide-eugenol luting cement for temporary fixation of dental restorations. The addition of Vaseline can reduce the final hardness of the cement, e.g., to make bridges more easily removable.

► Instructions for Use should not be discarded for the duration of product use.

## Fields of Application

- Temporary fixation of provisional restorations
- Temporary fixation of crowns, bridges, inlays, onlays
- Fixation of crowns and bridges onto implant abutments

## Precautionary Measures

For Patients  
The catalyst paste contains substances that may cause an allergic reaction in case of skin contact in certain individuals. Avoid use of this product in patients with known eugenol allergies.

If prolonged contact with oral tissue occurs, flush with large amounts of water. If an allergic reaction occurs, seek medical attention as needed, remove the product and discontinue future use.

## For Dental Personnel

The catalyst paste contains substances that may cause an allergic reaction in case of skin contact in certain individuals. To reduce the risk of allergic response, avoid exposure to this material, in particular to uncured material. If skin contact occurs, wash skin with soap and water.

The use of protective gloves and a no-touch technique is recommended. If the catalyst paste contacts the gloves, remove and discard the gloves, wash the hands immediately with soap and water and then re-glove.

If an allergic reaction occurs, seek medical attention as needed.

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

## Pulp Protection

The normal pulp protection used for temporary restorations should be applied in the case of deep cavities.

## Dosage and Mixing

Process RelyX Temp E at 18–23°C/64–73°F room temperature and 45–55% relative humidity.

► Dose equal lengths of base paste and catalyst on a mixing pad.

– Overdosing the catalyst will increase the bonding strength and prolong the working time.

– Carefully close the tubes after use. Do not mix up the lids, as contamination with residual paste causes hardening.

► The final hardness of the cement can be reduced by adding Vaseline in order to ensure easier removal, e.g., for final bridges. This simultaneously extends the working time.

Dosage: Maximum of 42 mg Vaseline (the size of a lentil) per 1 cm length of base paste (125 mg) and catalyst (80 mg).

► Mix the pastes thoroughly with a metal or plastic spatula for 30 seconds until the mixture is homogeneous.

## Application

► Before cementing, clean the cavity surfaces with a water spray and dry them. Water and alcohol residue on the prepared tooth structure decreases working time.

► Put a thin cement layer on the surfaces to be cemented and insert the restoration, applying light pressure.

## Times

	min:sek
Mixing:	0:30
Working time after start of mixing (at 23°C/74°F and 50% relative humidity)	2:30
Setting after insertion in the mouth (at 36°C/97°F and 100% relative humidity)	1:30

Higher temperature and humidity reduce working time.

## Removal of Excess Material

► Remove the excess material after the setting time with an instrument.

## Notes

Eugenol residue influences the polymerization of composite filling and luting materials. For this reason, RelyX Temp E cement residue must be thoroughly removed before definitive cementing of restorations.

## After Cementation

► Thoroughly examine the sulcus of the treated teeth and surrounding dentition. Remove any residual cement from the mouth.

## Storage and Shelf Life

Store the product at 15–25°C/59–77°F.

Do not use after expiration date.

## Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

## Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture.

3M Deutschland GmbH MAKES NO OTHER WARRANTIES

INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS

FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

## Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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