NETC Non-Eugenol Type Temporary Cement

2013. 10. 29 DM-54-6

NETC Complies with the requirements of ISO 3107 (Type I, Class 2B):1988(E)

INSTRUCTION FOR USE

APPLICATION

- Cementation of temporary crowns and bridges.

DIRECTIONS FOR USE

- Use **NETC** at ambient temperature of 18°C~23°C(64°F~73°F) and relative humidity of 45~55%
- Dispense two equal length of base and catalyst paste on mixing pad.
- Close the tube immediately after use.
- Use a spatula to thoroughly mix the base and catalyst for 20~30 seconds.
- Insert paste into temporary crown or bridge and seal onto the prepared dry tooth.
- Allow to set (4 minutes after start of mixing) and trim excess.

CHARACTERISTICS

- Easy to mix by paste-paste type.
- Working time of 1.5 minutes after start of mixing.
- Setting time of 4 minutes after start of mixing.
- Especially suitable for use on patients with sensitivity to eugenol.

CAUTIONS

- Do not use the cement in allergic persons.
- For dental use only.
- Keep away from child.

COMPONENTS

- 60g or 40g Base paste
- 20g or 14g Catalyst paste
- Mixing pad

STORAGE

- Store at temperature of 18°C~24°C(64°F~75°F)
- Close the tubes immediately after use.

SYMBOLS

- Use By
- Batch Code
- CE-Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC
- Authorized representative in the EC
- Caution, See instruction for use
- Manufacturer

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META BIOMED CO.LTD.