Scotchbond™ Universal Etchant

- en** Etching Gel
- ru** Протравочный гель
- bg** Еващ гел
- hr** Gel za jetkanje
- hu** Maratogél
- pl** Wytrawiacz w żelu
- ro** Gel demineralizant
- sk** Leptací gél
- si** Gel za jedkanje
- cs** Leptací gel
- tr** Asit jel
- et** Söövitusegel
- lv** Kodigs gēls
- lt** Ėsdinimo gelis
- uk** Протравлюючий гель

Instructions for Use
Инструкция по применению
Указания за употреба
Upute za upotrebu
Használati útmutató
Instrukcia užívania
Instrucțiuni de utilizare
Instruções de utilização
Informácie pre požívanie

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0123
4400768851/02

3M Deutschland GmbH
Dental Products
Carl-Schurz-Str. 1
41453 Neuss – Germany

3M, ESPE and Scotchbond are trademarks of 3M or
3M Deutschland GmbH. © 2014, 3M. All rights reserved.

en ENGLISH

Product Description
Scotchbond™ Universal Etchant is an etching gel for etching enamel and dentin using the acid etching technique. Scotchbond Universal Etchant contains 32% phosphoric acid by weight.

Indications

- Etching of enamel and dentin using the acid etching technique.

Precautionary Measures

Scotchbond Universal Etchant: Contact with eyes or skin may cause burns. Wear suitable protective clothing, gloves and eye/face protection. In case of contact, flush area with plenty of water and seek medical advice.

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- Follow the customary procedures to clean unprepared tooth structure (e.g., with pumice paste, polishing paste, or power blasting) and rinse thoroughly with a water spray.
- Cleaning pastes containing fluoride or oil should not be used because they can impair the effectiveness of the etching process.
- When working with prepared tooth structure, use a water spray to remove any loose preparation debris and dry the die or cavity with water- and oil-free air or with cotton pellets. Do not overdry.
- A dry area for treatment is a prerequisite for acid etching. We recommend using a rubber dam to ensure that the area remains adequately dry.

Application

- Remove the Scotchbond Universal Etchant syringe from the foil pouch, unscrew the cap, and attach an Etchant Dispensing Tip.
- Apply the etching gel to the precise area of the enamel and/or dentin and allow to react for 15 sec. Rinse thoroughly for 15 sec with water, using a suction at the same time.
- Dry with a cotton pellet; do not overdry.
- Depending on the adhesive system being used, air drying may not be recommendable. You will find precise instructions for drying in the instructions for use of the specific product.
- Avoid any contamination of the etched surfaces. If contamination occurs, repeat the etching, rinsing, and drying.
- Continue immediately with the bonding process.
- After application, remove the application tip from the syringe and replace the cap.

Hygiene

- Use common agents to clean Scotchbond Universal Etchant syringes and disinfect them in accordance with the commonly used hygienic regulations.

Disposal

The syringe must be completely emptied before being disposed of for recycling. Small amounts of Etching Gel can be disposed of with normal wastewater when well diluted with water or when neutralized.

Storage and Shelf Life

Store the product at 02°–25°C/36°–77°F.

Do not use after the expiration date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Information as of August 2012

RUS: Authorised representative of the manufacturer in RF:

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