### Application capsule

#### Product description:

**Meron AC** is a radiopaque glass ionomer cement for luting in application capsules for direct application. **Meron AC** is characterised by good rheological properties, a high level of compressive strength, fluoride release and a low film-thickness.

Each capsule contains a minimum of 0.19 ml of usable cement.

**Shades:** universal white.

#### Indications:

- Luting of metal-based crowns, bridges, inlays and onlays
- Luting of high-strength all-ceramic crowns and bridges (such as aluminium oxide, lithium disilicate and zirconium dioxide ceramics)
- Luting of metal-based and high-strength all-ceramic crowns and bridges (such as aluminium oxide, lithium disilicate and zirconium dioxide ceramics) on implant abutments
- Luting of metal root posts or indirect metal post core build-ups
- Luting of prefabricated steel crowns
- Luting of orthodontic bands

#### Contraindications:

**Meron AC** contains polyacrylic acid. **Meron AC** should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

#### Patient target group:

**Meron AC** is suitable for use in all patients without any age or gender restrictions.

#### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

#### User:

**Meron AC** should only be used by a professionally trained dental practitioner.

#### Use:

##### Preparation:

Pre-treatment of the restoration according to the manufacturer's instructions (e.g. sandblasting of the inside of metal restorations). If necessary, degrease using alcohol or acetone. A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp. Clean the prepared cores (e.g. with a cleaning paste), rinse with water and dry. Do not overdry the dentine. The surface should appear slightly moist.

##### Activating the application capsule and mixing the material:

Only extract the application capsule from the tubular bag immediately before use. Shake the application capsule briefly by hand before use, in order to loosen the powder. During activation, the liquid from the plunger must be pressed into the inside of the capsule as fully as possible. To achieve this, the end of the application capsule where the plunger of liquid protrudes is pressed down onto a hard surface (e.g. a table) as far as the plunger will move (see fig. 1). The capsule must only be activated directly before use and loaded immediately into a high-frequency mixer with at least 4,000 oscillations per minute. Mix the activated application capsule for 10 s, remove it from the mixer without delay and insert it in the holder of an application device (e.g. AC Applicator, see corresponding instructions for use). Working time and consistency may be influenced by the respective type of capsule mixer.

Fig. 1: Activating the application capsule

#### Application:

Carefully open the outlet by lifting the application tip until you feel a stop (see fig. 2). Place the opened application capsule into the holding bracket of an application instrument (e.g. AC Applicator, see respective instructions for use). The cannula can be aligned as required by turning the whole application capsule in the holder. By operating the lever, dispense the material directly onto the surfaces which have been prepared for luting and distribute it using, for example, the application tip of the application capsule. Always apply a slight excess of material. Insert the restoration immediately applying light pressure and hold it in position continuing to apply light pressure while curing takes place. Avoid unnecessary waiting times between activation, mixing and application, since the incipient setting may make dispensing and the positioning of the restoration more difficult.

Fig. 2: Opening the application capsule

At room temperature, **Meron AC** allows a window of approx. 3 minutes working time, before setting begins after 3 - 5 minutes. at 37 °C. Higher temperatures shorten and lower temperatures extend these working times. Prevent the ingress of water and saliva during application and setting phase.

Excess material can be removed particularly easily during the rubber-elastic phase (before complete curing) using a scaler or a curette, for example.

#### Warnings, precautionary measures:

- Avoid direct application to dentine near the pulp or to exposed pulp.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- **Meron AC** is not intended for non-reinforced all-ceramic restorations such as silicate ceramics, nor for indirect composite restorations, orthodontic brackets, ceramic inlays, ceramic onlays or veneers.
- Instruments used need to be rinsed immediately with cold water before residues of material cure. Following the removal of orthodontic bands, residues of material can be removed quickly and gently using an ultrasonic scaler.
- Ensure that the application capsule is mixed immediately after activation.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

#### Constituents (in descending order according to content):

Strontium aluminium borosilicate glass, water, polyacrylic acid, tartaric acid

#### Storage instructions and application method:

Store **Meron AC** in a dry place at temperatures of 15 °C to 23 °C. Only remove it from the foil immediately before use. Do not use after the expiry date.

#### Disposal:

Dispose of the product in accordance with local regulations.

#### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Meron AC** must be reported to VOCO GmbH and the responsible authority.

#### Note:

The Summary of Safety and Clinical Performance of **Meron** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at [www.voco.dental](http://www.voco.dental).

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VC 60 AA1242 E1 0524 99 © by VOCO