

Step-by-Step instruction



Assure that the seal of the package foil is intact prior to use. The components are intended for single use only.



Peel off the single dose blister foil carefully.



Lift the foil near the brush and remove the brush.



Peel back the remaining foil to expose the varnish.



Mix contents briefly to homogenize the contents.



Apply a thin, even layer to all affected surfaces of the dentition.



The single dose blister contains sufficient varnish to cover all affected teeth of both the upper and lower arch.



Do not rinse or suction immediately after use.



Do not brush or floss for at least four hours.