MD 550

Mundspülbecken-Reiniger
Spittoon bowl cleaner
Nettoyant des crachoirs
Pulente per sputacchiere
Limpiador de la escupidera

REF CCS550C4550

750 ml

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EN MD 550 Spittoon bowl cleaner

MD 550 is a special, ready-to-use, antibacterial preparation that cleans and cares for spittoon bowls of dental units. MD 550 removes limescale deposits, blood, residues of plaque staining tablets, impression materials, etc.

Composition: 100 g MD 550 contains 40 g Ethanol, < 1 g surfactants, < 5 g organic acids, auxiliary agents, water, benzyl salicylate.

Effective spectrum: Bactericidal, tuberculocidal, yeastcidal. Tested under low load in accordance with EN 13727, EN 13624, EN 14348 as well as EN 16615.

Directions for use: MD 550 is a ready-to-use solution. To open the safety cap, press the smooth surfaces together and turn. Squirt a little MD 550 into spittoon bowl and spread over the surface with a brush applying even pressure. Allow a few minutes for preparation to take effect, then rinse with water. The exposure time is 5 minutes (bactericidal / tuberculocidal / yeastcidal), (see table). Apply MD 550 as required, however, in the evening it must be done **before** disinfecting the suction system. For further instructions, please see the product information on the internet.

Caution: Flash point 23 °C. Take care with alcohol-sensitive materials (e.g. acrylic glass). Do not use foaming or abrasive disinfectants or cleaning agents on the cuspidor. Do not mix MD 550 with other products. Changes in the colour and odour of MD 550 during storage have no impact on efficacy. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

Warning. Flammable liquid and vapour. Causes serious eye irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear protective gloves and eye/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

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AT: DÜRR DENTAL Austria GmbH, Neurauf 4, 6170 Zirl, Austria, info.at@duerrdental.com
CH: CH-REP: Dürr Dental Schweiz AG, Grabenackerstrasse 27, 8156 Oberhasli, Switzerland, info.ch@duerrdental.com
FR: DÜRR DENTAL FRANCE S.A.R.L., 71 Rue des Hautes Pâtures, 92 000 Nanterre, France, info@duerrfr.com
BE / LU: DÜRR DENTAL BELGIUM B.V.B.A., Molenheidebaan 97, 3191 Hever, Belgium, info@duerr.be
IT: Dürr Dental Italia S.r.l., Via Remo Brambilla 16, 20863 Concorezzo (MB), Italy, italia@duerrdental.com