
- GB** Instructions for use
- GR** Οδηγίες Χρήσης
- CZ** Návod k použití
- HU** Használati utasítás
- LV** Lietošanas instrukcija
- PL** Instrukcja obsługi
- HR** Upute za uporabu
- RO** Instrucțiuni de utilizare
- TR** Kullanma talimatı
- RU** Инструкция по применению

6607800911

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**KULZER**  
MITSUI CHEMICALS GROUP

Manufacturer:  
Kulzer GmbH  
Leipziger Straße 2  
63450 Hanau (Germany)  
Made in Germany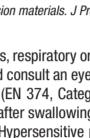

**Recommended model material**  
All commercially available modeling stones can be used.  
Our suggestion: Class II gips, MOLDANO®/MOLDADUR®  
Class IV gips, MOLDASTONE®/MOLDASYN®

**Dissolution**  
Impression should generally be treated by immersion for 10 minutes\* in 5.25% Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the products manufacturer.

\* Geleon Luis Adaro, DDS, PhD. Elvane Zanarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS. *PtD Effect of disinfectant agents on dimensional stability of elastomeric impression materials.* J Prosthet Dent, 81, 621-4,1999

**Flame test**  
Flame test. Do not incinerate labialization can cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rinse immediately with copious amounts of water and consult an eye specialist. Avoid skin contact. Wear protective glove/protective clothing/eye protection/glove protection/face protection EN 374, Category III/Typ B, e.g. Dermatril from KCL Germany. Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product.

According to EU Medical Device Regulation users/ patients are obliged to report occurred serious incidents with a medical device to its manufacturer and to the competent authority of the country, the incident occurred.

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