- GB** Instructions for use
- GR** Οδηγίες Χρήσης
- CZ** Návod k použití
- HU** Használati utasítás
- LV** Lietošanas instrukcija
- PL** Instrukcja obsługi
- HR** Upute za uporabu
- RO** Instrucțiuni de utilizare
- TR** Kullanma talimatı
- RU** Инструкція по примененію

oxasil®	EN ISO 4823	Colours	Dosage**		Mixing-time	Working-time†	Time in mouth
			BASE	Activator Paste			
Soft Putty	Type 0	yellow	1 spoon (10,7g/9,5 ml)	4-6 graduations (0,32g/0,29ml)	30 s	100	4:00
Putty	Type 0	yellow	1 spoon (14,4g/9,5 ml)	4-6 graduations (0,21g/0,19ml)	30 s	100	4:00
Mucosa	Type 2	orange	12 graduations (9,0g/7,6 ml)	12 graduations (0,65g/0,59ml)	30 s	130	4:00
Light Flow	Type 3	blue	12 graduations (9,0g/7,6 ml)	12 graduations (0,65g/0,59ml)	30 s	130	4:00
Very Light Flow	Type 3	blue	12 graduations (9,5g/7,6 ml)	12 graduations (0,65g/0,59ml)	30 s	130	4:00

*1 At room temperature of 23°C (73°F), 50% rel. humidity. Higher temperatures shorten, lower temperatures prolong these times.
 † 1 graduation = 1,3 cm (when using Kulzer mixing paste). The stated dosage should be observed.
 ** Type 0 = putty consistency
 Type 1 = heavy bodied consistency
 Type 2 = medium bodied consistency
 Type 3 = light bodied consistency

Special Advice
 Medical Device. For use by health care professionals only.
Product description
 Elastomeric, silicone-based precision impression material, condensation curing

Indication
 axial impression materials. The materials are for use in all standard impression techniques according to the professional's experience. E.g. use for fully and partly toothed jaws as well as for edentulous ones to generate impressions for total and partial dentures, reparations, provisional prosthesis, bite splints or the situation of the opposing jaw.

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