HySolate

Non Latex Dental Dam

Instructions for Use

Rx ONLY

COLTENE

Table of Contents

EN	Instructions for Use	1
DE	Gebrauchsanweisung.....	9
FR	Mode d'emploi.....	17
ES	Instrucciones de uso	25
BG	Инструкции за употреба	33
CS	Návod k použití.....	41
DA	Brugsanvisning.....	49
EL	Οδηγίες χρήσης.....	57
ET	Kasutusjuhend	65
HR	Upute za upotrebu	73
HU	Használati útmutató	81
IT	Istruzioni per l'uso	89
LT	Naudojimo instrukcijos.....	97
LV	Lietošanas instrukcija	105
MT	Istruzzjonijiet għall-Użu	113
NL	Gebruiksaanwijzing	121
PL	Instrukcja użytkia.....	129
PT	Instruções de utilização.....	137
RO	Instrucţiuni de utilizare	145
SK	Návod na použitie.....	153
SL	Navodila za uporabo	161
SV	Bruksanvisning	169
TR	Kullanma Talimatları.....	176

Intended Use/ Indications for Use:

Dental Dam is intended to be used intraorally to isolate one or more teeth and create a barrier to the surrounding area during a dental procedure.

Contraindications:

There are no contraindications.

Intended Users:

- Dental Practitioners
- Dental Assistants

Patient Target Group(s):

Suitable for all patient groups.

Clinical Benefits:

The use of Dental Dam has the following clinical benefits:

- Increased visibility through tooth/teeth isolation
- Reduced risk of the patient aspirating or swallowing instruments, tooth debris, medicaments or irrigation solutions
- Creation of a dry operating field
- Protection of soft tissue from contact with instruments
- Infection control barrier

Characteristics:

- Available in different colors and thicknesses
- Two sizes: 5" x 5" or 6" x 6" (127 mm x 127 mm or 152 mm x 152 mm)
- The lighter colors have the advantage of naturally illuminating the operating field whereas the darker colors can help with visual contrast
- Non-Latex

Storage:

Store Dental Dam between 32° F – 80° F (0°C – 26°C).

Keep dry and keep away from heat.

Residual Risks, Warnings and Precautions:

Refer to the Warnings and Precautions section for disclosure of risk associated with the use of the Dental Dam in line with its Intended Use.

Instructions

Note: If intending to use non-latex dental dam, always ensure the correct dam is chosen prior to use. To prevent mix-ups or cross contamination do not store near latex dental dam.

Dental Dam is intended to be used with various other devices including clamps and frames.

Refer to the instructions for use for the other devices being used.

The following general steps are provided as guidance and can be tailored to fit the specific case:

Step 1: Determine the Operating Field.

Step 2: Dental Dam Preparation.

- Determine hole positions
- Mark hole positions
- Punch holes

Step 3: Clamp/Retainer Selection

Step 4: Clamp/Retainer Application

Step 5: Dental Dam Placement

Parts List

Note: Not all Catalog Numbers are available in all regions

Catalog #	Description
H09105	HySolate Dental Dam Non-Latex Medium Green 6" x 6" (152 x 152 mm), 15 pcs
H09106	HySolate Dental Dam, Non-Latex, Medium, Green, 6" x 6" (152 x 152mm), 75 pcs.
H09928	HySolate Dental Dam Non-Latex Medium Green 5" x 5" (127 x 127 mm), 15 pcs
H09945	HySolate Flexi-Dam Non-Latex Purple 6" x 6" (152 x 152 mm), 30 pcs
H09946	HySolate Flexi-Dam Non-Latex Green 6" x 6" (152 x 152 mm), 30 pcs
H09947	HySolate Flexi-Dam Non-Latex Blue 6" x 6" (152 x 152 mm), 30 pcs
H00750	HySolate Flexi-Dam Framed Non-Latex Purple 3.91" x 4.12" (100 x 105 mm), 20 pcs
H00751**	HySolate Framed Flexi-Dam Sample Non-Latex Purple 6" x 6" (152 x 152 mm)

** Sample packs not available for order

Warnings and Precautions

Warnings	Statement
	Failure to properly follow the directions in this IFU may result in patient injury or compromise the integrity of the dental dam. Use the product only in accordance with its intended use, see Intended Use.
	Do not reuse Dental Dam. Dental Dam is intended to be used only once. Reuse of Dental Dam intraorally is a biological hazard and can result in cross contamination.
	Dental Dam should be inspected prior to use. Discard torn or brittle Dental Dam. Use of Dental Dam that has lost its performance characteristics can result in loss of functionality and compromise the integrity of the procedure.

Precautions	Statement
	Do not expose Dental Dam to high temperatures. Store Dental Dam away from heat sources.
	Dental Dam has a specified shelf life. Do not use the device after the Use-by Date because the device functionality and performance characteristics may be compromised.
	Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner
	Do not store near latex products.

Additional Information

Note: Dispose of device in accordance with local, regional, national, and/or international regulations. Devices used intraorally should be treated as a biohazard and disposed of properly per local regulations.

Note: Upon request, a hardcopy of the IFU will be supplied free of charge.

For users located in the EU: Any serious incident that has occurred in relation to the use of the device should be reported to Coltène/Whaledent, Inc. and the competent authority of the member state in which the user and/or patient is established.

Symbol Glossary

Symbol	Title and description of symbol
	Caution
 www.coltene.com	Consult electronic instructions for use - Indicates the need for the user to consult the instructions for use
	Medical Device – Indicates the item is a medical device
	Sale Restricted: The purchase of the device is restricted to licensed healthcare practitioners only.
	Use-by Date - Indicates the date after which the medical device is not to be used
	Catalog Number - Indicates the manufacturer's catalogue number so that the medical device can be identified
	Do not re-use - Indicates a medical device that is intended for use on a single patient during a single procedure
	CE mark

Symbol	Title and description of symbol
	Importer - Indicates the entity importing the medical device into the locale
	Manufacturer - Indicates the medical device manufacturer
	Batch code - Indicates the manufacturer's batch code so that the batch or lot can be identified
	Authorized representative in the European Community - Indicates the authorized representative in the European Community
	Not made with natural rubber latex

Storage

Store Dental Dam between 32° F – 80° F (0°C – 26°C).

Keep dry and keep away from heat.

Symbol	Title and description of symbol
	Keep dry - Indicates a medical device that needs to be protected from moisture
	Keep away from heat - Indicates a medical device that needs protection from heat
	Temperature Limit - Indicates the temperature limits to which the medical device can be safely exposed. The upper and lower limits of temperature shall be indicated adjacent to the upper and lower horizontal lines.

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Dent4You AG
Coltene/Whaledent Inc.
Coltene/Whaledent Ltd.
Coltene/Whaledent GmbH + Co. KG