



Conform Fit™ Gutta-Percha

The right fit and flow you can feel





Next-generation gutta-percha. Made possible by Conform Fit[™] technology.

Gutta-percha master cones should ideally fit the shapes created by your files. Yet, traditional hand-rolled gutta-percha points have made it hard to achieve a precise match. The next level of precision performance is here — gutta-percha master cones with Conform Fit technology.



The right fit you can feel. Variable tapered cones provide a snug fit that reaches the apex for the right tug back.



Improved flow to the end. Conform Fit gutta-percha has a better flow and transfers heat up to 4 mm beyond the heat source.



Enhanced patient safety. Master cones are not made with natural rubber latex.



The simplicity of a total solution. Precisionmatched shapes and sizes remove the guesswork.



Can be used at lower working temperature. Lower temperatures reduce the risk of harm to the periodontal ligament (PDL).




Convenient tab and color markings for easy identification.




The right fit you can feel

You need a consistent and confident cone fit. But finding the right tug back with hand-rolled gutta-percha points can be tough. Traditional gutta-percha may seem to fit but can actually be loose at the apex. Conform Fit[™] gutta-percha cones feature a variable taper with a 40% more consist shape than traditional gutta-percha. This creates a snug apical fit and an accurate tug back that you can feel.

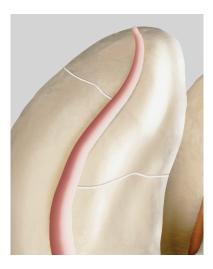





The unique gutta-percha formulation heats more consistently for better flow and allows deeper thermal penetration into the material. Heat transfers through the master cone up to 4 mm beyond the heat source—so you can be sure you have warm, flowable gutta-percha all the way to the apex. This enables the clinician to utilize a lower heat plugger temperature, thereby improving patient safety and achieving a complete fill.

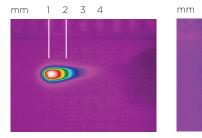


wv:05:1 seconds

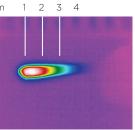


:08:2 seconds

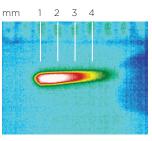

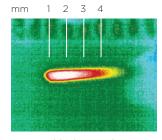

:014:2 seconds

:024:0 seconds

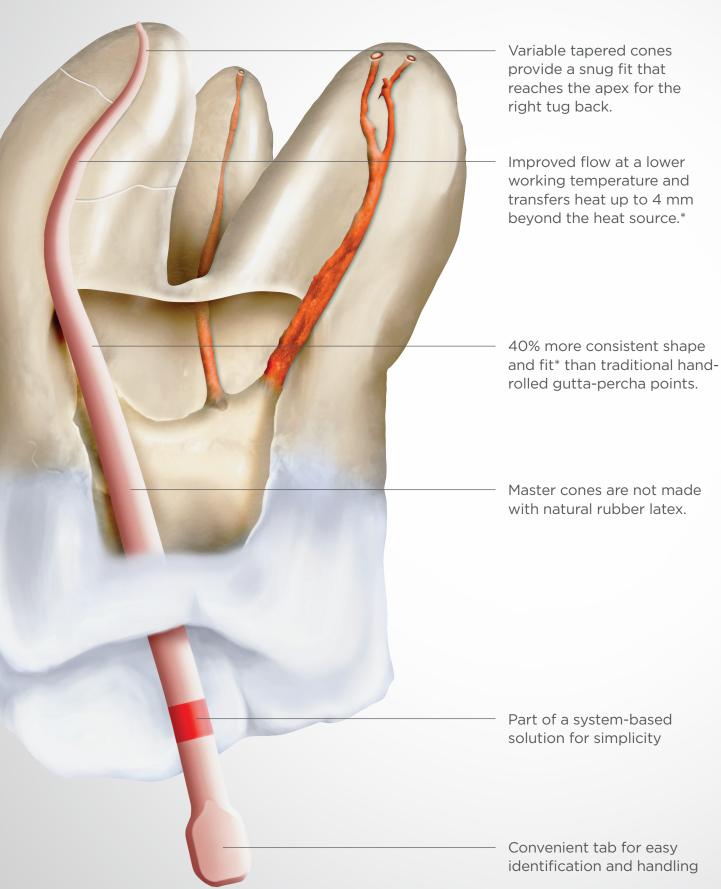

*Source: Internal testing. Data available.



Conform Fit[™] gutta-percha is not made with natural rubber latex. Our advanced formulation can be used at a lower working temperature, reducing the risk of harm to the periodontal ligament (PDL), which can be damaged by higher heat.



The simplicity of a total solution

Gutta-percha master cones with Conform Fit technology are part of a total system solution that takes the guesswork out of achieving a precise fit. Use them with corresponding files, gutta-percha points and paper points with confidence. The innovative tab design simplifies identification.





MAILLEFER

Conform Fit™ Gutta-Percha

Part of a total system solution

WaveO	ne® Gold (Conform F	it™ Gutta-Perch	а
Color	Size	Pack	ltem #	

00101	0120	I GIGIT	iterit in
Yellow	Small	60-pack	BOOWGGPFOOSML
Red	Primary	60-pack	BOOWGGPFOOPRI
Green	Medium	60-pack	BOOWGGPFOOMED
🗖 White	Large	60-pack	BOOWGGPFOOLRG
Assorted	S/P/M/L	60-pack	BOOWGGPFOOAST



ProTaper Gold[®] Conform Fit[™] Gutta-Percha Color Size Pack Item # Yellow F1 60-pack B00PGGPF000F1 F2 B00PGGPF000F2 Red 60-pack F3 Blue B00PGGPF000F3 60-pack F2/F3 BOOPGGPF00F23 60-pack Assorted F4/F5 BOOPGGPF00F45 Assorted 60-pack



ProTaper Next® Conform Fit™ Gutta-Percha						
Color	Size	Pack	ltem #			
■ Red ■ Blue Assorted Assorted	X2 X3 X2/X3 X4/X5	60-pack 60-pack 60-pack 60-pack	BOOPNGPFOOOX2 BOOPNGPFOOOX3 BOOPNGPFOOX23 BOOPNGPFOOX45			







Maillefer Instruments Holding Sàrl Chemin du Verger 3 CH-1338 Ballaigues Switzerland Made in the USA dentsplysirona.com All images are the property of Dentsply Sirona, Inc. unless otherwise noted.

