For use only by a dental professional in the recommended indications.
EN

COMPOSITE PRIMER

PRIMER FOR MICRO-FILLED COMPOSITE AND ACRYLIC RESIN

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Surface treatment of micro-filled resin (MFR) or micro-filled hybrid resin (GC GRADIA, GC GRADIA gum shades) prior to luting or repairing procedures.
- Surface treatment of acrylic resin prior to repairing procedure.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

A. Surface treatment of the restoration prior to luting procedure

- PREPARATION
Prepare the surface to be bonded to using a sandblaster (aluminum oxide) with a pressure of at least 3kg/cm² (2.94bar). Refer to the manufacturer's instructions for use. Cleanwith oil-free air.
Note:
Apply masking tape to the surfaces where no treatment is needed.
- APPLICATION
Dispense one or two drops of COMPOSITE PRIMER into a dispensing dish. Apply a thin layer to the surfaces to be bonded to using a brush.
Note:
1) Use immediately after dispensing. Replace the bottle cap immediately after use.
2) Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.
- LIGHT CURING
Light cure using a visible light curing unit (470 nm wavelength). Refer to the light manufacturer's instructions for use.
After light curing, do not remove the oxygen inhibited outer layer.
Halogen/LED : 20 seconds
G-Light : 10 seconds
Plasma arc : 3 seconds
- CEMENTATION
Apply a suitable luting cement on the treated surfaces of the restoration and continue with the luting procedure. Refer to the manufacturer's instructions for use.

B. Surface treatment of the restoration prior to repairing procedure

- PREPARATION
Prepare the surface to be bonded to using a diamond bur or sandblaster (aluminum oxide) with a pressure of at least 3kg/cm² (2.94 bar). Take the necessary safety measures for the intraoral use of the sandblaster. Refer to the manufacturer's instructions for use. Clean with oil-free air.
Note:
Apply masking tape to the surfaces where no treatment is needed.
- APPLICATION
Dispense one or two drops of COMPOSITE PRIMER into a dispensing dish. Apply a thin layer to the surface to be repaired using a brush.
Note:
1) Use immediately after dispensing. Replace the bottle cap immediately after use.
2) Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.
- LIGHT CURING
Refer to A.3. above. When using GC LABOLIGHT LV-III, light cure for 1 minute.
- REPAIR
Apply composite material such as GC GRADIA, GC GRADIA DIRECT, GC KALORE or GC GRADIA gum shades on the treated surfaces of the restoration and start to finish. Refer to the manufacturer's instructions for use.

STORAGE

Store in a cool, dark place (4-25°C) (39.2-77.0°F).
(Shelf life: 2 years from date of manufacture)

PACKAGE

Bottle of 3mL

CAUTION

- COMPOSITE PRIMER is flammable. Do not use near naked flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
- In case of contact with oral tissues, immediately remove with a sponge or cotton pellet. Flush with water after treatment is completed.
- In case of contact with skin, immediately remove with a sponge or cotton pellet. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- Do not mix with any other primers. Do not reuse a brush with other products.

Last revised : 06/2013

MANUFACTURED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenaan 33,
B-3001 Leuven, Belgium TEL +32 16 74 10 00

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