# Geti

## Ultrasonic cleaner

### B series

## User manual


Please read the operating instructions carefully before using the machine.  
Pay special attention to all warnings and safety instructions.  
Keep this manual for later reference.

## Features

- Stainless steel housing, tank and lid
- Ceramic heaters for better heating effect
- Stamp tank without welding gap for better waterproof.
- Cooling fan
- Side handles for easy handling

## Introduction

Thank you for purchasing the ultrasonic cleaner. Please take the time to read these operating instructions before use and retain them for future reference. Failure to follow these instructions may lead to serious personal injury and damage to property.

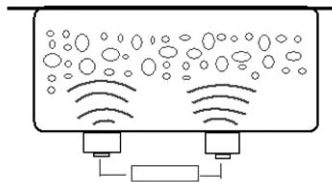Ultrasonic cleaning is based on the cavitation effect caused by high frequency ultrasonic wave vibration signal in the fluid. Microscopic bubbles are formed, and they implode violently causing the cavitation which create an intense scrubbing action on the surface of the item being cleaned. The bubbles are small enough to penetrate microscopic crevices, cleaning them thoroughly and consistently. Ultrasonic cleaning is extremely effective at removing dirt and grime which would normally require tedious manual cleaning by hand. It has been used to clean a wide variety of instruments and mechanical parts such as carburetors, returning them to almost "like new" condition without damage to delicate parts.

## Safety precautions

Appliances can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance.

## Preparation

1. Carefully unpack the cleaner and remove all traces of packing materials from it. Visually inspect the cleaner for any parts that may have become loose or damaged during transit.

### Contents:

- |                  |                           |               |
|------------------|---------------------------|---------------|
| a) Main machine  | b) Sound proof lid        | c) Power lead |
| d) Outlet filter | e) Mesh basket (optional) |               |

2. Place the cleaner on a flat, clean surface where the cooling fans will receive adequate ventilation and ensure all controls are set to off. Also ensure the drain tap is in the closed position.
3. Ensure the power lead is securely plugged into the cleaner and that no part of the lead is likely to come into contact with moisture.
4. Carefully fill at least 1/2 of the tank with a solvent solution. For demanding cleaning we recommend a small amount of washing up liquid, this will help increase the cleaning performance. The cleaner is now ready for use.

### Attention

- a) While the machine is working normally, ultrasonic and tank syntonny gives a well-proportioned sound, and no shudder on the surface of the water, yet there is spray made by the tiny bubbles. If there are discontinuous surges, please add or release a little of washing solution in the tank, stopping the surges is better for the objects cleaning.

- b) On the condition of making sure the cleaning object is cleaned, please run the machine discontinuously as far as possible(not more than 30min.), for long-time running will cause temperature increment of the case.

PLEASE READ THE FOLLOWING VERY CAREFULLY AS FAILURE TO COMPLY MAY INVALIDATE YOUR GUARANTEE

- 1) DO NOT run the cleaner continuously for more than one hour at a time, as doing so can damage the internal components.
- 2) DO NOT operate the unit without fluid in the tank. Always ensure the fluid is no higher than the max mark. Always ensure there is a minimum depth of 7cm.
- 3) DO NOT drop any item into the tank as this may cause damage to the transducer. Always place the items gently into the tank and use the basket whenever possible.
- 4) The more items that you place in your cleaning bath the less efficient it will clean. It is not advised to overlap items. Always allow plenty of clear space between the items.
- 5) Do keep the lid on during use. This will prevent splashes and reduce evaporation of the fluid.
- 6) Never immerse the machine or power cord in water or other liquid.
- 7) DO NOT touch the power plug with wet hands, especially when inserting or removing the plug.
- 8) DO NOT touch the unit if the machine has fallen into water during operation. Remove the power plug from the socket first.
- 9) DO NOT disassemble the machine, except by professionals.
- 10) UNPLUG the power source while filling or emptying the tank.
- 11) DO NOT spray water or liquid over the device and the control panel.
- 12) DO NOT operate the cleaner without proper grounding.
- 13) DO NOT place the device on a soft surface, where the vents could be blocked.
- 14) Always turn the heater off after using as leaving it on can cause the fluid to evaporate and damage the internal components.
- 15) Upon completion of the cleaning cycle, turn the heater button off and isolate the machine from the electrical supply.
- 16) Take care when adding or removing items from the cleaning tank as the fluid is likely to be hot and displaced fluid can damage the internal components. Any displaced fluid must be dried up immediately.
- 17) In the event of failure/emergency, disconnect the mains supply by removing the plug from the mains socket.

### Application ranges


DOMESTIC	
Jewelry	Necklaces, rings, bracelets, earrings etc.
Watches and glasses	Waterproof wristwatches, watch chains, glasses, sunglasses etc.
Toiletry items	Electric toothbrush heads, electric shaver heads, razor blades, combs, toothbrushes, dentures etc.
Discs	CDs, DVDs, DVD's, CD-R discs, laserdisc etc.

Office equipment	Printed heads, fountain pen nibs, wax or rubber stamps, Calligraphic pens/ nibs etc.
Metal items	Metal cutlery (knives/forks etc),old coins, metal badges, valves, machine nozzles, small metal parts etc.

<b>COMMERCIAL / INDUSTRIAL</b>
Airbrushes / Spray guns / Inkjet printer cartridges
Automotive / Aviation components/Engine cylinder heads
Motorcycle radiators
Dental & surgical instruments
Lenses & other optical components
Laboratory glassware & apparatus

**Digital display ultrasonic cleaner structure**

**Front view**

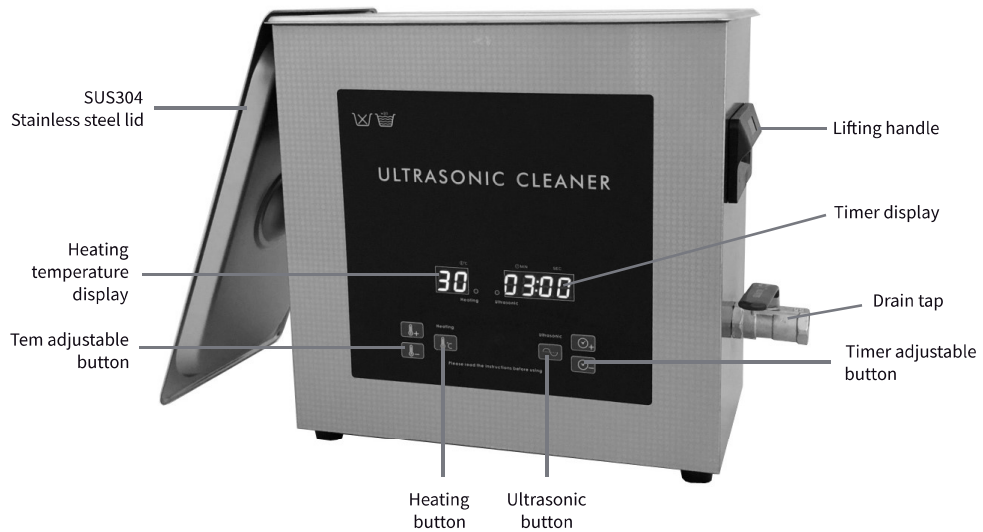## Back of the machine

## Operation

### 1. Start up ultrasonic function

- 1) Fill stainless steel tank with water.
- 2) Plug the cleaner into grounded outlet; When connected, the temperature defaults to be 25°C and timer defaults to be 3minutes.

- 3) Press the key   for shortly at a time means time increase/reduce 1minut at a time, press and hold means time will increase/reduce by 10minutes continuously. During working, you will hear the "sizzling" voice ,that means the cleaner running properly.

### 2. Start up heating function

- 1) Set temperature (if have): Press key  , shortly at a time means time increase/reduce 1°C at a time, press and hold there means temperature will increase/reduce by 10°C continuously. When actual temperature is higher than the setting time, The operation is invalid. The indicator light will be off when the temperature reach the setting temperature. The LED panel display the actual temperature when ultrasonic is working.

**After time and temperature setting, you can choose the following three cleaning modes.**

- 1) **SOFT mode** (if have): half ultrasonic power for objects not very dirty.
- 2) **Normal mode**: Whole power for much dirty objects, better cleaning effect on high-precision work pieces.
- 3) **Degas mode** (if have): Work intermittently, 6s on +2s off working mode. Turn on the function to remove air bubble from solution for better cleaning effect.

After cleaning, Empty the tank and clean both the outside and inside of the cleaner with clean and dry cloth for next use. **Note:** Do not pour water out until it's cooling, Hot water will hurt you and damage the machine itself.

### Different ways of cleaning

**General cleaning** - use only tap water. Use warm temperature about 50°C .

**Enhanced cleaning** - add few drops of standard washing up liquid, liquid soap, or detergent into water, other non-acidic cleaning agents can also be used.

**Extensive cleaning** - removing tarnish, carbon & rust from non-plated metals, it is recommended to use specialist cleaning solution in associated using ultrasonic cleaner.

### Specifications

Model	Tank size	Overall size	Capacity	Power	Frequency	Heating power	Time setting	Heating
	(L×W×H)mm	(L×W×H)mm	L	W	kHz	W	Min	°C
GUC 02B	150×140×100	190×170×230	2	60	40	100	1-99	0-80
GUC 03B	240×140×100	270×170×240	3	120	40	100	1-99	0-80
GUC 04B	300×155×100	312×162×240	4	120	40	100	1-99	0-80
GUC 06B	300×155×150	330×180×330	6	180	40	300	1-99	0-80
GUC 10B	300×240×150	330×270×330	10	240	40	300	1-99	0-80
GUC 13B	330×300×150	360×330×330	13	360	40	500	1-99	0-80
GUC 20B	500×300×150	550×330×330	20	480	40	500	1-99	0-80
GUC 27B	500×300×200	550×330×360	27	600	40	500	1-99	0-80

Do not dispose of the product at the end of its useful life together with municipal waste. Bring it to a collection point for recycling of electrical and electronic equipment. Information on the correct disposal method is given by the symbol on the product, in the operating instructions or on the packaging. Recycle materials according to their designation. Recycling materials or other forms of reuse of old appliances makes a significant contribution to protecting the environment. For information on authorized recycling centers, contact your local authority.

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