При use only by a dental professional in the recommended indications.

OPTIGLAZE

NANO FILLED LIGHT CURED PROTECTIVE COATING FOR COMPOSITE AND ACRYLIC INDIRECT RESTORATIONS

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

- Coating of indirect composite surfaces (fissures and interproximal areas).
- Coating of denture base surfaces
- Coating of provisional materials

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to acrylate or methacrylic monomer.

PRECAUTIONS

- It is not recommended to use OPTIGLAZE on high stress bearing areas (occlusal surfaces).
- The use of OPTIGLAZE does not replace the mechanical polishing of aesthetic indirect composite restorations such as GRADIA.

DIRECTIONS FOR USE

Preparation

After contouring the resin surface, finish with carbide bur or coarse silicon carbide paper. Clean the surface with a non-abrasive cleaner and then dry. If surface still contains oil residue, the surface can be further cleaned with ethanol (alcohol).

- NOTE:
- OPTIGLAZE does not adhere to a polished surface. Make sure surfaces to be coated are dry.
 - The air-curing surface to be coated should be removed before applying OPTIGLAZE as it may inhibit the proper curing of OPTIGLAZE.
 - The use of ethanol (alcohol) on acrylic resin may cause micro-cracks. A mild detergent can be used to clean the acrylic resin.

Application

- Dispense a few drops into a dispensing dish. Replace bottle cap immediately after dispensing.
- IMMEDIATELY apply (within 1 minute after dispensing) to the surface with a brush. DO NOT AIR BLOW.

Light Curing

Light cure with a suitable light curing device.	
LABOLIGHT DUO	90 сек.
LABOLIGHT LV-III, II	3 min.
STEPLIGHT SL-I	10 сек.
Distance from light source <3cm	20 сек.
Distance from light source >3cm	40 сек.

Halogen Light	20 сек.
Plasma Arc	8 сек.
LED Light (with light source having wavelength lower than 430nm)	40 сек.

NOTE:

- If surface is tacky, repeat light curing.
- If necessary, rotate the bottle at a half-turn or turn-over and light cure again to ensure all surfaces are completely cured.
- Additional build-up cannot be performed on surfaces coated with OPTIGLAZE. Remove the coated layer with a bur and then build-up.
- OPTIGLAZE cannot be cured by a conventional LED light curing device with blue LED.
- When using other manufacturers' light curing devices, use final curing mode. Refer to the respective manufacturers' instructions for use.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39-2-77.0°F). Shelf life: 3 years from date of manufacture.

PACKAGE

OPTIGLAZE 15mL (1)

CAUTION

- OPTIGLAZE is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
- OPTIGLAZE is volatile. Use in a well ventilated area. Replace cap immediately after dispensing.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of contact with skin, immediately remove with a sponge or cotton pellet. Flush with water.
- Wear protective gloves. In particular, avoid contact of uncured material with skin.
- Avoid getting material on clothing.
- Avoid inhalation or ingestion of material.
- Do not mix with other products.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity to some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

http://www.gceurope.com
They can also be obtained from your supplier.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration.

DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Last revised : 03/2017

0086

IGC

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasuga, Aichi 466-0844, Japan
DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Tabahi-shi, Tokyo 174-8585, Japan

EU: GC EUROPE NV
Researchpark Haasrode-Leuven 1240, Interleuvenaan 33,
B-3001 Leuven, Belgium TEL: +32 36 74 10 00
PRINTED IN JAPAN

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