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2. Intended Use

The AERObag® – Manual resuscitators are intended for single patient use for pulmonary resuscitation. The range of application for each version is:

- Adults (HBB17-E): Adults and children with a body weight of above 33 kg
- Pediatric (HBB17-K): Children and infants with a body weight from 10 kg to 33 kg
- Infants (HBB17-B): Infants and neonates with a body weight up to 10 kg

3. Indications

The resuscitator can be used for continued oxygen supply and assisted ventilation until a more definitive airway can be established or until the patient has recovered.

4. Contraindications

None known.

5. Intended User

This product must be used by personnel thoroughly trained in the techniques of pulmonary resuscitation.

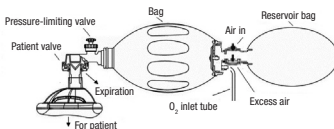
6. Warnings and Cautions

- Always visually inspect the product and perform a functionality test after unpacking, assembly and prior to use.
- Do not use the resuscitator if the functionality test fails.
- Do not use the product if contaminated by external sources, as this can cause infection.
- Do not use the product in toxic or hazardous atmospheres.
- Do not reuse the resuscitator if visible moisture or residues are left inside the device as this can cause infection and / or malfunction of the device.
- Designed for single patient use only. Use on multiple patients can cause cross infection.
- Do not smoke when working with supplemental oxygen. Do not use this device near open flames or other flammable substances. Fire may result.
- Oil or grease should not be used in close proximity to oxygen equipment as fire may result.
- Verify proper valve function. Verify that the patient is being ventilated by observing rise and fall of the patient's chest and color of lips and face during the resuscitation.
- High ventilation pressures may cause lung rupture or stomach distension on certain patients.
- Oxygen tubing (AERObag® – SS31-2) contains phthalates ($\checkmark_{CMR}$). Take special care with: Neonates, Children before puberty, Breastfeeding, pregnant women.

7. Setting up of the resuscitator and functionality test

7.1 Setting up the resuscitator

Prior to using the resuscitator, perform a full functionality test on the product. If the products are packed in a protective pouch, remove it before use and unpack the products. Visually inspect proper valve action while squeezing the resuscitator. Connect the patient valve into the outlet of the mask. If resuscitating with higher oxygen concentration, extend the oxygen tube and attach it to an oxygen nozzle adapter connected to a proper oxygen source. Connect the reservoir bag. Set the oxygen flow not to exceed 15 LPM or differently on the order of a physician.

7.2 Functionality test

- Close the pressure limiting valve and the patient connector with a thumb while compressing the resuscitator firmly to check the tightness of the valve, as well as proper fitting. The resuscitator shall offer resistance.
- Open the pressure limiting valve and close the patient connector with a thumb. The pressure limiting valve should now be activated. If operating as intended, it should be possible to hear the expiratory flow from the valve.
- Reservoir bag: supply a gas flow of 3.0 liter per minute to the oxygen bag. Check that the reservoir bag fills. If not, check the integrity of the two valve shutters, or for a torn reservoir bag or a blocked oxygen tubing.

7.3 Instructions for use

- Open the patients' mouth and clean the airway of all foreign matter and fluids by using recommended techniques. The use of an emergency aspirator is recommended. Tilt the head fully backwards and push the jaw upwards with the neck stretched to open the airway.

- 7.3.2 To assist ventilation, it may be beneficial to insert an artificial airway. Be careful that it does not push back the tongue and thus obstruct the throat.
- 7.3.3 Hold the mask tightly to the patients face, covering mouth and nose of the patient. Tilt the head fully backwards. Lift the jaw with the mask-holding hand. Squeeze the bag smartly and watch the chest expand.
- 7.3.4 Release the pressure on the bag by removing the hand holding it abruptly and allow the chest of the patient to deflate. Repeat this action based on recommended techniques. If continued resistance to insufflation is encountered, check the airway for possible obstruction and re-position the patient to ensure open airway. If adequate ventilation is not achieved with the resuscitator, immediately revert to air ventilation (mouth-to-mouth or mouth-to-nose).
- 7.3.5 The correct ventilation frequency may vary. Please follow the current ventilation frequency recommended by national or international guidelines.
- 7.3.6 If the patient vomits during mask ventilation, immediately clear the patients' airways of vomit by using recommended techniques, then freely compress the bag a few times before resuming ventilation.

8. Specifications

Oxygen Concentrations					
	Infant	Pediatric		Adult	
Oxygen (L/min)	4	10	3	5	10
Breathing Frequency (1/min)	30	20	12	12	12
Inflation Volume (ml)	40	250	500	500	500
With Reservoir (%)	98	98	60	86	98
Without Reservoir (%)	85	70	34	47	66

Performance Index			
	Infant	Pediatric	Adult
Body weight	≤10 kg	10 kg ~ 33 kg	≥33 kg
Stroke volume	150 ml	350 ml	800 ml
Resuscitator volume	340 ml	680 ml	1630 ml
Dimensions (Length x Diameter)	225 x 85 mm	262 x 105 mm	325 x 132 mm
Resuscitator weight w/o accessory	149 g	180 g	260 g
Pressure limiting valve	40 cmH ₂ O	40 cmH ₂ O	60 cmH ₂ O
Dead space (patient valve)	7 ml	7 ml	7 ml
Inspiratory resistance		<5 cmH ₂ O (at 50 L/min)	
Expiratory resistance		<5 cmH ₂ O (at 50 L/min)	
Bag reservoir volume	600 ml	1600 ml	2000 ml
Patient connector		ISO Ø 22/15 mm	
Recommended Operating temperature		-18 °C ~ +50 °C	
Storage		-40 °C ~ +60 °C	

The manual resuscitators conform to ISO 10651-4, as well as the Council Directive MDD/93/42/EEC concerning medical devices.

9. After use

Used products must be disposed of according to local procedures.

10. Components and accessories

10.1 Disposable PVC Face Mask HBM16-0V – HBM16-5V

The disposable PVC face mask is to be used as an accessory for the SEBS manual resuscitators, intended for oxygenating and ventilating the patients' airways.

- 10.1.1 Remove the PVC face mask from its packaging.
- 10.1.2 Inspect the cushion for proper inflation and for any damage or leakage. If necessary, use a syringe to inflate or remove air from the cushion to achieve the desired pressure.
- 10.1.3 Tightly apply the face mask to the patients face. Check that the joints are tight with no leakage.
- 10.1.4 If necessary, the surface of the mask can be manually cleaned with water and a mild detergent. Rinse thoroughly with clean water. Allow to dry and check that the mask is fully operational before using it again on the same patient. The product is designed for a single patient to avoid cross infection. If the product is not fully operational it shall be discarded.

10.2 PEEP Valve HPV16-20 and PEEP Valve Diverter HBB16-DV

The PEEP (Positive End Expiratory Pressure) valve is intended to be used as an accessory for SEBS manual resuscitators. This product helps prevent alveolar collapse and improves oxygenation by maintaining a certain positive end expiratory pressure. The peep valve is used when a pressure above the patients' lungs is needed in order to improve oxygenation.

- 10.2.1 Remove the PEEP valve and the diverter from its packaging. Make sure it's free of obstructions.
- 10.2.2 Adjust the top knob to the desired pressure value from 5 to 20 cmH₂O (0.5 - 2.0 kPa).
- 10.2.3 Connect the PEEP valve using the diverter to the patient valve interface of the SEBS manual resuscitator.

If, during the use of this device and its accessories or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authorities.

Katalogové číslo: TW8211-E | TW8211-E... | TW8221-K | TW8221-K... | TW8231-B | TW8231-B...

Objednací číslo: HBB17-E | HBB17-E...R | HBB17-K | HBB17-K...R | HBB17-B | HBB17-B...R

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Ltd., Area A, 6F, No. 588-1,
Holding Corp. GmbH
Edunex GmbH